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TIERWAAGE

USER MANUAL

LARGE VETERINARY SCALE

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BILANCIA DA PAVIMENTO

MANUAL DE INSTRUCCIONES

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PADLÓMÉRLEG

BRUGSANVISNING

DYREVÆGT

DE | EN | PL | CZ | FR | IT | ES | HU | DA

SBS-PS-150W

DE	Produktname:	Tierwaage
EN	Product name:	Large Veterinary Scale
PL	Nazwa produktu:	Waga platformowa
CZ	Název výrobku	Plošinová váha
FR	Nom du produit:	Balance vétérinaire
IT	Nome del prodotto:	Bilancia da pavimento
ES	Nombre del producto:	Báscula de suelo
HU	Termék neve	Padlómérleg
DA	Produktnavn	Dyrevægt
DE	Modell:	SBS-PS-150W
EN	Product model:	
PL	Model produktu:	
CZ	Model výrobku	
FR	Modèle:	
IT	Modello:	
ES	Modelo:	
HU	Modell	
DA	Model	
DE	Hersteller	expondo Polska sp. z o.o. sp. k.
EN	Manufacturer	
PL	Producent	
CZ	Výrobce	
FR	Fabricant	
IT	Produttore	
ES	Fabricante	
HU	Termelő	
DA	Producent	
DE	Anschrift des Herstellers	ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU
EN	Manufacturer Address	
PL	Adres producenta	
CZ	Adresa výrobce	
FR	Adresse du fabricant	
IT	Indirizzo del produttore	
ES	Dirección del fabricante	
HU	A gyártó címe	
DA	Producentens adresse	

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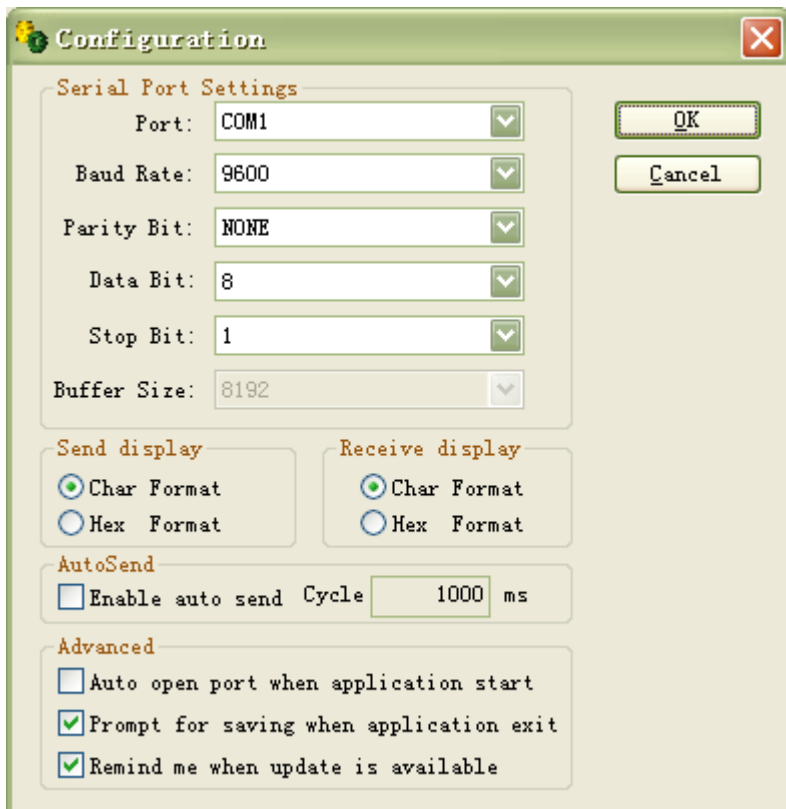
A configuration dialog box titled "Configuration" with a green header bar and a red close button in the top right corner. The dialog is divided into several sections. The "Serial Port Settings" section contains dropdown menus for Port (COM1), Baud Rate (9600), Parity Bit (NONE), Data Bit (8), and Stop Bit (1), along with a Buffer Size field (8192). The "Send display" and "Receive display" sections each have radio buttons for "Char Format" (selected) and "Hex Format". The "AutoSend" section has a checkbox for "Enable auto send Cycle" (unchecked) and a text field for "1000 ms". The "Advanced" section has three checkboxes: "Auto open port when application start" (unchecked), "Prompt for saving when application exit" (checked), and "Remind me when update is available" (checked). "OK" and "Cancel" buttons are located on the right side.

Configuration

Serial Port Settings

Port: COM1

Baud Rate: 9600

Parity Bit: NONE

Data Bit: 8

Stop Bit: 1

Buffer Size: 8192

Send display

☒ Char Format

☐ Hex Format

Receive display

☒ Char Format

☐ Hex Format

AutoSend

☐ Enable auto send Cycle 1000 ms

Advanced

☐ Auto open port when application start

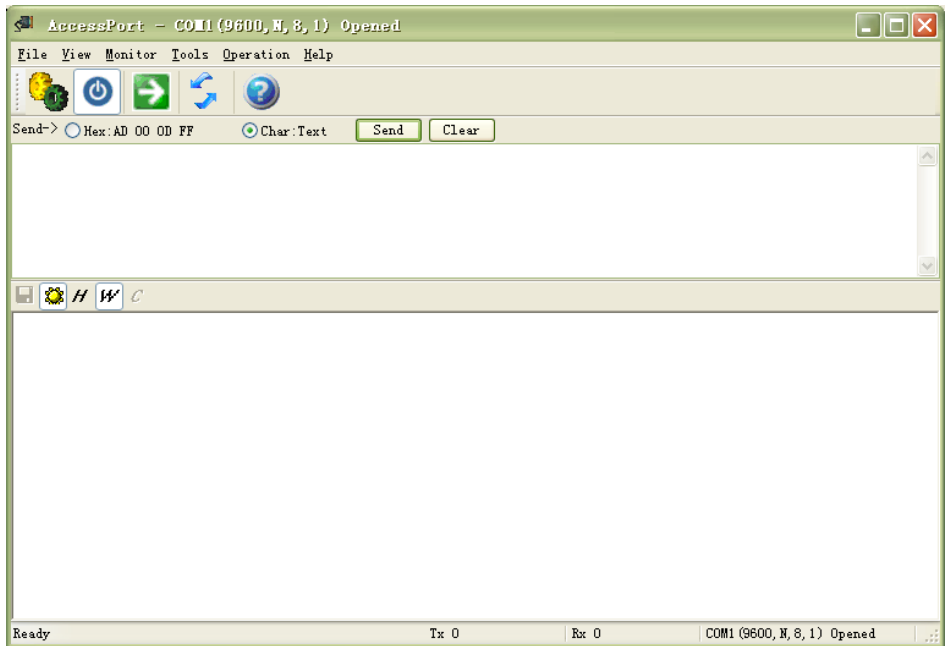
☒ Prompt for saving when application exit

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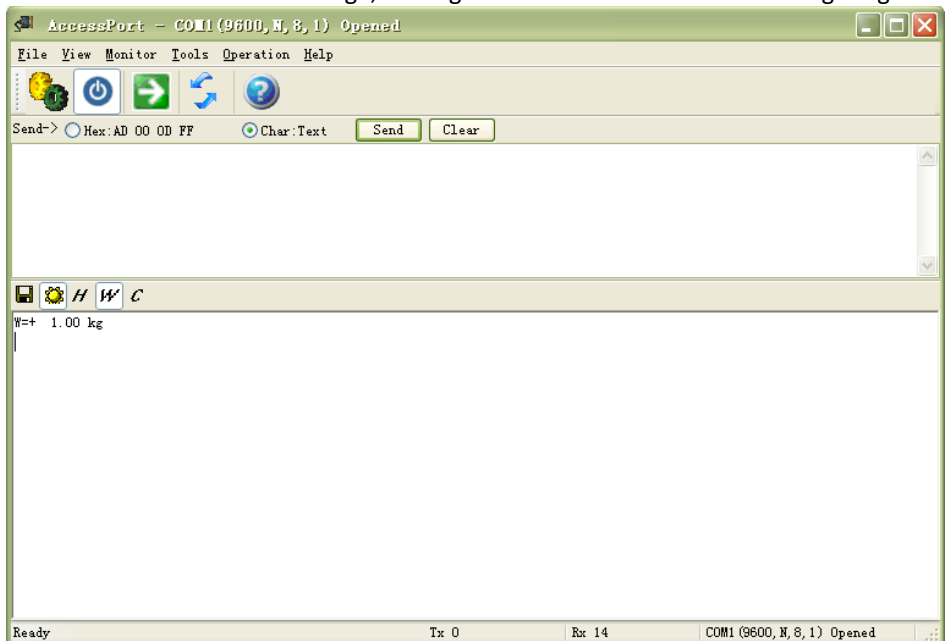
This User Manual has been translated for your convenience using machine translation. Reasonable efforts have been made to provide an accurate translation; however, no automated translation is perfect nor is it intended to replace human translators. The official User Manual is the English version. Any discrepancies or differences created in the translation are not binding and have no legal effect for compliance or enforcement purposes. If any questions arise related to the accuracy of the information contained in the User Manual, please refer to the English version of those contents which is the official version.

Technical data

Parameter description	Parameter value
Product name	Large veterinary scale
Model	SBS-PS-150W
Batteries	6 x AAA
Power supply	9 V DC / 300 mA
Maximum load	150 kg
Minimum load	1 kg
Breakdown	50 g

1. General Description

This manual is intended to assist you for safe and reliable use. The product is designed and manufactured strictly according to technical specifications using the latest technology and components and maintaining the highest quality standards.

**CAREFULLY READ AND UNDERSTAND THIS MANUAL
BEFORE STARTING THE WORK.**

To ensure long and reliable operation of the device, make sure to operate and maintain it properly in accordance with the guidelines in this instruction manual. The technical data and specifications in this manual are up-to-date. The manufacturer reserves the right to make changes in order to improve the quality. Taking the technical progress and the possibility of reducing noise into account, the unit is designed and built in such a way so that risks resulting from noise emissions are reduced to the lowest possible level.

Explanation of symbols

	The product complies with applicable safety standards.
	Read the manual before use.
	Recyclable product.
	CAUTION! or WARNING! or REMEMBER! describing a given situation (general warning sign).
	CAUTION! Risk of electric shock!

CAUTION! The figures in this manual are illustrative only and may vary in some details from the actual appearance of the product.

2. Safety of use

CAUTION! Read all safety warnings and instructions. Failure to follow the warnings and instructions may result in electric shock, fire and/or serious injury or death.

The term "device" or "product" in the warnings and in the description of the instructions refers to
Large veterinary scale.

2.1. Electrical safety

Applies to the battery charger:

- The plug of this device must fit into the outlet. Do not modify the plug in any way. Original plugs and matching outlets reduce the risk of electric shock.
- Avoid touching grounded parts, such as pipes, heaters, ovens, and refrigerators. There is an increased risk of electric shock if your body is grounded and touches the device while exposed to direct rain, wet pavement, or while working in a damp environment. If water enters the device, there is an increased risk of damage to the unit and electric shock.
- Do not touch the device with wet or damp hands.
- Do not use the cord in an unintended manner. Never use it to carry the device or to pull the plug out of the socket. Keep the cord away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cords increase the risk of electric shock.

- e) When operating the product outdoors, use an extension cord designed for outdoor use. Using an extension cord designed for outdoor use reduces the risk of electric shock.
- f) If you cannot avoid using the product in a wet environment, use a residual current device (RCD) to connect the product to electrical mains. Using an RCD reduces the risk of electric shock.
- g) Do not use the device if the power cord is damaged or shows signs of wear. A damaged power cord should be replaced by a qualified electrician or the manufacturer's service department.
- h) To avoid electric shock, do not immerse the cable, plug, or the device itself in water or other liquid. Do not use the appliance on wet surfaces.
- i) When cleaning or using the appliance, never immerse it in water or other liquids.
- j) Do not allow the machine to get wet. Risk of electric shock!
- k) Use only the included power adapter!

2.2. Safety in the workplace

- a) Do not use the unit in a hazardous zone, for example in the presence of flammable liquids, gases or dust.
- b) If in doubt as to whether the product is working properly or found to be damaged, contact the manufacturer's technical service.
- c) Only the manufacturer's service department may repair the appliance. Do not attempt to repair the product on your own!
- d) In the event of ignition or a fire, use dry powder or CO2 extinguishers only to suppress the fire if the appliance is live.
- e) Keep this manual for future reference. If the product is to be handed over to a third party, hand it over with this user manual.
- f) Keep packaging components and small installation parts out of the reach of children.

Remember! Keep children and other bystanders safe while operating the appliance.

2.3. Personal safety

- a) The product is not intended to be used by individuals (including children) with reduced mental, sensory or intellectual capacity or lack of experience and/or knowledge, unless they are supervised by an individual responsible for their safety or have been given instructions by the responsible individual on how to operate the product.
- b) The appliance is not a toy. Children should be watched to ensure that they do not play with the appliance.

2.4. Safe use of the product

- a) Keep unused appliance out of the reach of children and anyone unfamiliar with the appliance or this manual. Products are dangerous when used by inexperienced users.
- b) Keep the product in good working order. If damaged, have the device repaired before use.
- c) Keep the product out of the reach of children.
- d) Repairs and maintenance should be carried out by qualified personnel using only original spare parts. This will ensure the safety of use.
- e) To ensure the designed operational integrity of the device, do not remove factory-installed covers or loosen screws.
- f) Clean the device regularly to prevent permanent dirt build-up.
- g) The product is not a toy. Cleaning and maintenance must not be performed by children without adult supervision.
- h) Do not tamper with the device to alter its performance or design.
- i) Keep the unit away from sources of fire and heat.
- j) Do not overload the device.

CAUTION! Although the product has been designed to be safe, with adequate safeguards, and despite the additional safety features provided to the user, there is still a slight risk of accident or injury when handling the unit. Caution and common sense are advised when using the product.

3. Instructions for use

The device is designed to measure the weight of elements placed on the pan. Do not exceed the maximum load of the scale - it may damage it!

The user is responsible for any damage resulting from misuse.

3.1. Product overview

- 1 - Control panel with display.
- 2 - Weighing platform.

kg/lb - indicator of the currently used weighing unit.

ON/OFF - scale switch.

HOLD - holding the weight result on the display.

UNIT - selection of the weighing unit.

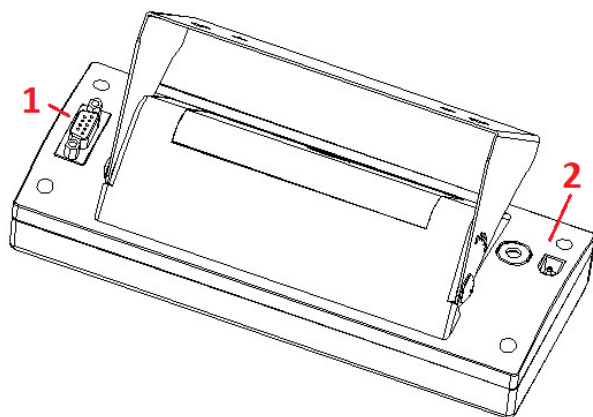
ZERO (also used as TARE) The **zeroing** function should only be used when there is nothing on the pan and the display does not show "0". The scale may not read "0" due to dust, dirt or moisture on the weighing pan or when a heavy object is

dropped on the scale. The zeroing function essentially corrects the true zero on the scale and informs it that there is nothing on the pan.

The **tare** function is the weight of the package itself. It is the difference between the total weight of the goods with packaging and the weight of the goods without packaging. The tare is used to remove the weight of an item that should not be included in the overall weight measurement.

PRINT - sends information from the display to the computer software.

Control panel with display - rear view:

1 - RS232 socket

2 - Power adapter plug socket

3.2. Preparation for operation

POSITIONING OF THE UNIT

The ambient temperature must not exceed 40°C and ambient humidity should not exceed 85%. Place the unit in a way that ensures good air circulation. Maintain a minimum clearance of 10 cm from any wall of the unit. Keep the unit away from any hot surfaces. Always operate the unit on a level, stable, clean, fireproof and dry surface and out of the reach of children and persons of impaired mental, sensory and intellectual functions. Place the unit in such a way that the mains plug can be reached at any time. Ensure that the power supply to the unit corresponds to that specified on the identification plate!

3.3. Working with the unit

3.3.1. Function settings

- Press and hold the HOLD key, then press the ON/OFF key to turn on the scale. The display will show A-ON or A-OFF. Press the ZERO key to turn on or off the automatic power off:
 - ON indicates that the auto power off function is enabled
 - OFF means that the auto power off function is disabled
- Press the UNIT button so the display shows L-ON or L-OFF. Press the ZERO button to select the automatic backlight function:
 - ON means that the automatic backlight function is activated
 - OFF means that the automatic backlight function is disabled
- Press the UNIT button, the display will show P-XX. Press the ZERO key to select from "OFF"/"key"/"con"/"Stb":
 - "OFF" means that the RS232 function is inactive
 - "KEY" means that the data is sent when the PRINT key is pressed
 - "Con" means that the data will be sent continuously
 - "Stb" means that the data is sent when the reading is stable
- Press the UNIT key and the display will show J-ON or J-OFF. Press the [ZERO] key to select ON or OFF:
 - ON means to show zero after power on
 - OFF means showing the value of the load placed after turning on the power

3.3.2. Setting the operating mode of the indicator

- Press and hold the UNIT and HOLD keys, then press the ON/OFF key to turn on the scale. The display will show Ani.Sw. Press the UNIT key to enter the function settings menu - press the ZERO key to select ON or OFF:
 - ON stands for animal mode
 - OFF means normal weighing mode,
 - press the HOLD key to confirm the selection.
- Press the HOLD key again to enter the next menu. The display will show Ani.Kd; press the UNIT key to enter the function setting menu. Then press the ZERO key to select the anti-shock range of the animal scale from 50, 100, 200, 400 - press the HOLD key to confirm the selection.
- Press the HOLD key again to enter the next menu. The display will show Ani.fd; press the UNIT key to enter the function setting menu. Then press the ZERO key to select the animal weight filter range from 0.5, 3, 5, 10 - press the HOLD key to confirm the selection.

- Press the HOLD key again to enter the next menu. The display will show Ani.Ft; press the UNIT key to enter the function settings menu. Then press the ZERO key to select the filtering time of the animal weight among 1, 2, 3, 4 - press the [HOLD] key to confirm the selection.

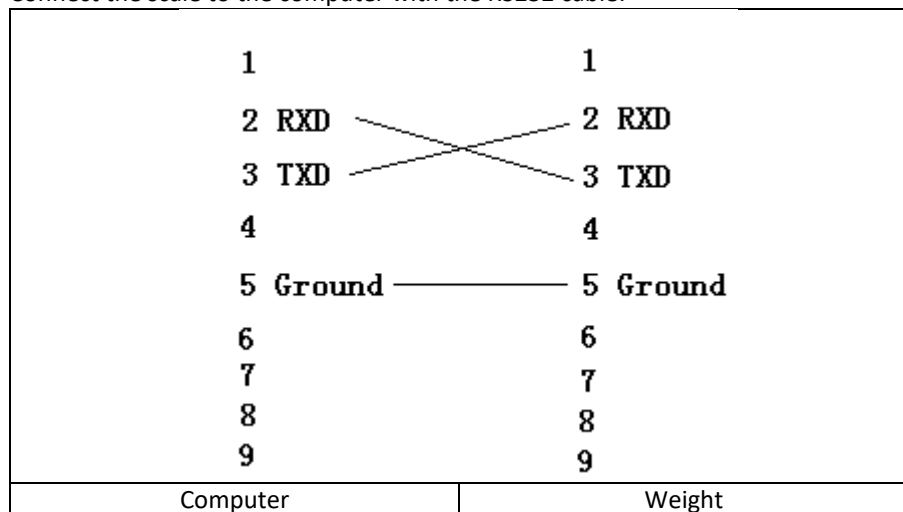
3.3.3. Calibration

Note: Calibration is performed at the factory. Do not recalibrate the scale unless it is very inaccurate.

- Press and hold the ZERO key then press the ON/OFF key. The display will show CAPU=. Then press the UNIT key to select the maximum load unit in kg or lb and press the HOLD key to confirm the selection.
- Press the HOLD key again to enter the next menu. The display will show CAP, so press the UNIT key to set the maximum load of the scale. Then press the UNIT key to move the flashing digit to the right. Press the ZERO key to increase the value of the flashing digit and press the HOLD key to confirm your selection.
- Press the HOLD key again to enter the next menu. The display shows CALU =, so press the UNIT key to select the kg or lb calibration unit. Then press the HOLD key to confirm the selection.
- Press the HOLD key again to enter the next menu. The display will show CAL, so press the UNIT key to set the value of the calibration weight. Then press the UNIT key to move the flashing digit to the right and press the ZERO key to increase the value of the flashing digit. Press the HOLD key to confirm the selection (the value of the adjusting weight should be greater than 2/3 of full capacity).
- Press the HOLD key again to enter the next menu. The display will show DIV, so press the UNIT key to select the scale. Press the HOLD key to confirm the selection.
- Press the HOLD key again to enter the next menu. The display will show ADC, so press the UNIT key to select 20. Press the HOLD key to confirm the selection.
- Press the PRINT key. The display shows CAL, then the AD value. Next press the UNIT key when the stable indicator appears and the display shows a flashing value of the calibration weight. Place the displayed weight on the platform. When the display shows a stabilized result, press the UNIT key, so ---- will appear on the display, the value of AD. Turn off the scale. Calibration is complete and you can remove the weights.
- Turn on the scale, place the tested weight to check if the indications are correct. If not, repeat the above procedure.

3.3.4. Cooperation with a computer

Connect the scale to the computer with the RS232 cable:


Turn on the scale. Run the "AccessPort.exe" program (to be downloaded from [www.....](#)).

In the menu, select "View" -> "Language" -> "English"

In the menu, select: "Tools" -> "Configuration" - "Display":

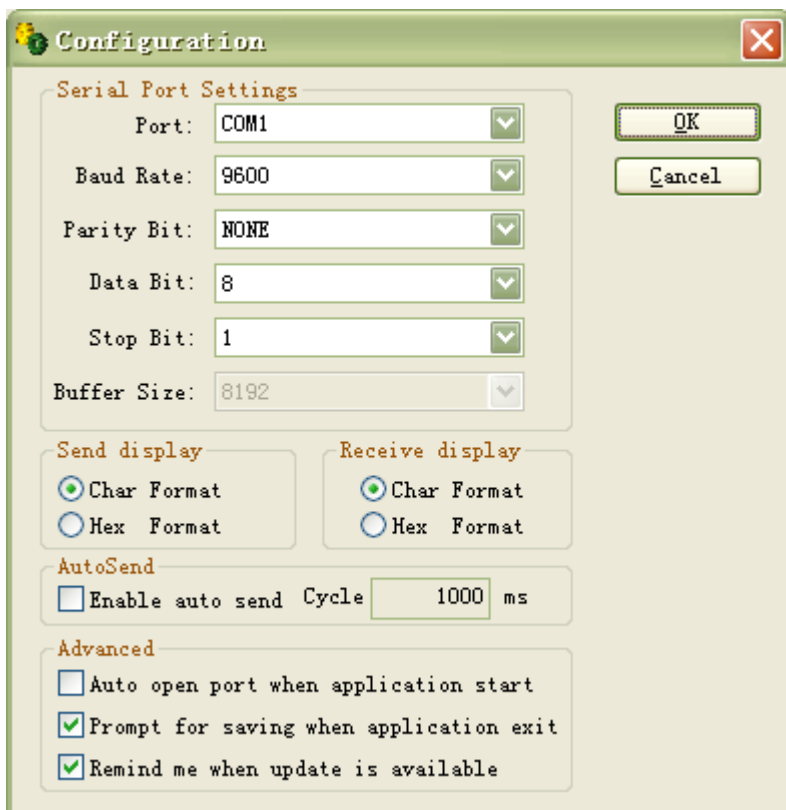NOTE: Select the COM port on the computer.

A configuration dialog box titled "Configuration" with a green header bar and a red close button. It contains several sections for setting serial port parameters and display options.

Serial Port Settings

Port: COM1

Baud Rate: 9600

Parity Bit: NONE

Data Bit: 8

Stop Bit: 1

Buffer Size: 8192

Send display

☒ Char Format

☐ Hex Format

Receive display

☒ Char Format

☐ Hex Format

AutoSend

☐ Enable auto send Cycle 1000 ms

Advanced

☐ Auto open port when application start

☒ Prompt for saving when application exit

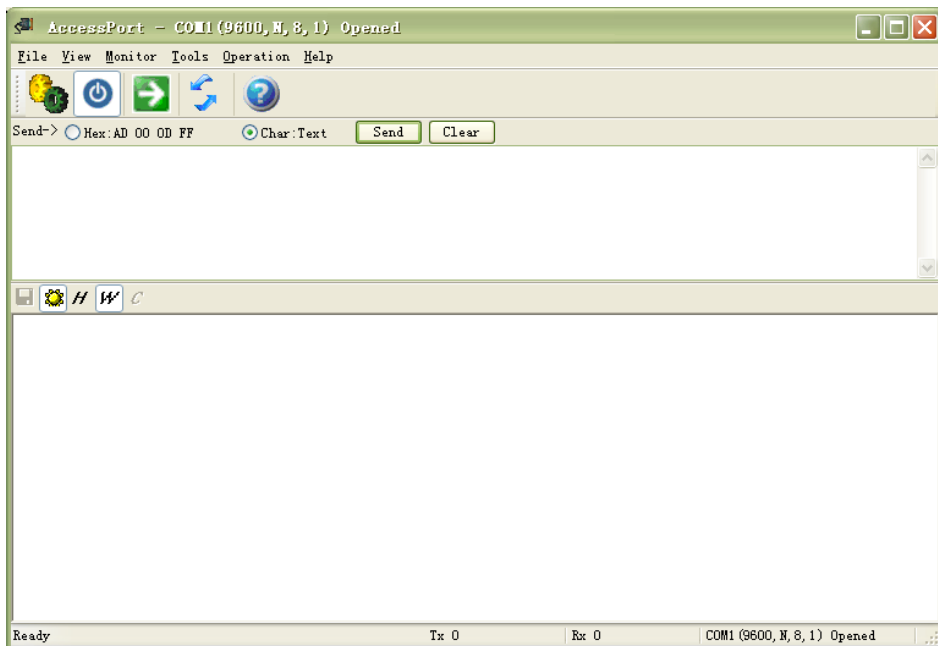
☒ Remind me when update is available

OK


Cancel

Press OK.

View:

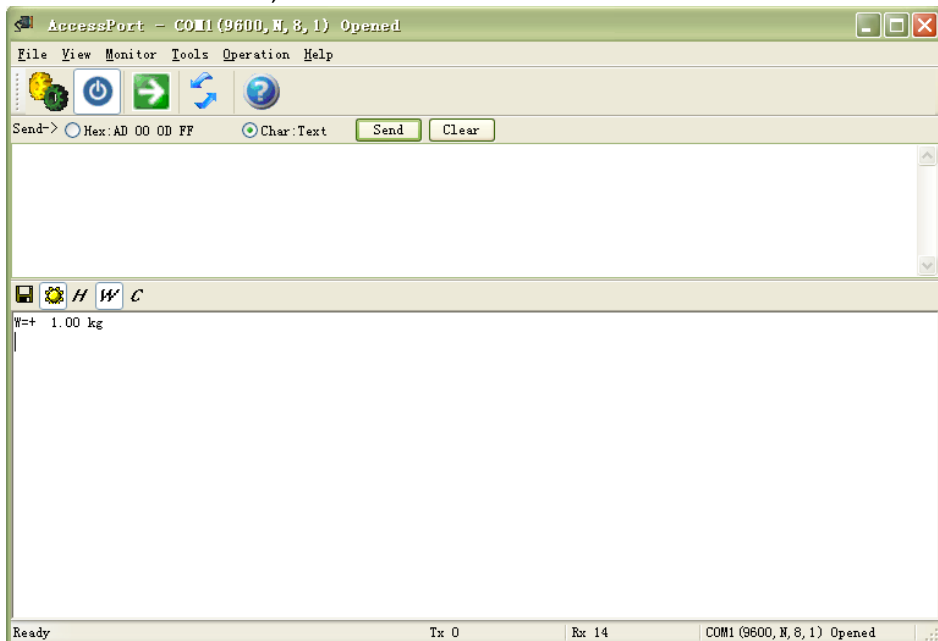Press PRINT on the scale, the result will be shown on the screen:

3.4. Cleaning and maintenance

- a) Pull out the mains plug before each cleaning and when the unit is not in use.
- b) Use only non-corrosive cleaning agents for cleaning the surfaces.
- c) After each cleaning, all parts should be thoroughly dried before the device is reused.
- d) Store the unit in a dry and cool place protected from moisture and direct sunlight.
- e) Do not spray the unit with a stream of water or immerse it in water.
- f) Make sure that no water enters through the ventilation openings in the casing.
- g) Perform regular inspections of the unit checking technical fitness and any damages.
- h) Use a soft cloth for cleaning.
- i) Do not leave the batteries in the unit when it will not be used for a long time.
- j) Do not use sharp and/or metal objects (e.g. wire brush or metal spatulas) as they may damage the surface of the material from which the unit is made.
- k) Do not clean the unit with acidic substances, medical agents, thinners, fuel, oils or other chemicals. It may cause damage to the device.

INSTRUCTIONS FOR THE SAFE DISPOSAL OF BATTERIES.

The devices are equipped with **AAA 1.5V** batteries.

Remove used batteries from the unit in the same way you put them in.

Return the batteries to a unit responsible for their disposal.

DISPOSAL OF USED UNITS.

At the end of its useful life, this product should not be disposed of with normal household waste but should be taken to a collection point for the recycling of electrical and electronic equipment. This is indicated by the symbol on the product, operating instructions or packaging. The materials used in this appliance are recyclable according to their marking. By reuse, recycling or applying other forms of use of waste machines, you make a significant contribution to the protection of our environment.

Your local administration will provide you with information about the appropriate disposal point for used appliances.

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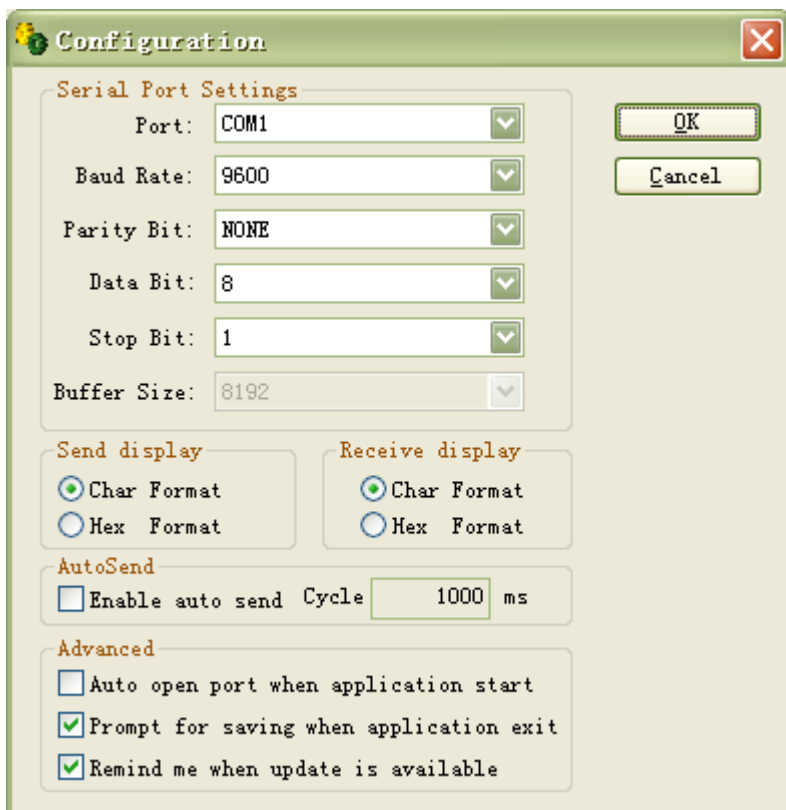
A configuration dialog box titled "Configuration" with a green header bar and a red close button. It contains several sections: "Serial Port Settings" with dropdowns for Port (COM1), Baud Rate (9600), Parity Bit (NONE), Data Bit (8), Stop Bit (1), and Buffer Size (8192); "Send display" and "Receive display" sections with radio buttons for "Char Format" (selected) and "Hex Format"; an "AutoSend" section with a checkbox for "Enable auto send Cycle" (unchecked) and a text field for "1000 ms"; and an "Advanced" section with checkboxes for "Auto open port when application start" (unchecked), "Prompt for saving when application exit" (checked), and "Remind me when update is available" (checked). "OK" and "Cancel" buttons are on the right.

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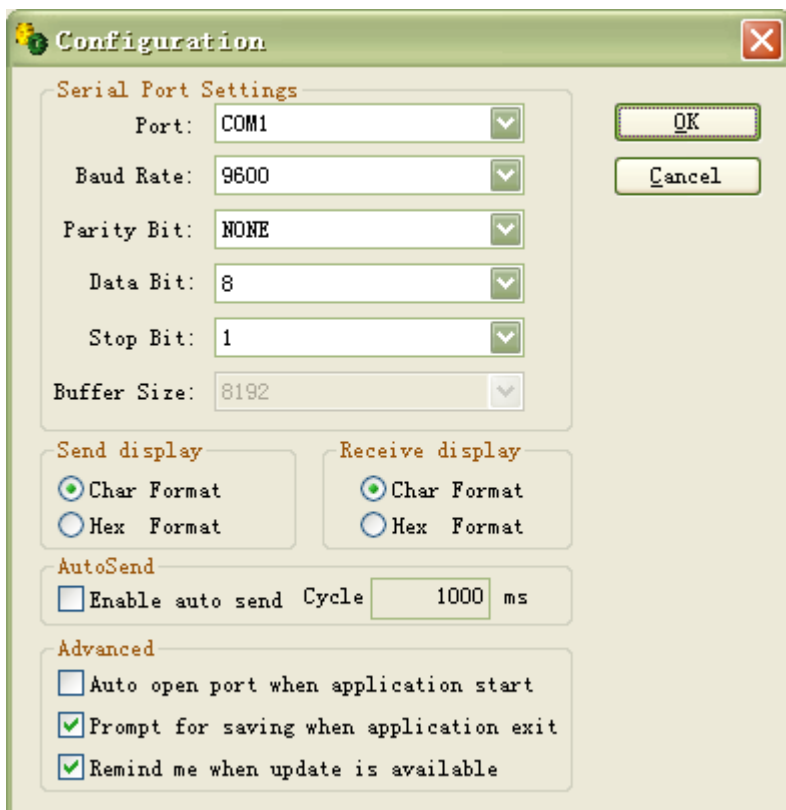
The image shows a 'Configuration' dialog box with a green title bar and a red close button. It contains several sections: 'Serial Port Settings' with dropdowns for Port (COM1), Baud Rate (9600), Parity Bit (NONE), Data Bit (8), Stop Bit (1), and Buffer Size (8192); 'Send display' and 'Receive display' sections with radio buttons for 'Char Format' (selected) and 'Hex Format'; an 'AutoSend' section with a checkbox for 'Enable auto send Cycle' (unchecked) and a text field for '1000 ms'; and an 'Advanced' section with checkboxes for 'Auto open port when application start' (unchecked), 'Prompt for saving when application exit' (checked), and 'Remind me when update is available' (checked). 'OK' and 'Cancel' buttons are on the right.

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