USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

ELECTRIC MASSAGE TABLE

DE	Produktname	Elektrische Massageliege
EN	Product name	Electric Massage Table
PL	Nazwa produktu	Elektryczny stół do masażu
CZ	Název výrobku	Elektrický masážní stůl
FR	Nom du produit	Table de massage électrique
IT	Nome del prodotto	Lettino da massaggio elettrico
ES	Nombre del producto	Camilla de masaje eléctrica
HU	Termék neve	Elektromos masszázstágy
DA	Produktnavn	Elektrisk massagebriks
FI	Tuotteen nimi	Sähköinen hierontapöytä
NL	Productnaam	Elektrische massagetafel
NO	Produktnavn	Elektrisk massasjebenk
SE	Produktnamn	Elektrisk massagebänk
PT	Nome do produto	Mesa de massagem elétrica
SK	Názov produktu	Elektrický masážny stôl
BG	Име на продукта	Електрическа масажна маса
EL	Όνομα προϊόντος	Ηλεκτρικό τραπέζι μασάζ
HR	Naziv proizvoda	Električni masažni stol
LT	Produkto pavadinimas	Elektrinis masažo stalas
RO	Numele produsului	Masă de masaj electrică
SL	Ime izdelka	Električna masažna miza
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		DIVINE
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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This User Manual has been translated using machine translation. We have made every effort to ensure the translation is accurate, but please note that automated translations are not perfect and are not meant to replace human translators. The official version of the User Manual is in English. Any differences between the translated version and the original English are not legally binding. If you have any questions about the accuracy of the translation, please refer to the English version, which is the official reference. More language versions are available upon request via info@expondo.com.

1. Technical data

Table 1: Technical data of the product

Parameter description	Parameter value
Product name	Electric Massage Table
Model	DIVINE
Dimensions (LxWxH)	197 x 96 x 81 cm
Adjustment angle, backrest [°]	-13 - 45
Adjustment angle, footrest [°]	0 - 20
Backrest dimensions [mm]	750 x 800
Bed length	197 cm
Bed width	70 cm
Headrest dimensions [mm]	270 x 170 x 90
Lifting mechanism	Electric
Maximum height	81 cm
Maximum load	200 kg
Minimum height	63 cm
Motor quantity	3
Power [W]	300
Voltage [V]	230
Weight	70.5 kg

2. General description

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement. The device is designed to reduce noise emission risks to a minimum, taking into account technological progress and noise reduction opportunities.

PLEASE NOTE! DRAWINGS IN THIS MANUAL ARE FOR ILLUSTRATION PURPOSES ONLY AND IN SOME DETAILS MAY DIFFER FROM THE ACTUAL PRODUCT.

3. Usage safety

ATTENTION! READ ALL SAFETY WARNINGS AND ALL INSTRUCTIONS. FAILURE TO FOLLOW THE WARNINGS AND INSTRUCTIONS MAY RESULT IN ELECTRIC SHOCK, FIRE AND/OR SERIOUS INJURY OR EVEN DEATH.

The terms "device" or "product" are used in the warnings and instructions to refer to: Electric Massage Table

The equipment is tested in accordance with the current European norms (EN) for this product type and therefore corresponds also with the newest safety regulations. In addition, the equipment meet/complies with all EEC directives that are required in line with the CE identification/labeling.

The manufacturer cannot be held liable for damages or consequential damages, respectively, which result from noncompliance with the following instructions:

- Only certified electricians or service partners may carry out repairs on electrical appliances. Unprofessional repairs can cause/create serious risks for the operator.
- The liability for potential damages is discharged if the product is used inappropriately, incorrectly operated or not professionally repaired.
- This product may only be used with the [provided/original] adaptor – see technical data.
- In cases of malfunctioning during operation, the power plug of the adaptor has to be removed from the power outlet previous to every cleaning and maintenance and after every use.
- Never remove the power plug from the power outlet by pulling on the cord or with wet hands.
- The power cord should never be dragged over wet edges or jammed.
- Children or handicapped persons are not permitted to use the product without supervision.
- Do not operate the equipment in wet surroundings like a bathroom or swimming pools.
- Piercing with sharps objects, like needles, among other things, is prohibited
- Occasionally and depending on use, the product is to be inspected carefully for damages to the synthetic leather cover and the cables.

- In the event of a repair the massage bed is to be shipped only in the original package to prevent additional transport damages.

ATTENTION! DESPITE THE SAFE DESIGN OF THE DEVICE AND ITS PROTECTIVE FEATURES, AND DESPITE THE USE OF ADDITIONAL ELEMENTS PROTECTING THE OPERATOR, THERE IS STILL A SLIGHT RISK OF ACCIDENT OR INJURY WHEN USING THE DEVICE. STAY ALERT AND USE COMMON SENSE WHEN USING THE DEVICE.

4. Use guidelines

The user is liable for any damage resulting from unintended use of the device.

4.1. Device description

4.2. Preparing for use

4.2.1. Appliance location

The temperature of environment must not be higher than 40°C and the relative humidity should be less than 85%. Ensure good ventilation in the room in which the device is being used. There should be at least 10 cm distance between each side of the device and the wall or other objects. The device should always be used when positioned on an even, stable, clean, fireproof and dry surface, and be out of the reach of children and persons with limited mental and sensory functions. Position the device such that you always have access to the power plug. The power cord connected to the appliance must be properly grounded and correspond to the technical details on the product label.

Disassemble the device and all its components and clean them before the first use.

4.3. Assembling the device

Take the massage bed out of the box and then remove the foil.

⚠ ATTENTION! DO NOT REMOVE THE FOIL WITH SCISSORS OR KNIVES.


Connect the remote control and plug the power cord into an outlet.

Now you can start adjusting the height by using the remote control. As there is only one available port, you can either connect the remote control or the foot control.


You can use it, as the pictures below show.

4.4. Operation Instruction


4.4.1. Back angle adjustment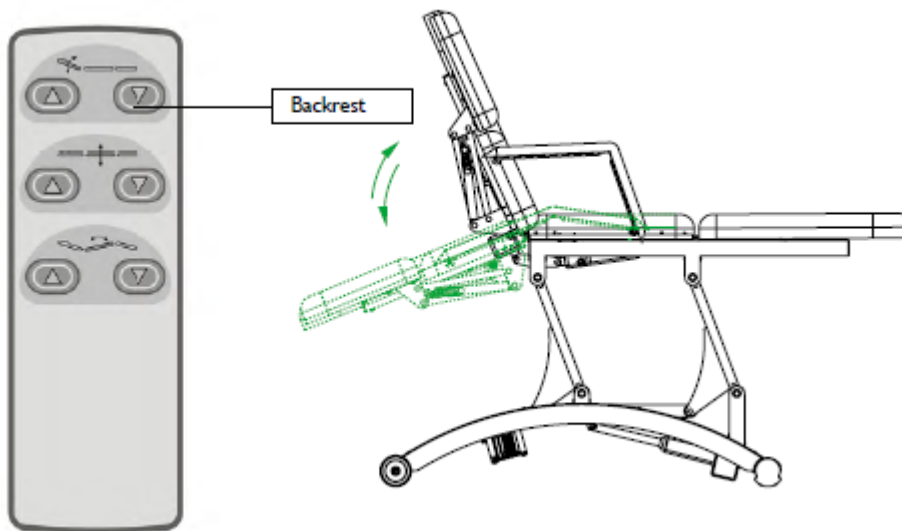


4.4.1. Up and down adjustment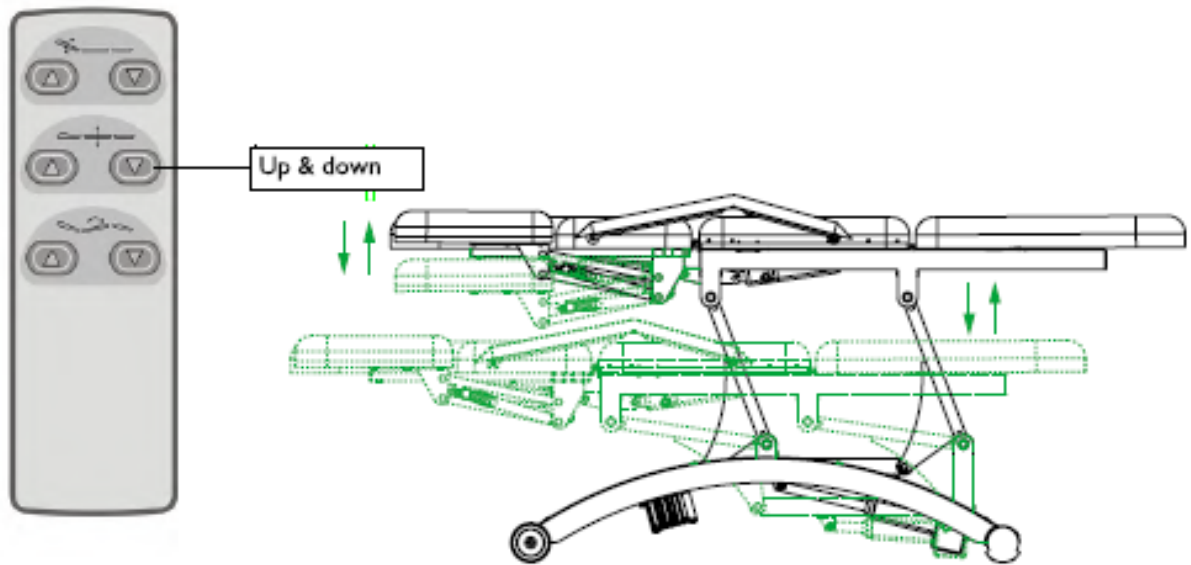

4.4.2. Seat angle adjustment

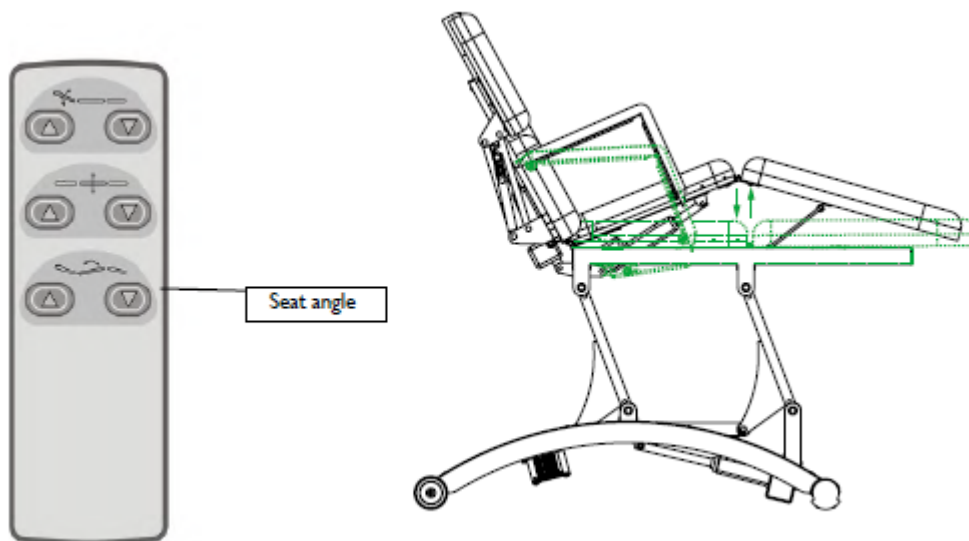
4.4.3. Combined adjustment (back angle, up & down, seat angle can be adjusted together)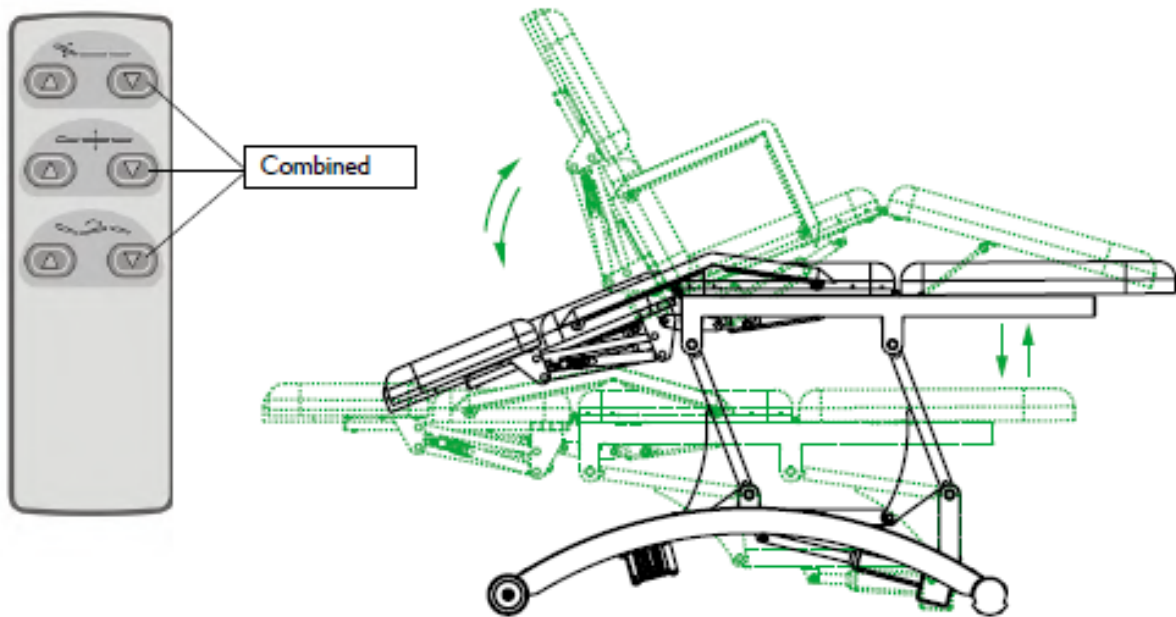


4.4.4. Moving

To move the massage bed, please lift it at the bottom end and roll the bed to the desired location.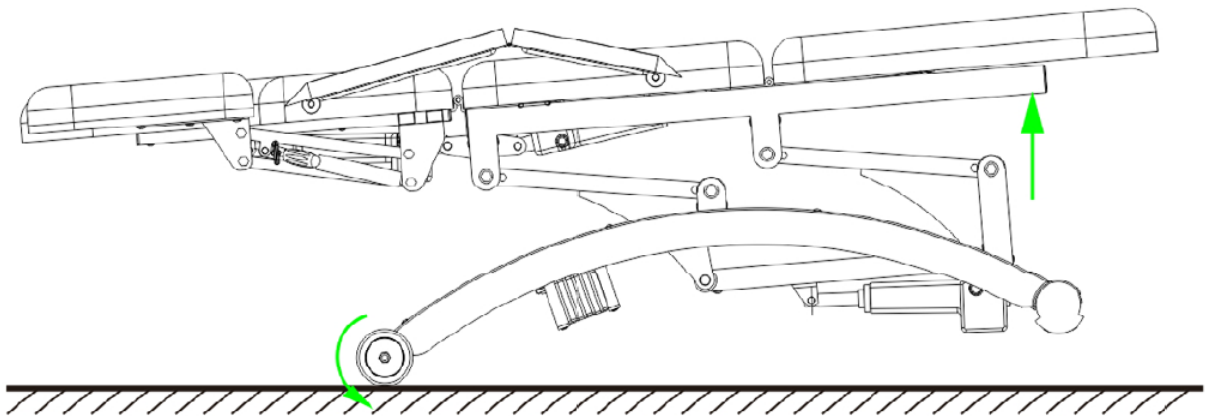

4.4.5. Precautions

When you start to adjust the backrest, footrest angle or the height, do not put your hand, foot or head to the position 12345678 showed on the picture below.

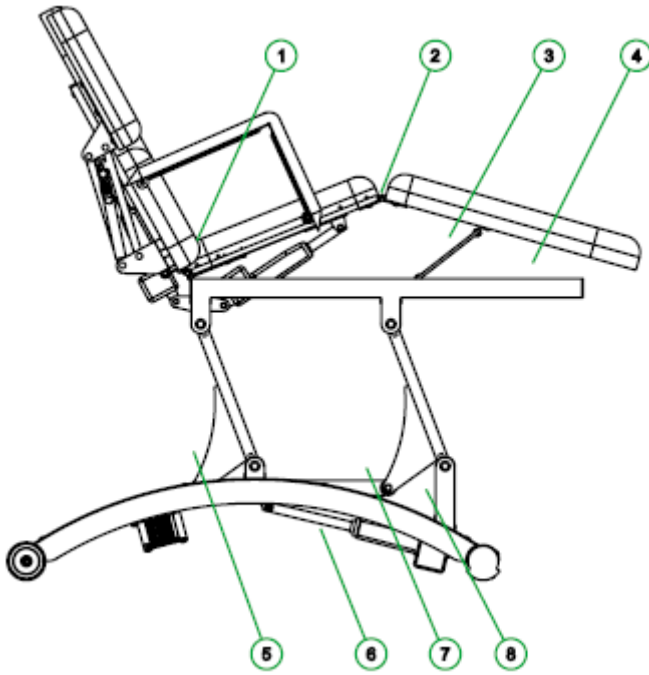4.5. Cleaning and maintenance

After every treatment, please use the neutral reagent clear leather, the dust, stains on the plastic cover, in order to prevent premature aging of the leather and the plastic cover.

Never immerse the equipment in water! The cover was made out of soft and easycare synthetic leather. Regular dirt and stains can be removed with a solution of mild soap and lukewarm water. Afterwards, rub the cleaned area dry with a soft and clean cloth.

4.6. Environment protection

Raw material recovery instead of waste disposal: The equipment, accessory and packaging should be ecologically recycled. (Applies only to electrical components)

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