# DIVINE

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User manual | Manuel d'utilisation | Istruzioni per l'uso | Manual de instrucciones | Instrukcja

Inhalt | Contenu | Content | Contenuto | Treść | Contenido

Deutsch 3
English 8
Polski 14

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## 2. INSTALLATION STEPS

- 1) Open the box; place the bed to the proper position.
- 2) Connect the power and then you can use it, as the pictures below show.

### 3. BRINGING INTO SERVICE

0

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v

Take the massage bed out of the box and then remove the foil.

## Attention: Do not remove the foil with scissors or knives.

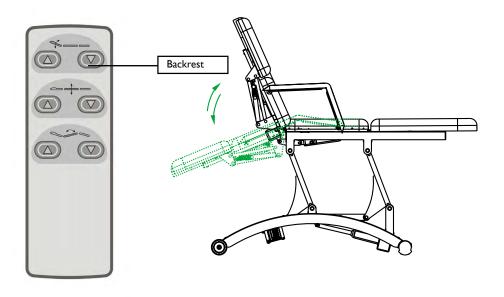
Connect the remote control and plug the power cord into an outlet.

Now you can start adjusting the height by using the remote control. As there is only one available port, you can either connect the remote control or the foot control.


## **OPERATION INSTRUCTION**

# I) Back angle adjustment



# **SEAT ANGLE ADJUSTMENT**
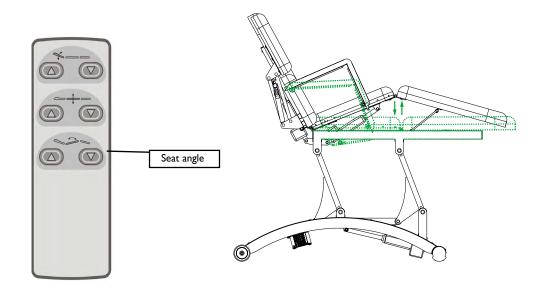
# **COMBINED ADJUSTMENT**(BACK ANGLE, UP & DOWN, SEAT ANGLE CAN BE ADJUSTED TOGETHER)
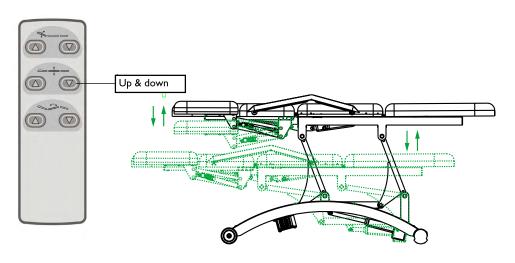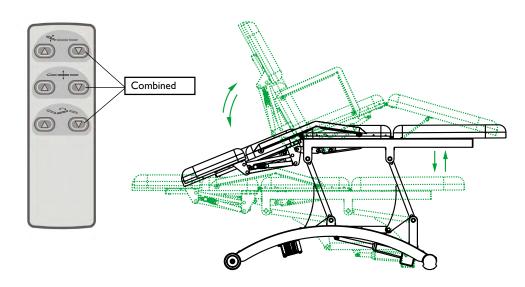
# **UP AND DOWN ADJUSTMENT**



#### MOVING

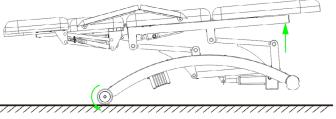To move the massage bed, please lift it at the bottom end and roll the bed to the desired location.

#### **PRECAUTIONS**

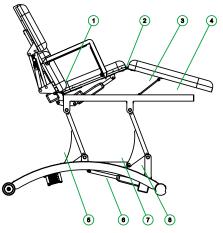When you start to adjust the backrest, footrest angle or the height, do not put your hand, foot or head to the position 12345678 showed on the picture below.

#### **MAINTENANCE**

After every treatment, please use the neutral reagent clear leather, the dust, stains on the plastic cover, in order to prevent premature aging of the leather and the plastic cover.

#### 4. SAFETY INSTRUCTIONS

The equipment is tested in accordance with the current European norms (EN) for this product type and therefore correspondes also with the newest safety regulations. In addition, the equipment meet/complies with all EEC directives that are required in line with the CE identification/labeling.

The manufacturer cannot be held liable for damages or consequential damages, respectively, which result from noncompliance with the following instructions:

12

- Only certified electricians or service partners may carry out repairs on electrical appliances. Unprofessional repairs can cause/create serious risks for the operator.
- The liability for potential damages is discharged if the product is used inappropriately, incorrectly operated or not professionally repaired.
- This product may only be used with the [provided/original] adaptor see technical data.
- In cases of malfunctioning during operation, the power plug of the adaptor has to be removed from the power outlet previous to every cleaning and maintenance and after every use.
- Never remove the power plug from the power outlet by pulling on the cord or with wet hands.
- The power cord should never be dragged over wet edges or jammed.
- Children or handicapped persons are not permitted to use the product witout supervision.
- Do not operate the equipment in wet surroundings like a bathroom or swimming pools.
- · Piercing with sharps objects, like needles, among other things, is prohibited
- Occasionally and depending on use, the product is to be inspected carefully for damages to the synthetic leather cover and the cables.
- In the event of a repair the massage bed is to be shipped only in the original package to prevent additional transport damages.

#### 5. ENVIRONMENT PROTECTION

Raw material recovery instead of waste disposal: The equipment, accessory and packaging should be ecologically recycled.

(Applies only to electrical components)

#### 6. MAINTENANCE AND CARE

Never immerse the equipment in water! The cover was made out of soft and easy-care synthetic leather. Regular dirt and stains can be removed with a solution of mild soap and lukewarm water. Afterwards, rub the cleaned area dry with a soft and clean cloth.

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Bedienungsanleitung | User manual | Manuel d'utilisation | Istruzioni per l'uso | Manual de instrucciones

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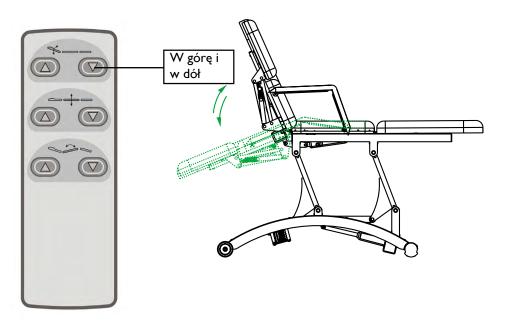
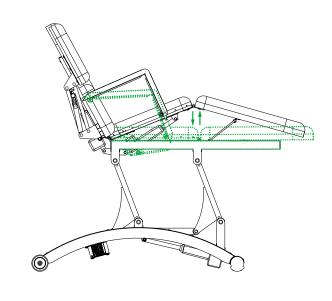
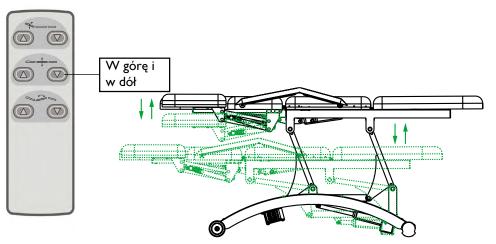
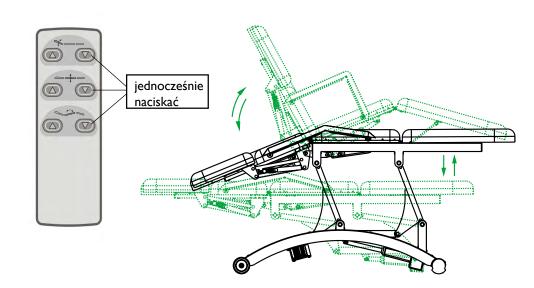
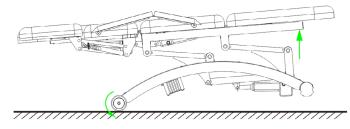
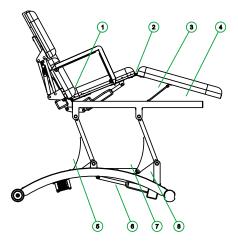
#### EN

We hereby certify that the appliances listed in this manual are CE compliant.

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