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USER MANUAL | MANUEL D'UTILISATION | ISTRUZIONI PER L'USO | MANUAL DE INSTRUCCIONES | INSTRUKCJA

HIGH-TEMP STERILISER HTS-80

Deutsch	3
English	6
PolSKI	9
Français	12
Italiano	15
Español	18

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LEGEND

-  Please read this user manual carefully.
-  Caution! Risk of fire or explosion.
-  Caution! Contact with hot surfaces may cause burns

GENERAL SAFETY INFORMATION FOR THE USE OF ELECTRICAL DEVICES:

To avoid injury from fire or electric shock, please make sure to follow these safety instructions when using the device. Please read the instructions carefully and make sure that you have understood them well. Keep the manual at hand to be able to read it again at any time. Always use grounded power outlets providing an adequate voltage (as indicated on the device). If you have any doubt, let an electrician check that your outlet is properly grounded. Never use a damaged power cable. Do not use the device in damp or wet environment, or if your hands or body are damp or wet. Protect the device from sunlight. Use the device in a protected environment to avoid damaging the equipment or endangering others. Make sure the device can cool down properly and avoid placing it too close to other heat-producing devices. Before cleaning, always unplug the device. Use a soft damp cloth for cleaning. Avoid using detergents and make sure that no liquid enters the device. The inner elements of this device don't require any maintenance from the user. An unauthorized opening of the device may void the guarantee.

SAFETY GUIDELINES

1. Please read and understand these instructions carefully before operating the device! Please observe the safety guidelines carefully in order to prevent any damage due to improper use!
2. Please keep this manual at hand for future reference. If the device is handed over to third parties, this manual must be handed over too.
3. Only use this device indoors and for its intended use.
4. The guarantee does not cover damages resulting from improper use or operation.
5. Before first use, ensure that the voltage and current requirements of the device match the available power supply.
6. This device should not be used by persons (including children) with limited physical, sensory or mental aptitude, or lacking the necessary experience and/or knowledge – unless they are supervised by a qualified person or have received safety instructions from this person.
7. **ELECTRICAL SHOCK HAZARD!** Do not attempt to repair the device yourself. In case of device failure, repair must be done by a qualified professional.
8. Please check the main plug and power cable regularly. If the power cable is damaged, it has to be replaced by the manufacturer or its customer service or another qualified person, in order to prevent hazards.
9. Prevent damages to the power cable by avoiding squeezing, bending or rubbing on sharp edges and keep it away from hot surfaces and open flames.
10. **CAUTION DANGER TO LIFE!** During cleaning, never immerse the device in water or other liquids. Never put the device under running water or pour other liquids on it. Never pour / dispense liquids over the device. If any liquid should get inside the device, immediately unplug it and contact the service team.
11. Don't open the device under any circumstances.
12. Ensure sufficient air circulation. The device should be placed over 10cm away from walls and other electrical devices. Ventilate well the working area.
13. Do not place the device close to combustible materials.
14. During first use, the heating element might produce some smoke. This is absolutely normal. Just ensure sufficient ventilation so that that the smoke doesn't accumulate in the room.
15. Some parts of the device can get extremely hot. To avoid any injury, be careful when you touch the device.
16. Never leave the device and its packaging unattended.
17. Clean the device regularly to avoid a durable encrustation of dirt.

TECHNICAL DATA

Product name	High-temp Sterilizer
Model	HTS-80
Art. No.	4101
Rated voltage [V] / Frequency [Hz]	230 V / 50 Hz
Temperature [°C]	250
Power [W]	80

PURPOSE

This high-temperature steriliser is a device designed for the quick sterilisation of beauty tools and accessories (such as nail clippers, tweezers, etc.) as well as other small metal tools at high temperature. Quartz balls maintain a high temperature and keep the tools and accessories in place during the sterilisation process.

In case of damage resulting from an improper use of the device, the user shall assume all liability.

BEFORE FIRST USE

On receipt of the parcel, make sure the packaging is undamaged before opening it. If the packaging is damaged, please contact your transport company and distributor within 3 days. Be as specific as possible when describing the damages. Do not turn the package upside down! When transporting the package, please ensure that it is kept horizontal and stable. The packaging materials (cardboard, plastic straps, polyurethane foam) should be kept at hand, so that the device can be sent back to the service center, if needed, in the best conditions!

ENVIRONMENT OF USE

The room temperature must not be higher than 45°C and the relative humidity should be under 85%. Keep the device away from hot surfaces. Operate the device on flat, stable, clean, refractory and dry surfaces, out of the reach of children and people with mental disabilities. Make sure that the power outlet is always within reach. The power outlet must be properly grounded and match the technical requirements of the device.

OPERATING PRINCIPLE

1. main switch
2. timer switch button
3. LCD display
4. sterilisation chamber
5. chamber lid
6. quartz balls

CONNECTING / STARTING

Do not switch the device on if the chamber is empty – put the quartz balls inside the chamber. Make sure that the main switch is in the OFF position. Plug the power cord into the outlet. Turn the device on with the main switch: the display shows „8.8.“, followed by „—“. Press the timer button (TIMER), the display now shows „15“ and the device starts its 15-minute heating process. The remaining heating time is counted down on the display. Three short beeps indicate that the heating process is over: the display shows „0.0.“ The steriliser is now ready for use.

Place the tools / instruments among the quartz balls and press the TIMER button - this will launch a 20-second-sterilisation cycle. A single beep indicates that the sterilisation has been completed; the display shows „0.0.“. The device is ready for the next sterilisation cycle.

The device is equipped with a safety feature interrupting heating after 8 hours of continuous use. If the interruption occurs when the steriliser in the standby mode (STAND-BY, the display shows „0.0.“) the heating will turn off, a short beep will be heard five times and the display will show „—““. If the 8-hour limit is passed during use, the heating will be interrupted after completion of the sterilisation cycle and the display will show „0.0.“.

Should you want to use the device for more than 8 hours, turn the steriliser off with the main switch and then turn it on again. The device will need 15 minutes to warm up.

WARNING!

- Do not use the device in damp areas (such as bathrooms).
- Pre-clean the metal tools and instruments before sterilisation.
- Clean the quartz balls and the sterilisation after use.
- Do not use damaged quartz balls

TRANSPORTATION AND STORAGE

When transporting the device, avoid shaking it, dropping it and turning it upside down. Store it in a cool, dry place, free of any corrosive gas.

CLEANING AND MAINTENANCE

- Unplug the device before cleaning.
- Use a mild, non-corrosive detergent.
- Let the device dry before using it again.
- Store the device in a dry, cool location, free from moisture and direct sunlight.
- The quartz balls can be washed in a sieve under running water with a mild detergent.

REGULAR CONTROL OF THE DEVICE

Regularly ensure that the device is in good condition. Should it be damaged, please stop using it immediately and contact your customer service to solve the problem.

What to do in case of problem?

Please contact your customer service and prepare the following information:

- Invoice number and serial number (the latter is to be found on the technical plate on the device).
- If relevant, a picture of the damaged, broken or defective part.
- It will be easier for your customer service advisor to identify the source of the problem if you give a detailed and precise description of the matter

The more details you give, the quicker the customer service will be able to solve your problem!

CAUTION: Never open the device without the authorization of your customer service. This may void the guarantee!

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