USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄY T TÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

BEAUTY BED

| | |
|----|---|
| DE | 3 |
| EN | 4 |
| PL | 5 |
| CZ | 6 |
| FR | 7 |
| IT | 8 |
| ES | 9 |

| | |
|----------------------------|---|
| PRODUKTNAME | KOSMETIKLIEGE |
| PRODUCT NAME | BEAUTY BED |
| NAZWA PRODUKTU | FOTEL KOSMETYCZNY |
| NÁZEV VÝROBKU | KOSMETICKÉ LEHÁTKO |
| NOM DU PRODUIT | LIT D'ESTHÉTIQUE |
| NOME DEL PRODOTTO | LETTINO ESTETISTA |
| NOMBRE DEL PRODUCTO | CAMILLA DE ESTÉTICA |
| MODELL | |
| PRODUCT MODEL | |
| MODEL PRODUKTU | PHYSA BERGAMO WHITE |
| MODEL VÝROBKU | PHYSA BERGAMO BLACK |
| MODÈLE | PHYSA BERGAMO DARK BEIGE |
| MODELLO | |
| MODELO | |
| IMPORTEUR | |
| IMPORTER | |
| IMPORTER | |
| DOVOZCE | EXPONDO POLSKA SP. Z O.O. SP. K. |
| IMPORTATEUR | |
| IMPORTATORE | |
| IMPORTADOR | |
| ADRESSE VON IMPORTEUR | |
| IMPORTER ADDRESS | |
| ADRES IMPORTERA | |
| ADRESA DOVOZCE | UL. NOWY KISIELIN-INNOWACYJNA 7, 66-002 ZIELONA GÓRA POLAND, EU |
| ADRESSE DE L'IMPORTATEUR | |
| INDIRIZZO DELL'IMPORTATORE | |
| DIRECCIÓN DEL IMPORTADOR | |

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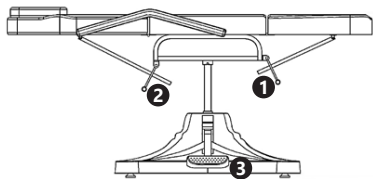
Read instructions before use.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details it may differ from the actual product.

The original operation manual is in German. Other language versions are translations from German.

1. Lever for adjusting the footrest angle
2. Lever for adjusting the backrest angle
3. Seat height adjustment lever

Assembly

Before use: assemble the product correctly and put it in the right place.

| | | | |
|---|--|-------|---|
| A | | M8x25 | 6 |
| B | | M8 | 6 |
| C | | M8 | 6 |

ATTENTION! This product's exploded view can be found on the last page of the operating instructions (pp. 10).

ATTENTION! Do not overload the device. Exceeding the maximum permissible user weight can damage the product.

Do not shake or hit the product because the seat may fall down and injure the user.

Do not place the product in direct sunlight as this may cause discoloration.

Backrest adjustment

- 1) Pull and hold the backrest adjustment lever.
- 2) Grip the backrest with your other hand and set the backrest at the desired angle.

NOTE: Hold the backrest handle by one hand in the course of pulling and holding the backrest adjustment lever. Otherwise, the backrest may fall uncontrollably and cause harm to the user.

Adjustment of the footrest.

- 1) Pull and hold the footrest adjustment lever.
- 2) Hold the footrest with your other hand and set it at the desired angle.

NOTE: Hold the footrest handle by one hand in the course of pulling and holding the footrest adjustment lever. Otherwise, the footrest may fall uncontrollably and cause harm to the user.

Seat height adjustment

Press the seat height adjustment lever in order to increase the seat height. Do not press the lever to the very end. In order to lower the seat height, it is necessary to press the lever to the very end and hold the lever. Release the lever when the bed is lowered to the desired height.

NOTE

Do not suddenly burden the device with a weight above 100kg. It can lead to damage of the device. The device is not used to store objects on it.

Cleaning and maintenance

- Use cleaners without corrosive substances to clean each surface.
- Note! The product should be washed after each use. It is recommended to use a mild disinfectant for this purpose.
- Fabrics with a different colour than the upholstery may cause discoloration on the product.
- Use a soft cloth for cleaning.
- Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.

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