USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING

PHYSA MARSEILLE WHITE
PHYSA MARSEILLE BLACK
PHYSA MARSEILLE BEIGE
PHYSA MARSEILLE RED
PHYSA MARSEILLE BLUE
PHYSA TOULOUSE WHITE
PHYSA TOULOUSE BLACK
PHYSA TOULOUSE BEIGE
PHYSA TOULOUSE RED
PHYSA TOULOUSE BLUE
PHYSA MONTPELLIER WHITE
PHYSA MONTPELLIER BLACK
PHYSA MONTPELLIER BEIGE
PHYSA MONTPELLIER RED
PHYSA MONTPELLIER BLUE
PHYSA BORDEAUX WHITE
PHYSA BORDEAUX BLACK
PHYSA BORDEAUX BEIGE
PHYSA BORDEAUX RED
PHYSA BORDEAUX BLUE
PHYSA BORDEAUX TURQUISE

FOLDING MESSAGE BED

DE	4
EN	9
PL	14
CZ	19
FR	24
IT	29
ES	34
HU	39
DA	44
FI	49
NL	54
NO	59
SE	64
PT	69

PRODUKTNAME	KLAPPBARES MASSAGEBETT
PRODUCT NAME	FOLDING MESSAGE BED
NAZWA PRODUKTU	SKŁADANE ŁÓŻKO DO MASAŻU
NÁZEV VÝROBKU	SKLÁDACÍ MASÁŽNÍ LEHÁTKO
NOM DU PRODUIT	TABLE DE MASSAGE PLIANTE
NOME DEL PRODOTTO	LETTINO DA MASSAGGIO PIEGHEVOLE
NOMBRE DEL PRODUCTO	CAMILLA DE MASAJES PLEGABLE
TERMÉKNÉV	ÖSSZECUKHATÓ MASSÁZSÁGY
PRODUKTNAMN	SAMMENKLAPPELIG MESSAGESENG
TUOTTEEN NIMI	KOKOONTAITETTAVA HIERONTAPÖYTÄ
PRODUCTNAAM	OPVOUWBAAR MESSAGEBED
PRODUKTNAMN	SAMMENLEGGBAR MASSASJEBENK
PRODUKTNAMN	HOPFÄLLBAR MESSAGEBÄNK
NOME DO PRODUTO	MESA DE MASSAGENS DOBRÁVEL
MODELL	PHYSA MARSEILLE WHITE
PRODUCT MODEL	PHYSA MARSEILLE BLACK
MODEL PRODUKTU	PHYSA MARSEILLE BEIGE
MODEL VÝROBKU	PHYSA MARSEILLE RED
MODÈLE	PHYSA MARSEILLE BLUE
MODELLO	PHYSA TOULOUSE WHITE
MODELO	PHYSA TOULOUSE BLACK
TERMÉKMODELL	PHYSA TOULOUSE BEIGE
PRODUKTMODEL	PHYSA TOULOUSE RED
TUOTTEEN MALLI	PHYSA TOULOUSE BLUE
PRODUCTMODEL	PHYSA MONTPELLIER WHITE
PRODUKTMODELL	PHYSA MONTPELLIER BLACK
PRODUKTMODELL	PHYSA MONTPELLIER BEIGE
MODELO DO PRODUTO	PHYSA MONTPELLIER RED
	PHYSA MONTPELLIER BLUE
	PHYSA BORDEAUX WHITE
	PHYSA BORDEAUX BLACK
	PHYSA BORDEAUX BEIGE
	PHYSA BORDEAUX RED
	PHYSA BORDEAUX BLUE
	PHYSA BORDEAUX TURQUISE
HERSTELLER / ANSCHRIFT DES HERSTELLERS MANUFACTURER / MANUFACTURER ADDRESS PRODUCENT / ADRES PRODUCENTA VÝROBCE / ADRESA VÝROBCE FABRICANT / ADRESSE DU FABRICANT PRODUTTORE / INDIRIZZO DEL PRODUTTORE FABRICANTE / DIRECCIÓN DEL FABRICANTE GYÁRTÓ / GYÁRTÓ CÍME PRODUCENT / PRODUCENTENS ADRESSE VALMISTAJA / VALMISTAJAN OSOITE PRODUCENT / ADRES PRODUCENT PRODUSENT / PRODUSENTENS ADRESSE TILLVERKARE / TILLVERKARENS ADRESS PRODUTOR / ENDEREÇO DO PRODUTOR	EXPONDO POLSKA SP. Z O.O. SP. K. UL. NOWY KISIELIN-INNOWACYJNA 7 66-002 ZIELONA GÓRA ZIELONA GÓRA POLAND, EU

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
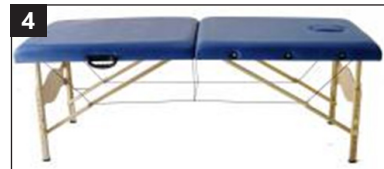
3.2.2. MONTAGE

MODELLE: PHYSA MARSEILLE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA TOULOUSE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA BORDEAUX WHITE, (BLACK, BEIGE, RED, BLUE, TURQUoise)

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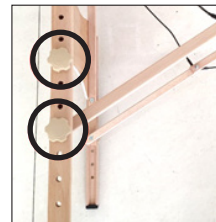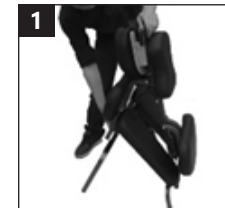
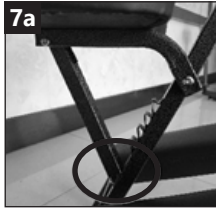
Product name	FOLDING MESSAGE BED		
Model	PHYSA BORDEAUX WHITE	PHYSA BORDEAUX BLACK	
Maximum load [kg]	180		
Table height adjustment range [mm]	590-800		
Dimensions [mm]	1850x600x590-800		
Weight [kg]	14		

Product name	FOLDING MESSAGE BED	
Model	PHYSA BORDEAUX BEIGE	PHYSA BORDEAUX RED
Maximum load [kg]	180	
Table height adjustment range [mm]	600-810	
Dimensions [mm]	2110x810 x690-810	2080x805 x730-810
Weight [kg]	13,4	
Product name	FOLDING MESSAGE BED	
Model	PHYSA BORDEAUX BLUE	PHYSA BORDEAUX TURQUISE
Maximum load [kg]	180	
Table height adjustment range [mm]	590-800	
Dimensions [mm]	2135x810 x700-810	2115x800 x735-810
Weight [kg]	13,5	13,3

1. GENERAL DESCRIPTION

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement.

LEGEND

	Read instructions before use.
	WARNING! or CAUTION! or REMEMBER! Applicable to the given situation (general warning sign).
	ATTENTION! Hand crush hazard!
	Only use indoors.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details may differ from the actual product. The original operation manual is in German. Other language versions are translations from German.

2. USAGE SAFETY

ATTENTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in an electric shock, fire and/or serious injury or even death.

The terms „device“ or „product“ are used in the warnings and instructions to refer to FOLDING MESSAGE BED.

2.1. SAFETY GUIDELINES

- Only the manufacturer's service centre may make repairs to the product. Do not attempt to make repairs yourself!
- Do not use the device when tired, ill or under the influence of alcohol, narcotics or medication which can significantly impair the ability to operate the device.
- The device is not designed to be handled by persons (including children) with limited mental and sensory functions or persons lacking relevant experience and/or knowledge unless they are supervised by a person responsible for their safety or they have received instruction on how to operate the device.
- The device is not a toy. Children must be supervised to ensure that they do not play with the device.
- Keep the device in perfect technical condition. Before each use check for general damage and especially check for cracked parts or elements and for any other conditions which may impact the safe operation of the device. If damage is discovered, hand over the device for repair before use.
- Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- To ensure the operational integrity of the device, do not remove factory-fitted guards and do not loosen any screws.
- When transporting and handling the device between the warehouse and the destination, observe the occupational health and safety principles for manual transport operations which apply in the country where the device will be used.
- Do not move, adjust or rotate the device in the course of work.
- Clean the device regularly to prevent stubborn grime from accumulating.
- The device is not a toy. Cleaning and maintenance may not be carried out by children without supervision by an adult person.
- Keep the device away from sources of fire and heat.
- Do not overload the device.
- Exceeding the maximum permissible user weight can damage the product.
- Before using the product, make sure that all legs are the same height. Stretching the legs of the bed to different heights can make it unstable.

ATTENTION! Despite the safe design of the device and its protection facilities and despite the use of additional elements protecting the operator, there is still a slight risk of an accident or injury when using the device. When using the device stay alert and use common sense.

3. USE GUIDELINES

The product is an accessory for massage.

The user is liable for any damage resulting from unintended use of the device.

3.1. DEVICE DESCRIPTION

ATTENTION! This product's view can be found on the last pages of the operating instructions (pp. 74).

- Handle
- Button
- Hole for armrests
- Face cradle
- Headrest
- Front arm shelf
- Armrest
- Rope tension
- Knee rest
- The seat

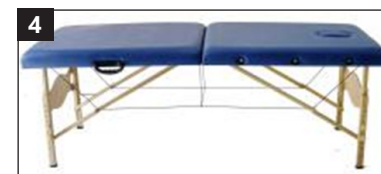
3.2. PREPARING FOR USE

3.2.1. APPLIANCE LOCATION


Keep the product away from hot surfaces. The device should always be used when positioned on an even, stable, clean, fireproof and dry surface, and be out of the reach of children and persons with limited mental and sensory functions.

3.2.2. ASSEMBLY


MODELS: PHYSA MARSEILLE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA TOULOUSE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA BORDEAUX WHITE, (BLACK, BEIGE, RED, BLUE, TURQUISE)

- Release the safety device.
- Lay the table on its side and stretch the table legs to the desired height.
 - MODELS: PHYSA MARSEILLE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA TOULOUSE WHITE, (BLACK, BEIGE, RED, BLUE)
 To adjust the table height, unscrew the knobs, remove the movable leg and insert it into the holes at the desired height, then screw the knobs back on. Note: Make sure the legs are the same height.

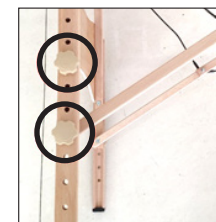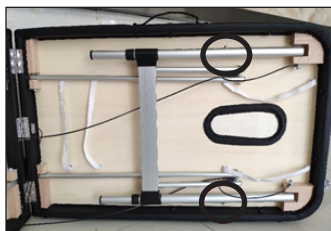- MODELS: PHYSA BORDEAUX WHITE, (BLACK, BEIGE, RED, BLUE, TURQUISE)
To adjust the table height, press the button on its leg and set the appropriate height. Note: Make sure the legs are the same height.



- Remove all accessories from inside.
- Lift the table and place its legs on the floor.
- Install the armrests.
- Install the front arm-shelf on the headrest holder.
- Adjust the headrest and lock it.
- Correctly assembled table.

NOTE: the PHYSA MARSEILLE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA BORDEAUX WHITE, (BLACK, BEIGE, RED, BLUE, TURQUISE) models have an additionally adjustable leg rest.

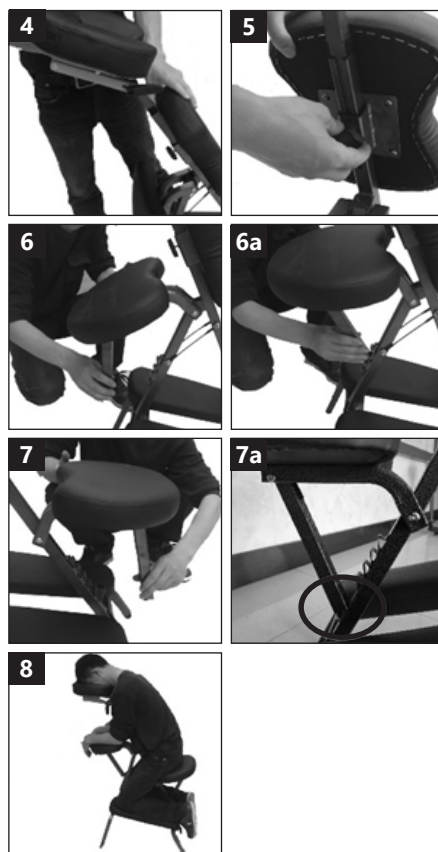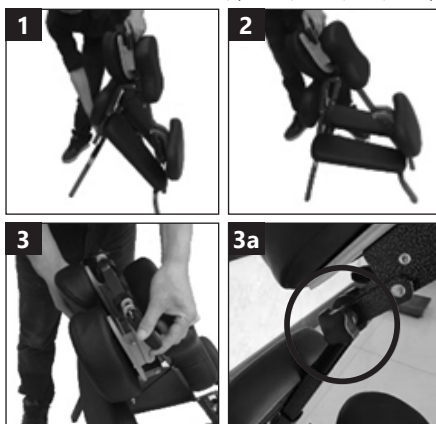PHYSA MONTPELLIER WHITE, (BLACK, BEIGE, RED, BLUE)

- Set up the bottom of the chair.
- Make sure the chair is firmly on the floor.
- Unlock the adjustable handle and fold out the headrest. Unlock the knob and the handle to change the angle of the headrest. Adjust the headrest to the desired height and lock it.
- Adjust the chest-rest position.
- Lock the chest-rest position with the knob.
- Fold out the arm shelf and lock it at the selected position.
- Fold out the seat and lock the bracket.
- Chair properly assembled.

3.2.3. DISASSEMBLY

MODELS: PHYSA MARSEILLE WHITE, (BLACK, BEIGE, RED, BLUE) PHYSA TOULOUSE WHITE, (BLACK, BEIGE, RED, BLUE), PHYSA BORDEAUX WHITE, (BLACK, BEIGE, RED, BLUE, TURQUISE)

- Remove all accessories (headrest, armrest, front armrest).
- Lay the table on its side.
- Place the accessories under the elastic strap.
- Fold the table legs, then fold the mattress and lock the safety device.

3.3. CLEANING AND MAINTENANCE

- Use only non-corrosive cleaners to clean the surface.
- After cleaning the device, all parts should be dried completely before using it again.
- Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight. It is recommended to store the product in the case and not to rest it against objects that could deform it.
- Use a soft, damp cloth for cleaning.
- Do not use sharp and/or metal objects for cleaning (e.g. a wire brush or a metal spatula) because they may damage the surface material of the appliance.
- Note! The product should be washed after each use. It is recommended to use a mild disinfectant for this purpose.
- Do not use cleaners containing liquid dye as they can discolour the upholstery.
- Do not wax or use detergents and / or leather cleaning agents as these may damage the coating.
- Do not use alcohol-based cleaners to clean the product.

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