USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

ELECTRIC BEAUTY BED

DE	Produktname	Kosmetikliege
EN	Product name	Beauty bed
PL	Nazwa produktu	Fotel kosmetyczny
CZ	Název výrobku	Kosmetické lehátko
FR	Nom du produit	Table de soins esthétique
IT	Nome del prodotto	Lettino estetico
ES	Nombre del producto	Camilla estética
HU	Termék neve	Kozmetikai kezelőágy
DA	Produktnavn	Kosmetikbriks
FI	Tuotteen nimi	Kosmetiikkahoitosänky
NL	Productnaam	Schoonheidsbehandelbed
NO	Produktnavn	Kosmetisk behandlingsbenk
SE	Produktnamn	Kosmetisk behandlingsbänk
PT	Nome do produto	Marquesa estética
SK	Názov produktu	Kozmetické lehátko
BG	Име на продукта	Козметично легло
EL	Όνομα προϊόντος	Κρεβάτι αισθητικής
HR	Naziv proizvoda	Kozmetički ležaj
LT	Produkto pavadinimas	Kosmetologinė lova
RO	Numele produsului	Pat cosmetic
SL	Ime izdelka	Kozmetična miza
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		PHYSA ROME PISTACHIO PHYSA VELLETRI WHITE PHYSA GENOA WHITE PHYSA ERCOLANO GREY PHYSA ACIREALE POWDER PINK PHYSA ROVIGO BLACK PHYSA PORDENONE POWDER PINK PHYSA BATTIPAGLIA WHITE PHYSA RHO WHITE P HYSA CHIETI CREAM PHYSA SCAFATI POWDER PINK
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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Technical data

Description of parameter	Parameter value			
Name of the product	Electric Beauty Bed			
Model	PHYSA VELLETRI WHITE	PHYSA GENOA WHITE	PHYSA ACIREALE POWDER PINK	PHYSA ROVIGO BLACK
Supply voltage [V~] / Frequency [Hz]	230/50			
Rated power [W]	350			300
Dimensions [width x depth x height; mm]	2070x800x670 -1115	1800x860x680 -910	1810x790x650 -910	1920x840x700 -905
Weight [kg]	83.2	76.9	67	70.2
Maximum working load [kg]	150			
Backrest angle adjustment [°]	0 - 71	0 - 84	0 - 65	0 - 77
Footrest adjustment [°]	0 - 80	0 - 82	0 - 77	0 - 80

Description of parameter	Parameter value
--------------------------	-----------------

Name of the product	Electric Beauty Bed				
Model	PHYSA PORDENONE POWDER PINK	PHYSA BATTIPAGLIA WHITE	PHYSA RHO WHITE	PHYSA CHIETI CREAM	PHYSA SCAFATI POWDER PINK
Supply voltage [V~] / Frequency [Hz]	230/50				
Rated power [W]	350		300	350	
Dimensions [width x depth x height; mm]	1905x800x670-870	1835x900x640-890	2070x830x615-935	1800x850x650-975	1820x830x650-910
Weight [kg]	69	75.2	70.7	80	94
Maximum working load [kg]	150				
Backrest angle adjustment [°]	0 - 90	0 - 81	78	0 - 72	69
Footrest adjustment [°]	0 - 46	0 - 47	42	0 - 85	74

Description of parameter	Parameter value
--------------------------	-----------------

Name of the product	Pedicure chair	
Model	PHYSA ROME PISTACHIO	PHYSA ERCOLANO GREY
Supply voltage [V~] / Frequency [Hz]	230/50	
Rated power [W]	300	
Dimensions [width x depth x height; cm]	196 x 78 x 116	182 x 83 x 134
Weight [kg]	85.9	87
Maximum working load [kg]	150	
Backrest angle adjustment [°]	100 - 180	
Footrest adjustment [°]	0 - 70	

ATTENTION: The order in which the technical performance values are listed in the table above corresponds to the order in which the models are listed.

1. General description

These instructions are intended to assist you in safe and reliable use. The product is designed and manufactured strictly according to technical specifications using the latest technology and components and maintaining the highest quality standards.

READ THE INSTRUCTIONS CAREFULLY AND UNDERSTAND THEM BEFORE USE.

To ensure long and reliable operation of the device, care should be taken to operate and maintain the device correctly in accordance with the instructions. The technical data and specifications contained in these instructions are up-to-date. The manufacturer reserves the right to make changes for the purpose of quality improvement.

Explanation of symbols

	The product meets the requirements of relevant safety standards.
	Read the instructions before use.
	Recyclable product.
	CAUTION! or WARNING! or REMEMBER! describing a given situation (general warning sign).
	CAUTION! Risk of electric shock!
	For indoor use only.

CAUTION! The illustrations in this manual are for illustrative purposes only and may differ in some details from the actual product appearance.

The original version of the instructions is the German version. Other language versions are translations from German.

2. Safe use

CAUTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in electric shock, fire and/or serious injury or death.

The term "device" or "product" in the warnings and instructions refers to the beauty bed.

Electrical safety

- a) The device plug must fit into the socket. Do not modify the plug in any way. Original plugs and matching sockets reduce the risk of electric shock.
- b) Avoid touching earthed items such as tubes, radiators, heaters and refrigerators. There is an increased risk of electric shock if your body is earthed and touches the device exposed to direct rain, wet floor and humid environment. If water penetrates into the device, there is an increased risk of damage to the device and electric shock.
- c) Do not touch the device with wet or damp hands.
- d) Do not use the cable in a wrong way. Never use it to carry the device or to pull the plug out of the socket. Keep the cable away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cables increase the risk of electric shock.
- e) If you cannot avoid using the device in a wet environment, use a residual current device (RCD). Using an RCD reduces the risk of electric shock.
- f) Do not use the device if the power cable is damaged or shows clear signs of wear. A damaged power cable should be replaced by a qualified electrician or the manufacturer's service.
- g) To avoid electric shock, do not immerse the cable, plug or device itself in water or other liquid. Do not use the device on wet surfaces.
- h) CAUTION – DANGER TO LIFE! When cleaning or using the device, never immerse it in water or other liquids.
- i) Do not use the device in high humidity rooms or in a direct vicinity to a water tanks.

2.2. Safety in the work area

- a) Keep the work area tidy and well lit. Disorder or poor lighting can lead to accidents. Be careful, watch what you are doing and use common sense when working with the device.
- b) If in doubt as to whether the device is working properly, contact the manufacturer's service.
- c) Repairs to the device may only be carried out by the manufacturer's service. Do not carry out repairs yourself!
- d) In the event of fire or fire, use only powder or snow (CO₂) extinguishers to extinguish fire in the device where there is voltage.
- e) Check the condition of the safety stickers regularly. Replace them if they are illegible.

-
- f) Keep these instructions for future reference. If the device is to be passed on to third parties, the instructions for use must be handed over together with the device.
 - g) Keep packaging parts and small assembly parts out of the reach of children.
 - h) Keep the device away from children and animals
 - i) When using this device together with other devices, the other instructions for use must also be followed.

Remember! Keep children and other bystanders safe while operating the device.

2.3. Personal safety

- a) Do not operate the device if you are tired, ill or under the influence of alcohol, drugs or medication which might impair your ability to operate the device.
- b) The device is not intended to be used by persons (including children) with reduced mental, sensory or intellectual functions or lack of experience and/or knowledge, unless they are supervised by a person responsible for their safety or have been given instructions by them on how to operate the device.
- c) The device may only be operated by persons who are physically fit, capable of handling the device and who have been adequately trained, who have read these instructions and have received training in health and safety.
- d) Be careful and use common sense when operating the device. A short moment of distraction during operation may lead to serious personal injury.
- e) To prevent accidental start-up, make sure the switch is in the OFF position before connecting the device to the power source.
- f) The device is not a toy. Children should be supervised to ensure that they do not play with the device.

2.4. Safe use of the device

- a) Do not overload the device. Use tools that are suitable for the given application. A correctly selected device ensures better and safer performance of work.
- b) Do not use the device if the ON/OFF switch does not function properly (does not switch on and off).
- c) The device must be disconnected from the power supply before adjustment, cleaning and maintenance. This preventive measure reduces the risk of accidental activation.

-
- d) Keep unused devices out of the reach of children and anyone not familiar with the device or these instructions. Devices are dangerous in the hands of inexperienced users.
 - e) Keep the device in good working order.
 - f) Keep the device out of the reach of children.
 - g) Repair and maintenance of the device should be carried out by qualified persons using only original spare parts. This will ensure a safe use of the device.
 - h) To ensure the designed operational integrity of the device, do not remove factory-installed covers or loosen any screws.
 - i) When transporting and moving the device from the place of storage to the place of use, follow the health and safety regulations for manual transport that apply in the country in which the device is used.
 - j) Avoid situations where the device stops under heavy load during operation. This can cause overheating of the drive elements and subsequent damage to the device.
 - k) Clean the device regularly to prevent the permanent accumulation of dirt.
 - l) The device is not a toy. Cleaning and maintenance must not be performed by children without adult supervision.
 - m) It is forbidden to tamper with the device in order to change its parameters or construction.
 - n) Keep the device away from sources of fire and heat, direct sunlight and moisture.
 - o) Exceeding the maximum permissible user weight may cause damage to the product.
 - p) Do not sit at the end of the chair. This may cause the product to fall over.
 - q) The weight must be distributed evenly on the chair to avoid it tipping over.
 - r) Do not stand on the chair.
 - s) Check from time to time that the screws have not come loose. Tighten if necessary.
 - t) Regularly lubricate the moving parts of the device which are used to adjust the position.
 - u) The device should be set up at least 80 cm from the nearest wall or other obstruction to ensure free movement around the device.
 - v) Do not use the device to store other objects on it.
 - w) Do not apply mechanical pressure to the device.
 - x) Do not obstruct the headrest, footrest or backrest during adjustment.
 - y) The devices have motors that should run for 2 minutes, after which the device needs an 18 minute break.

CAUTION! Although the device has been designed to be safe and has been equipped with adequate safeguards, and despite the use of additional safety measures, there is still a small risk of accident

or injury when operating the device. Caution and common sense are advised when using the device.

3. Rules of use

The beauty bed is intended for use during cosmetic treatments. The product has an electrically adjustable position. The product is outside the scope of MDD 93/42/EEC for medical devices.

The user is responsible for any damage caused by improper use.

Description of the device

PHYSA ROME PISTACHIO / PHYSA ERCOLANO GREY / PHYSA SCAFATI POWDER PINK MODELS:

PHYSA ACIREALE POWDER PINK / PHYSA ROVIGO BLACK / PHYSA VELLETRI WHITE /
PHYSA GENOA WHITE / PHYSA BATTIPAGLIA WHITE / PHYSA RHO WHITE / PHYSA
CHIETI CREAM MODELS:

1. Headrest
2. Backrest
3. Armrest
4. Seat
5. Remote control
6. Base
7. Footrest

3.2. Preparation for work

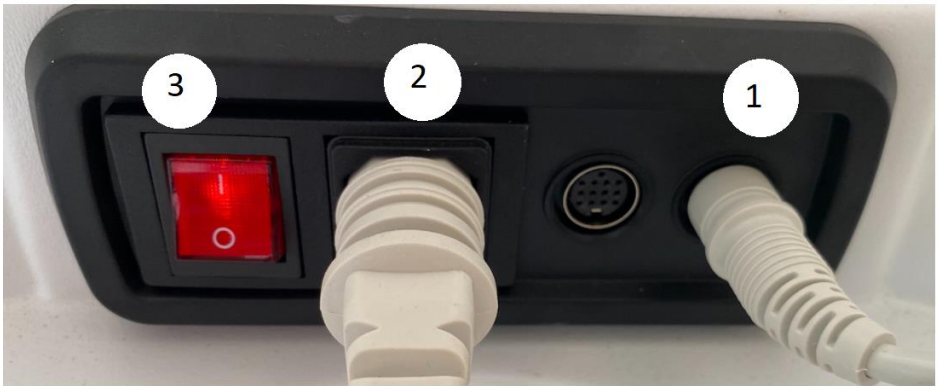
LOCATION OF THE DEVICE

The ambient temperature should not exceed 40°C and the relative humidity should not exceed 85%. Keep the device away from any hot surfaces. Always operate the device on a level, stable, clean, fireproof and dry surface and out of the reach of children and persons with reduced mental, sensory and intellectual abilities. Position the device in such a way that the power supply plug can be reached at any time. Make sure that the device's power supply corresponds to the data on the rating plate!

3.3. Working with the device

3.3.1. Starting the device:

- a) Unpack the device, remove the foil. Check that the device is not damaged.
- b) Insert the end of the remote control cable into the socket on the device [1].
- c) Insert the end of the power cord into the socket on the device [2].
- d) Plug the plug of the power cord into an electrical outlet.
- e) Use the ON/OFF switch [3] to switch the device on (switch to "I" position).

3.3.2. Description of the remote control functions:

The positions of the device are adjusted using the remote control. Each button is responsible for a different control function.

PHYSA ROME PISTACHIO / PHYSA ERCOLANO GREY / PHYSA RHO WHITE MODELS:

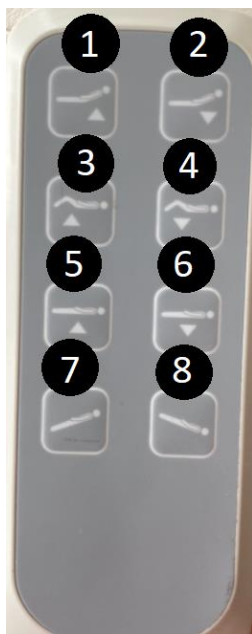
1. Lifting the backrest
2. Lowering the backrest
3. Lifting the seat
4. Lowering the seat
5. Lifting the device
6. Lowering the device

PHYSA RHO WHITE MODEL:

3. Lifting the footrest
4. Lowering the footrest

PHYSA VELLETRI WHITE / PHYSA GENOA WHITE / PHYSA ACIREALE POWDER PINK /
PHYSA ROVIGO BLACK / PHYSA PORDENONE POWDER PINK / PHYSA BATTIPAGLIA
WHITE / PHYSA CHIETI CREAM / PHYSA SCAFATI POWDER PINK MODELS:

1. Lifting the backrest
2. Lowering the backrest
3. Lifting the footrest
4. Lowering the footrest
5. Lifting the device
6. Lowering the device
7. Lifting the seat
8. Lowering the seat

PHYSA ROME PISTACHIO / PHYSA ERCOLANO GREY models have a gas spring which is responsible for adjusting the position of the footrest:

1. Depress the gas spring which is located under the footrest to deepen the angle of the footrest.

-
2. Lift the spring to return the footrest to its original position.

3.3.3. Ending work with the device:

- a) When finished working, switch off the device using the ON/OFF switch.
- b) Remove the mains plug from the power socket.

3.4. Cleaning and maintenance

- a) Before cleaning and when the device is not in use, pull the power supply plug.
- b) Use only non-corrosive agents for cleaning the surface.
- c) Store the device in a dry and cool place protected from moisture and direct sunlight.
- d) Do not spray the device with a stream of water and do not immerse it in water.
- e) Regularly inspect the device for technical failure and damage. If necessary, replace worn parts.
- f) Use a soft, damp cloth for cleaning.
- g) Do not use sharp and/or metal objects for cleaning (e.g. a wire brush or metal scraper) as these may damage the surface of the device.
- h) Do not clean the device with oily substances.

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