USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES

PHYSA LYON WHITE
PHYSA LYON BEIGE
PHYSA LYON BLACK
PHYSA NANTES WHITE
PHYSA NANTES DARK BLUE
PHYSA NANTES BLACK
PHYSA NANTES LIGHT GREEN
PHYSA LYON WHITE_PH
PHYSA LYON BEIGE_PH
PHYSA LYON BLACK_PH
PHYSA NANTES WHITE_PH
PHYSA NANTES TURQUOISE_PH
PHYSA NANTES BLACK_PH
PHYSA NANTES LIGHT GREEN_PH
PHYSA LYON GRAY_PH
PHYSA NANTES BEIGE_PH
PHYSA NANTES BURGUNDY_PH

MESSAGE TABLE

| | |
|----|----|
| DE | 3 |
| EN | 8 |
| PL | 13 |
| CZ | 18 |
| FR | 23 |
| IT | 28 |
| ES | 31 |

| | |
|---------------------------|--|
| PRODUKTNAME | MASSAGELIEGE |
| PRODUCT NAME | MASSAGE TABLE |
| NAZWA PRODUKTU | STÓŁ DO MASAŻU |
| NÁZEV VÝROBKU | MASÁŽNÍ LEHÁTKO |
| NOM DU PRODUIT | TABLE DE MASSAGE |
| NOME DEL PRODOTTO | LETTINO MASSAGGIO |
| NOMBRE DEL PRODUCTO | CAMILLA DE MASAJE |
| MODELL | PHYSA LYON WHITE PHYSA LYON BEIGE PHYSA LYON BLACK PHYSA NANTES WHITE PHYSA NANTES DARK BLUE PHYSA NANTES BLACK PHYSA NANTES LIGHT GREEN PHYSA LYON WHITE_PH PHYSA LYON BEIGE_PH PHYSA LYON BLACK_PH PHYSA NANTES WHITE_PH PHYSA NANTES TURQUOISE_PH PHYSA NANTES BLACK_PH PHYSA NANTES LIGHT GREEN_PH PHYSA LYON GRAY_PH PHYSA NANTES BEIGE_PH PHYSA NANTES BURGUNDY_PH |
| PRODUCT MODEL | |
| MODEL PRODUKTU | |
| MODEL VÝROBKU | |
| MODÈLE | |
| MODELLO | |
| MODELO | |
| HERSTELLER | |
| MANUFACTURER | |
| PRODUCENT | |
| VÝROBCE | EXPONDO POLSKA SP. Z O.O. SP. K. |
| FABRICANT | |
| PRODUTTORE | |
| FABRICANTE | |
| ANSCHRIFT DES HERSTELLERS | |
| MANUFACTURER ADDRESS | |
| ADRES PRODUCENTA | |
| ADRESA VÝROBCE | UL. NOWY KISIELIN-INNOWACYJNA 7, 66-002 ZIELONA GÓRA POLAND, EU |
| ADRESSE DU FABRICANT | |
| INDIRIZZO DEL PRODUTTORE | |
| DIRECCIÓN DEL FABRICANTE | |

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| Parameter – Beschreibung | Parameter – Wert | |
|--|---|--|
| Produktname | Massageliege | |
| Modell | PHYSA LYON WHITE PHYSA LYON BEIGE PHYSA LYON BLACK PHYSA LYON WHITE_PH PHYSA LYON BEIGE_PH PHYSA LYON BLACK_PH PHYSA LYON GRAY_PH | PHYSA NANTES WHITE PHYSA NANTES DARK BLUE PHYSA NANTES BLACK PHYSA NANTES LIGHT GREEN PHYSA NANTES WHITE_PH PHYSA NANTES TURQUOISE_PH PHYSA NANTES TURQUOISE_PH PHYSA NANTES BLACK_PH PHYSA NANTES LIGHT GREEN_PH PHYSA NANTES BEIGE_PH PHYSA NANTES BURGUNDY_PH |
| Nennspannung [V~/]/ Frequenz [Hz] | 230/50 | |
| Nennleistung [W] | 100 | |
| Schutzklasse | I | |
| IP schutzklasse | IPX0 | |
| Arbeitsmodus | 2 min „ON“/ 18 min „OFF“ | |
| Maximale Belastbarkeit [kg] | 200 | |
| Maximaler Neigungswinkel des Sitzes | – | 10° |
| Maximaler Neigungswinkel der Rückenlehne | 60° | 55° |
| Tischhöhenverstellbereich [mm] | 680-900 | 640-868 |
| Abmessungen HxBxT [mm] | (680-900) x1870x730 | (640-868) x1855x665 |
| Gewicht [kg] | 63 | 70 |

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1. GENERAL DESCRIPTION

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure a trouble-free operation, use it in accordance with this user manual and regularly perform the maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement.

LEGEND

| | |
|--|---|
| | The product satisfies the relevant safety standards. |
| | Read instructions before use. |
| | The product must be recycled. |
| | WARNING! or CAUTION! or REMEMBER! Applicable to the given situation (general warning sign). |
| | Only use inside. |
| | Protection class I device with a protective terminal. |

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details it may differ from the actual product.

The original operation manual is in German. Other language versions are translations from German.

2. USAGE SAFETY

Electrical equipment:

ATTENTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in an electric shock, fire and/or serious injury or even death.

Whenever „device“ or „product“ are used in the warnings and instructions, it shall mean massage table. Do not use in very humid environments or in the direct vicinity of water tanks. Prevent the device from getting wet. Risk of electric shock!

2.1. ELECTRICAL SAFETY

- The plug has to fit the socket. Do not modify the plug in any way. Using original plugs and matching sockets reduces the risk of electric shock.
- Avoid touching earthed elements such as pipes, heaters, boilers and fridges. There is an increased risk of electric shock if the earthed device is exposed to rain, in direct contact with a wet surface or operating in a humid environment. Water ingress into the device increases the risk of damage to the device and of electric shock.
- Do not touch the device with wet or damp hands

- Use the cable only in accordance with its designated use. Never use it to carry the device or to pull the plug out of a socket. Keep the cable away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cables increase the risk of electric shock.
- If using the device in a humid environment cannot be avoided, a residual current device (RCD) should be applied. The use of an RCD reduces the risk of electric shock.

2.2. SAFETY AT THE WORKPLACE

- Make sure the workplace is orderly and well lit. A messy or poorly lit workplace may lead to accidents. Try to anticipate what may happen, observe what is going on and use common sense when working with the device.
- Do not use the device in an explosion hazard zone, for example in the presence of flammable liquids, gasses or dust. The device generates sparks which may ignite dust or fumes.
- Upon discovering damage or irregular operation, immediately switch the device off and report it to a supervisor without delay.
- If there are any doubts as to the correct operation of the device, contact the manufacturer's support service.
- Only the manufacturer's service point may repair the device. Do not attempt any repairs independently!
- If a fire starts, use solely powder or carbon dioxide (CO₂) fire extinguishers suitable for use on live devices to put it out.

2.3. PERSONAL SAFETY

- Do not use the device when tired, ill or under the influence of alcohol, narcotics or medication which can significantly impair the ability to operate the device.
- The machine is not designed to be handled by persons (including children) with limited mental and sensory functions or persons lacking relevant experience and/ or knowledge unless they are supervised by a person responsible for their safety or they have received instructions on how to operate the machine.
- To prevent the device from accidentally switching on, make sure the switch is on the OFF position before connecting to a power source.

2.4. SAFE DEVICE USE

- Do not overload the device. Exceeding the maximum permissible user weight can damage the product.
- Do not use the device if the ON/OFF switch does not function properly (does not switch the device on and off). Devices which cannot be switched on and off using the ON/OFF switch are hazardous, should not be operated and have to be repaired.
- When not in use, store in a safe place, away from children and people not familiar with the device, who have not read the user manual. The device may pose a hazard in the hands of inexperienced users.
- Maintain the device in a good technical state. Before each use check for general damage and especially check for cracked parts or elements and for any other conditions which may impact the safe operation of the device. If damage is discovered, hand over the device for repair before use.

- Keep the device out of the reach of children.
- Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- To ensure the operational integrity of the device, do not remove factory fitted guards and do not loosen any screws.
- When transporting and handling the device between the warehouse and the destination, take into account the occupational health and safety principles for manual transport operations which apply in the country where the device will be used.
- It is forbidden to move, adjusting and rotating the device in the course of work.
- The device is not designed for store objects on it.
- Before use: assemble the product correctly and put it in the right place.
- Do not shake or hit the product because the seat may fall down and injure the user.
- Do not place the product in direct sunlight as this may cause discoloration.
- Do not use sharp devices when working with the product; sharp edges can damage the surface of the product.
- Operate the product in a dry environment, do not place the product in high humidity areas.

ATTENTION! Despite the safe design of the device and its protection facilities and despite the use of additional elements protecting the operator, there is still a slight risk of an accident or injury when using the device. When using the device stay alert and use common sense.

3. USE GUIDELINES

Massage table is an element of equipment used to perform massage. The product is outside the scope of MDD 93/42/EEC regulations for medical devices.

The user is liable for any damage resulting from nonintended use of the device.

3.1. DEVICE DESCRIPTION

PHHYSA LYON WHITE, PHHYSA LYON BEIGE, PHHYSA LYON BLACK, PHHYSA LYON WHITE_PH, PHHYSA LYON BEIGE_PH, PHHYSA LYON BLACK_PH, PHHYSA LYON GRAY_PH

PHYSA NANTES WHITE, PHYSA NANTES DARK BLUE
 PHYSA NANTES BLACK, PHYSA NANTES LIGHT GREEN
 PHYSA NANTES WHITE_PH, PHYSA NANTES TURQUOISE_PH,
 PHYSA NANTES TURQUOISE_PH, PHYSA NANTES BLACK_
 PH, PHYSA NANTES LIGHT GREEN_PH
 PHYSA NANTES BEIGE_PH, PHYSA NANTES BURGUNDY_PH

1. Disassembled headrest
- 1a. Place of assembly of the headrest
2. Adjustable backrest
3. Seat
4. Basis
5. Lock rod of wheels
6. Wheels
7. Adjustable armrests
8. Pilot hanger
9. Footrest


3.2. PREPARING FOR USE
 Appliance location

Put the device on a stable, dry and flat surface, taking care to keep the minimum space 0,8 m of free space around. Place the device close to the power supply source.

Installing the appliance

The products are equipped with a disassembled headrest, after dismantling there remains an opening that facilitates breathing in the course of lying on the stomach. The headrest is mounted by pressing it into the opening.

PHYSA LYON WHITE, PHYSA LYON BEIGE, PHYSA LYON BLACK, PHYSA LYON WHITE_PH, PHYSA LYON BEIGE_PH, PHYSA LYON BLACK_PH, PHYSA LYON GRAY_PH

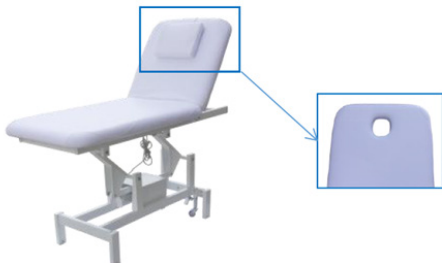PHYSA NANTES WHITE, PHYSA NANTES DARK BLUE
 PHYSA NANTES BLACK, PHYSA NANTES LIGHT GREEN
 PHYSA NANTES WHITE_PH, PHYSA NANTES TURQUOISE_PH,
 PHYSA NANTES TURQUOISE_PH, PHYSA NANTES BLACK_
 PH, PHYSA NANTES LIGHT GREEN_PH
 PHYSA NANTES BEIGE_PH, PHYSA NANTES BURGUNDY_PH


3.3.1. DEVICE USE

Handling

The device should be connected to a power supply compatible with the data given on the nameplate. The height and arrangement of the device is controlled by a remote control or a foot pedal. In order to start a given function, it is necessary to press and hold the button on the remote control or the pedal. Releasing the button or pedal will cause stop of the device in the set position.


Description of the functions of the remote control and the foot pedal:

PHYSA LYON WHITE, PHYSA LYON BEIGE, PHYSA LYON BLACK, PHYSA LYON WHITE_PH, PHYSA LYON BEIGE_PH, PHYSA LYON BLACK_PH, PHYSA LYON GRAY_PH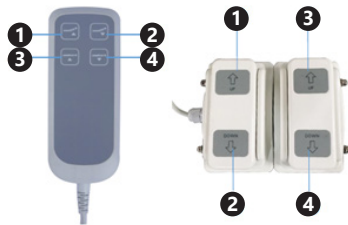

1. Backrest inclination angle: up
2. Backrest inclination angle: down
3. Seat height adjustment: up
4. Seat height adjustment: down

PHYSA NANTES WHITE, PHYSA NANTES DARK BLUE
 PHYSA NANTES BLACK, PHYSA NANTES LIGHT GREEN
 PHYSA NANTES WHITE_PH, PHYSA NANTES TURQUOISE_
 PH, PHYSA NANTES TURQUOISE_PH, PHYSA NANTES
 BLACK_PH, PHYSA NANTES LIGHT GREEN_PH
 PHYSA NANTES BEIGE_PH, PHYSA NANTES BURGUNDY_PH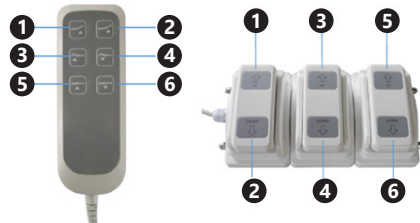

1. Backrest inclination angle: up
2. Backrest inclination angle: down
3. Tilt angle of the footrest: up
4. Tilt angle of the footrest: down
5. Seat height adjustment: up
6. Seat height adjustment: down

The operating mode of motors recommended by the manufacturer looks as follows: 2 minutes of operation, and then 18 minutes break. Failure to follow the recommendations can cause damage of the motor.


3.3.2. THE DEVICE AND ITS INTENDED CONTROL ELEMENT

| The main device model | Control element model |
|--|-----------------------|
| PHYSA LYON WHITE PHYSA LYON BEIGE PHYSA LYON BLACK PHYSA LYON WHITE_PH PHYSA LYON BEIGE_PH PHYSA LYON BLACK_PH PHYSA LYON GRAY_PH | PHY-FP-1 |
| PHYSA NANTES WHITE PHYSA NANTES DARK BLUE PHYSA NANTES BLACK PHYSA NANTES LIGHT GREEN PHYSA NANTES WHITE_PH PHYSA NANTES TURQUOISE_PH, PHYSA NANTES TURQUOISE_PH, PHYSA NANTES BLACK_PH PHYSA NANTES LIGHT GREEN_PH PHYSA NANTES BEIGE_PH PHYSA NANTES BURGUNDY_PH | PHY-FP-2 |

1. CONTROL CONNECTION DIAGRAM:


- I. The solution for a 2-motor device
- II. The solution for a 3-motor device

1) Control box: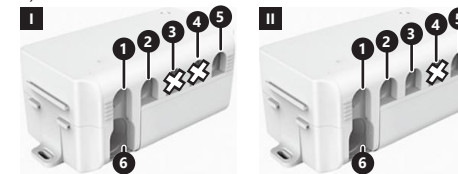


1. Motor No. 1 connection slot
2. Motor No. 2 connection slot
3. Motor No. 3 connection slot
4. Motor No. 4 connection slot
5. Control elements connection slot: remote control, foot pedal or splitter cable
6. Power cable connection slot

2) Motor numbering diagram:
 • Remote control: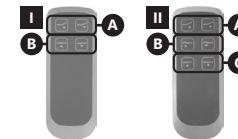


• Foot pedal:

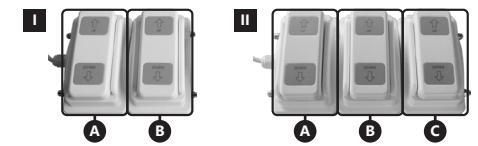
- A. Motor number 1
- B. Motor number 2
- C. Motor number 3

2. INSTALLING CONNECTORS IN THE BOX


- 1) Before installing or removing motor connectors, power cable, remote control or splitter cable, disconnect the power supply, and then remove the box cover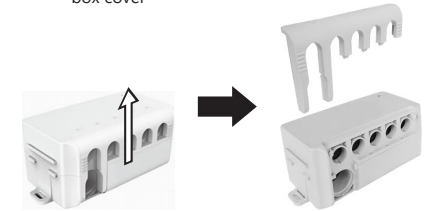

- 2) Plug in only one control connection (remote control or foot pedal) to the box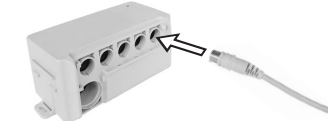

- 3) If you use a splitter cable, connect it to the box (the control slot), and then plug in the remote control and foot pedal to the splitter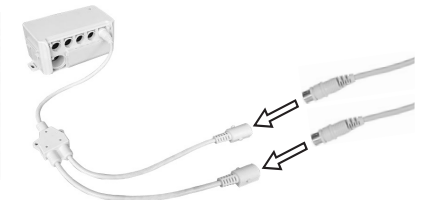


3.3.3. USAGE TIPS

- In order to ensure good hygiene of the workplace, it is advisable to use a removable cover. After every use the covers must be subjected to sterilization in order to guarantee cleanliness.
- The device should be systematically checked for technical efficiency, paying special attention to the condition of components exposed to faster wear, i.e.:
 - » screws,
 - » hanger,
 - » electric motor

Transportation

The wheels that facilitate transport to another place are mounted in the device. There must be no load on the table.

Grasp the lock rod of the wheels and press it until it jumps to the bottom position and the wheels will be lowered by raising the table legs up.

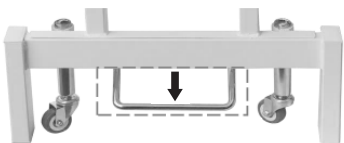The table can be transported carefully. In order to lock the wheels, it is necessary to set the rod in its original position.

3.4. CLEANING AND MAINTENANCE

- Always unplug the device before cleaning it.
- Use cleaners without corrosive substances to clean each surface.
- Note! The product should be washed after each use. It is recommended to use a mild disinfectant for this purpose.
- Never spray the device with water jets.
- Use a soft cloth for cleaning.
- Fabrics with a different color than the upholstery may cause discoloration of the product.
- After cleaning the device, all parts should be dried before reusing it.
- Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.
- Do not use cleaners containing liquid dye as they can discolour the upholstery.

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| Opis parametru | Wartość parametru | |
|---------------------------------------|--|---|
| Nazwa produktu | Stół do masażu | |
| Model | PHHYSA LYON WHITE PHHYSA LYON BEIGE PHHYSA LYON BLACK PHHYSA LYON WHITE_PH PHHYSA LYON BEIGE_PH PHHYSA LYON BLACK_PH PHHYSA LYON GRAY_PH | PHHYSA NANTES WHITE PHHYSA NANTES DARK BLUE PHHYSA NANTES BLACK PHHYSA NANTES LIGHT GREEN PHHYSA NANTES WHITE_PH PHHYSA NANTES TURQUOISE_PH PHHYSA NANTES TURQUOISE_PH PHHYSA NANTES BLACK_PH PHHYSA NANTES LIGHT GREEN_PH PHHYSA NANTES BEIGE_PH PHHYSA NANTES BURGUNDY_PH |
| Napięcie [V~/] Częstotliwość [Hz] | 230/50 | |
| Moc znamionowa [W] | 100 | |
| Klasa ochrony | I | |
| Klasa ochrony IP | IPX0 | |
| Cykl pracy | 2 min „ON” / 18 min „OFF” | |
| Maksymalne obciążenie [kg] | 200 | |
| Maksymalny kąt nachylenia siedziska | – | 10° |
| Maksymalny kąt nachylenia oparcia | 60° | 55° |
| Zakres regulacji wysokości stołu [mm] | 680-900 | 640-868 |
| Wymiary WxSxG [mm] | (680-900) x1870x730 | (640-868) x1855x665 |
| Waga [kg] | 63 | 70 |

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