# USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG  
INSTRUKCJA OBSŁUGI  
NÁVOD K POUŽITÍ  
MANUEL D'UTILISATION  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
HASZNÁLATI ÚTMUTATÓ  
BRUGSANVISNING  
KÄYTTÖOHJE  
GEBRUIKSAANWIJZING  
BRUKSANVISNING  
INSTRUÇÕES DE UTILIZAÇÃO  
POUŽÍVATEĽSKÁ PRÍRUČKA  
РЪКОВОДСТВО ЗА УПОТРЕБА  
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
UPUTE ZA UPORABU  
NAUDOJIMO INSTRUKCIJA  
MANUAL DE UTILIZARE  
NAVODILA ZA UPORABO

---

FOLDING MASSAGE BED

|    |                           |                                                                                                                                                                                               |
|----|---------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| DE | Produktname:              | KLAPPBARES MASSAGEBETT                                                                                                                                                                        |
| EN | Product name:             | FOLDING MESSAGE BED                                                                                                                                                                           |
| PL | Nazwa produktu:           | SKŁADANE ŁÓŻKO DO MASAŻU                                                                                                                                                                      |
| CZ | Název výrobku             | SKLÁDACÍ MASÁŽNÍ LEHÁTKO                                                                                                                                                                      |
| FR | Nom du produit:           | TABLE DE MASSAGE PLIANTE                                                                                                                                                                      |
| IT | Nome del prodotto:        | LETTINO DA MASSAGGIO PIEGHEVOLE                                                                                                                                                               |
| ES | Nombre del producto:      | CAMILLA DE MASAJES PLEGABLE                                                                                                                                                                   |
| HU | Termék neve               | ÖSSZECSUKHATÓ MASSZÁZSÁGY                                                                                                                                                                     |
| DA | Produktnavn               | SAMMENKLAPPELIG MASSAGESENG                                                                                                                                                                   |
| DE | Modell:                   | PHYSA COLMAR RED<br>PHYSA COLMAR BEIGE<br>PHYSA COLMAR BLUE<br>PHYSA COLMAR WHITE<br>PHYSA COLMAR BLACK<br>PHYSA VANNES BLACK<br>PHYSA VANNES WITHE<br>PHYSA DINAN BLACK<br>PHYSA DINAN WITHE |
| EN | Product model:            |                                                                                                                                                                                               |
| PL | Model produktu:           |                                                                                                                                                                                               |
| CZ | Model výrobku             |                                                                                                                                                                                               |
| FR | Modèle:                   |                                                                                                                                                                                               |
| IT | Modello:                  |                                                                                                                                                                                               |
| ES | Modelo:                   |                                                                                                                                                                                               |
| HU | Modell                    |                                                                                                                                                                                               |
| DA | Model                     |                                                                                                                                                                                               |
| DE | Hersteller                | expondo Polska sp. z o.o. sp. k.                                                                                                                                                              |
| EN | Manufacturer              |                                                                                                                                                                                               |
| PL | Producent                 |                                                                                                                                                                                               |
| CZ | Výrobce                   |                                                                                                                                                                                               |
| FR | Fabricant                 |                                                                                                                                                                                               |
| IT | Produttore                |                                                                                                                                                                                               |
| ES | Fabricante                |                                                                                                                                                                                               |
| HU | Termelő                   |                                                                                                                                                                                               |
| DA | Producent                 |                                                                                                                                                                                               |
| DE | Anschrift des Herstellers | ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra   Poland, EU                                                                                                                           |
| EN | Manufacturer Address      |                                                                                                                                                                                               |
| PL | Adres producenta          |                                                                                                                                                                                               |
| CZ | Adresa výrobce            |                                                                                                                                                                                               |
| FR | Adresse du fabricant      |                                                                                                                                                                                               |
| IT | Indirizzo del produttore  |                                                                                                                                                                                               |
| ES | Dirección del fabricante  |                                                                                                                                                                                               |
| HU | A gyártó címe             |                                                                                                                                                                                               |
| DA | Producentens adresse      |                                                                                                                                                                                               |

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## 1. General description

The manual is intended to assist in safe and reliable use. The product is designed and manufactured strictly according to technical specifications using the latest technology and components, and maintaining the highest quality standards.

**CAREFULLY READ AND UNDERSTAND THIS MANUAL  
BEFORE STARTING THE WORK.**

In order to ensure long and reliable operation of the device, it is necessary to take care of its correct operation and maintenance in accordance with the instructions of this manual. The technical data and specifications in this manual are up to date. The manufacturer reserves the right to make changes in order to improve quality. Taking

into account technical progress and the possibility of reducing noise, the device has been designed and built in such a way that the risk resulting from noise emission is limited to the lowest level.

## Symbol explanation

|                                                                                   |                                                                                                             |
|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------|
|  | Please read the manual before use.                                                                          |
|  | <b>CAUTION!</b> or <b>WARNING!</b> or <b>REMEMBER!</b> describing a given situation (general warning sign). |
|  | <b>CAUTION!</b> Risk of crushing hands!                                                                     |
|  | For indoor use only.                                                                                        |

**CAUTION!** The illustrations in this manual are for illustrative purposes and in some details may differ from the actual product.

The original manual is the German version. The other language versions are translations from German.

## 2. Safety of use

**CAUTION!** Read all safety warnings and all manuals. Failure to follow the warnings and instructions may result in serious injury or death.

---

The term "device" or "product" in the warnings and descriptions of the instructions refers to PORTABLE MASSAGE TABLE.

## 2.1. Safety at the workplace

- a) Keep your working place tidy and well lit. Untidiness or poor lighting can lead to accidents. Be forward-looking, watch what you are doing and use common sense when using this device.
- b) If in doubt whether the product is working properly or if it is damaged contact the manufacturer's service.
- c) If in doubt whether the device is working properly, contact the manufacturer's service.
- d) Repairs to the device may only be carried out by the manufacturer's service. It is not allowed to make repairs yourself!
- e) Keep the operation manual for future reference. Should the device be passed on to third parties, the operation manual should also be handed over with it.
- f) Keep packaging elements and small assembly parts out of the reach of children.
- g) Keep the device away from children and animals.

**Remember!** Protect children and other bystanders while operating the device.

## 2.2. Personal safety

- a) It is forbidden to operate the device when you are tired, sick, under the influence of alcohol, drugs or medications that significantly limit the ability to operate the device.
- b) The device is not intended to be used by people (including children), who are mentally, sensory or intellectually impaired or without adequate experience and/or knowledge, unless they are supervised by a person responsible for their safety or have been given instructions on how to operate the device.
- c) The device may be operated by physically fit, capable and properly trained persons who have read this manual and have been instructed in occupational health and safety.
- d) Be careful and use common sense when operating the device. A moment of inattention while operating the device may result in serious personal injury.
- e) The device is not a toy. Children should be supervised to ensure that they do not play with the device.

---

## 2.3. Safe use of the device

- a) When not in use the devices should be stored out of the reach of children and people unfamiliar with the device or this instruction manual.
- b) Keep the device in good technical condition. Before each work check for general damage or damage related to moving parts (cracks in parts and components or any other conditions that may affect the safe operation of the device). If damaged, have the device repaired before use.
- c) The device should be kept out of the reach of children.
- d) Repair and maintenance of devices should be performed by qualified persons using only original spare parts. This will ensure safe use.
- e) To ensure the designed operational integrity of the device, do not remove factory-installed covers or loosen screws.
- f) When transporting and moving the device from the place of storage to the place of use, observe the health and safety regulations of manual transport work applicable in the country where the devices are used.
- g) It is forbidden to move, rearrange or rotate the device during operation.
- h) The device must be cleaned regularly in order to prevent the permanent accumulation of dirt.
- i) It is forbidden to interfere with the construction of the device in order to change its parameters or construction.
- j) Keep the devices away from sources of fire and heat.
- k) Do not overload the device.
- l) Make sure that the legs of the product are at the same level to avoid tipping over, which may cause serious personal injury.
- m) Keep sharp objects away from the product as they may damage the cover material.
- n) The product is not designed for medical/rehabilitation purposes.

**WARNING!** Although the device has been designed to be safe, with adequate means of protection and despite the use of additional user safety elements, there is still a slight risk of accident or injury while working with the device. It is recommended to exercise caution and common sense when using it.

## 3. Operating rules

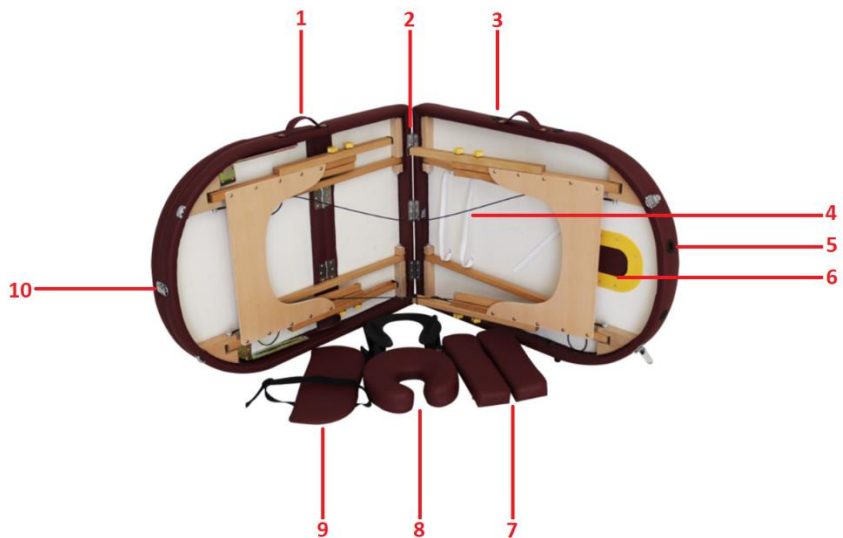
The product is an accessory for massage.

The product is intended for household use only!

**The user is responsible for any damage resulting from misuse.**

---

### 3.1. Device description

1. Handle
2. Hinge
3. Hole for the armrest
4. Rubber securing accessories
5. Hole for the headrest
6. Face opening
7. Side armrests
8. Headrest
9. Hanging armrest

---

## 10. Safety buckle

### 3.2. Preparation for operation

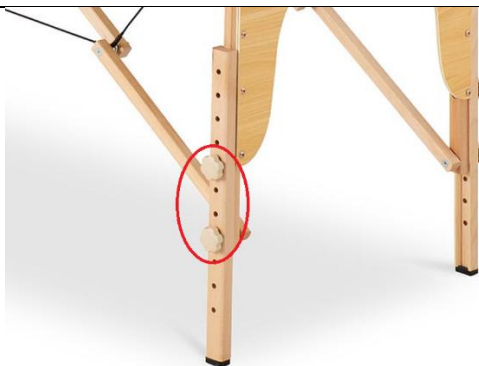
#### DEVICE PLACEMENT

Keep the device away from any hot surfaces. The device should always be used on a level, stable, clean, fireproof and dry surface and out of reach of children and people with reduced mental, sensory and mental functions.

#### DEVICE ASSEMBLY

1. Release the safety buckle
2. Put the table on its side and extend the legs to the desired height. To adjust the height of the table, unscrew the knobs, select the appropriate height by sliding the side part of the leg, simultaneously moving the knobs to the selected hole, then screw all knobs. Warning: Make sure the legs are at the same height.

3. Remove the secured accessories from inside the table.
4. Install the armrests by sliding them into the holes located on the sides of the backrest.
5. Attach the headrest on the front of the device by sliding it into the appropriate holes. Hang the front armrest on the headrest holder.
6. Determine the appropriate inclination angle for the leg rest. When lifting the backrest on both sides, the latches will indicate the appropriate height. To lower the backrest, raise the backrest to its maximum position, then lower the backrest to its initial position.

7. The table is correctly mounted.

## **DISMANTLING THE DEVICE**

1. Remove all accessories (headrest, armrest, front armrest).
2. Lay the table on its side.
3. Place the accessories under the elastic strap.
4. Fold the table legs, then fold the mattress and lock the buckle.

5. Place the folded table in the carry bag.

### 3.3. Cleaning and maintenance

- a) Use only non-corrosive products for surface cleaning.
- b) After each cleaning, all parts must be thoroughly dried before the device is used again.
- c) Store the device in a cool and dry place, protected from moisture and direct sunlight.
- d) Use a soft, damp cloth for cleaning.
- e) Do not use sharp and/or metal objects (e.g. wire brush or metal spatula) for cleaning as these may damage the surface of the device material.

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