USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
URUPE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

TREATMENT COSMETIC LAMP

DE	Produktname	Kosmetische Behandlungslampe
EN	Product name	Treatment cosmetic lamp
PL	Nazwa produktu	Lampa kosmetyczna zabiegowa
CZ	Název výrobku	Ošetřovací kosmetická lampa
FR	Nom du produit	Lampe cosmétique de traitement
IT	Nome del prodotto	Lampada cosmetica per trattamenti
ES	Nombre del producto	Lámpara cosmética de tratamiento
HU	Termék neve	Kezelési kozmetikai lámpa
DA	Produktnavn	Behandlingskosmetisk lampe
FI	Tuotteen nimi	Hoitolamppu
NL	Productnaam	Behandelingscosmeticalamp
NO	Produktnavn	Behandlingskosmetisk lampe
SE	Produktnamn	Behandlingskosmetisk lamp
PT	Nome do produto	Lâmpada cosmética de tratamento
SK	Názov produktu	Liečebná kozmetická lampa
BG	Име на продукта	Лампа за козметично лечение
EL	Όνομα προϊόντος	Λάμπα καλλυντικής θεραπείας
HR	Naziv proizvoda	Lampa za kozmetičke tretmane
LT	Produkto pavadinimas	Gydymo kosmetikos lempa
RO	Numele produsului	Lampă cosmetică de tratament
SL	Ime izdelka	Kozmetična svetilka za nego
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		PHY-TCS-02 PHY-TCS-03
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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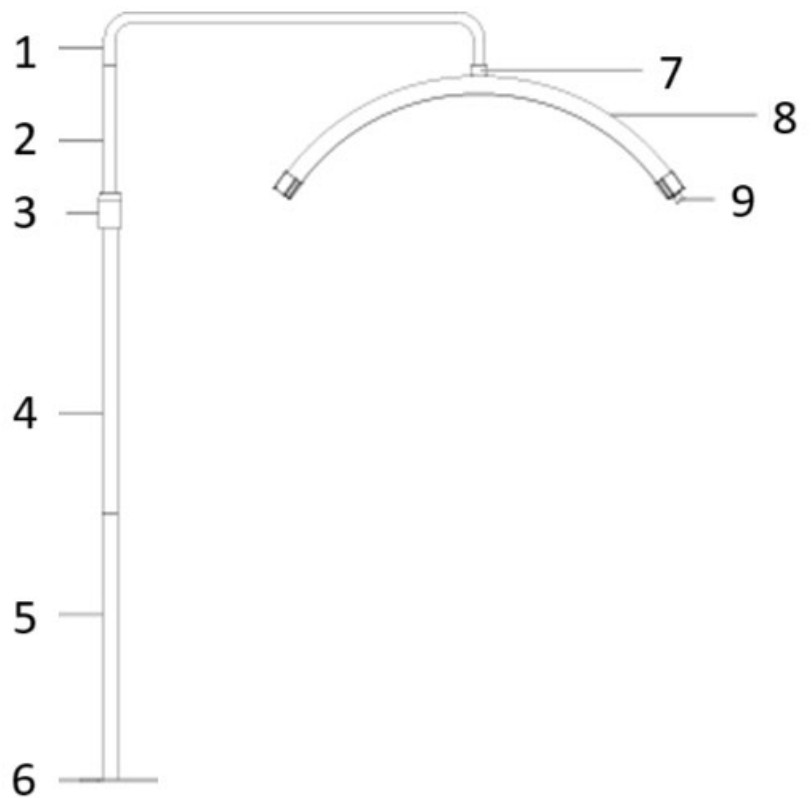
This User Manual has been translated for your convenience using machine translation. Reasonable efforts have been made to provide an accurate translation; however, no automated translation is perfect nor is it intended to replace human translators. The official User Manual is the English version. Any discrepancies or differences created in the translation are not binding and have no legal effect for compliance or enforcement purposes. If any questions arise related to the accuracy of the information contained in the User Manual, please refer to the English version of those contents which is the official version.

Technical data

Parameter description	Parameter value	
Product name	Treatment Cosmetic Lamp	
Model	PHY-TCS-02	PHY-TCS-03
Input voltage	230V	
Power	40 W	
Lumen	3600	
Number of LEDs	320	
Lifetime	50000 h	
Dimensions (LxWxH)	75 x 33 x 12 cm	
Arm length	55 cm	
Lamp diameter	680 mm	
Stand size	38 x 22 x 0.6 cm	


Safety Information

Please read the manual carefully and follow the installation instructions before use. Ensure the lighting is securely mounted on the bracket. Use the lighting only within the specified voltage range. Do not pull out the cable to turn off the power, and avoid touching the plug with wet hands. Keep the ring light in a dry and well-ventilated area. When not in use, disconnect the power to prevent short circuits.

1. Assembly Instructions


Attach Part A to the Lamp

- ① Pull the power cable through the right opening of Part A.
- ② Rotate the screw on Part A to secure it to the lamp.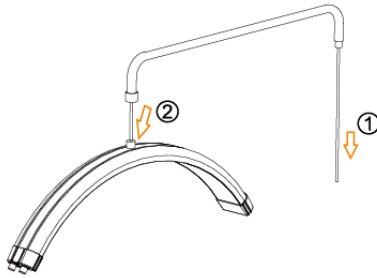


Connect the Base, Stand Part D, and Stand Part C

- ① Rotate Stand Part D to connect it to the base.
- ② Thread the DC cable through the hole at the bottom of the base and pull it out through Stand Part D.
- ③ Feed the DC cable into the hole of Stand Part C and pull it out from the top.
- ④ Rotate Stand Part C to connect it with Stand Part D.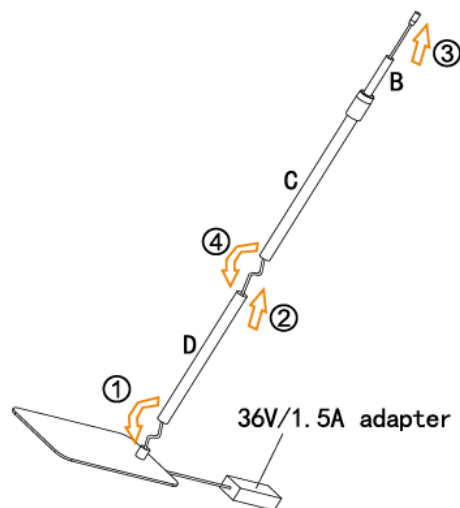


Connect the Power Cable and DC Cable

- ① Connect the power cable to the DC cable port and rotate it to secure the connection.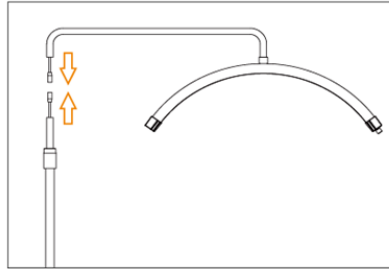

Connect the Stand and Lamp

- ① Loosen the screw and remove Stand Part B from Part C.
- ② Attach Stand Part B to Part A by rotating it, then tighten the screw to secure the connection.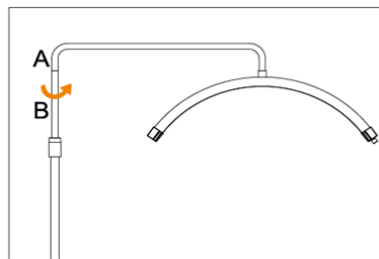

Note: For safe use, ensure the lamp and base are aligned in the same direction.

2. Using the Device


Adjustment Instructions

Power Adjustment:

Rotate the power switch and brightness adjustment knob to turn on the power. The brightness can be adjusted incrementally from 1% to 100%. Rotate the knob clockwise to increase brightness and counterclockwise to decrease it. Each step adjusts the brightness by 1%.

CCT Adjustment:

The colour temperature can be adjusted from 3000K to 5600K. Rotate the CCT dimmer knob clockwise to increase the colour temperature and counterclockwise to decrease it. Each step changes the colour temperature by 100K.

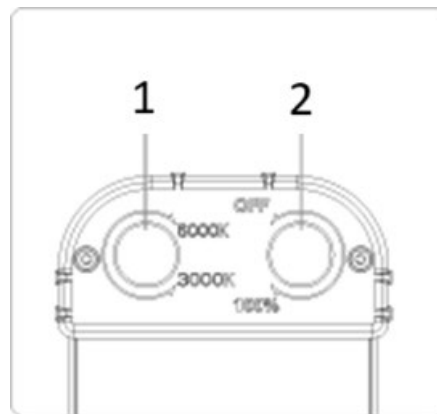
1.CCT dimmer

2.Power switch and brightness adjustment knob

3. Parts list

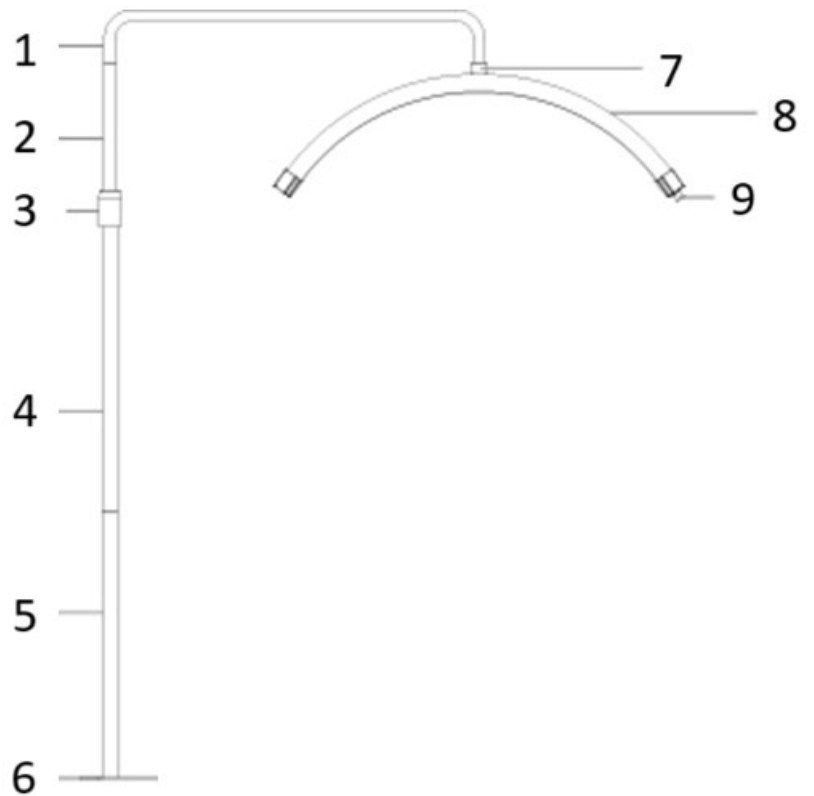No.	Description
1.	Stand Part A
2.	Stand Part B
3.	Tightening/Loosening Screw
4.	Stand Part C
5.	Stand Part D
6.	Base
7.	Lamp Locking Screw
8.	Lamp
9.	CCT switch / dimmer / power switch

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