# USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG  
INSTRUKCJA OBSŁUGI  
NÁVOD K POUŽITÍ  
MANUEL D'UTILISATION  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
HASZNÁLATI ÚTMUTATÓ  
BRUGSANVISNING  
KÄYTTÖOHJE  
GEBRUIKSAANWIJZING  
BRUKSANVISNING  
INSTRUÇÕES DE UTILIZAÇÃO  
POUŽÍVATEĽSKÁ PRÍRUČKA  
РЪКОВОДСТВО ЗА УПОТРЕБА  
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
UPUTE ZA UPORABU  
NAUDOJIMO INSTRUKCIJA  
MANUAL DE UTILIZARE  
NAVODILA ZA UPORABO

---

BEAUTY BED

<b>DE</b>	Produktname	Schönheitsbett
<b>EN</b>	Product name	Beauty bed
<b>PL</b>	Nazwa produktu	Łóżko kosmetyczne
<b>CZ</b>	Název výrobku	Kosmetická postel
<b>FR</b>	Nom du produit	Lit de beauté
<b>IT</b>	Nome del prodotto	Letino per la bellezza
<b>ES</b>	Nombre del producto	Cama de belleza
<b>HU</b>	Termék neve	Szépségágy
<b>DA</b>	Produktnavn	Skønhedsseng
<b>FI</b>	Tuotteen nimi	Kauneussänky
<b>NL</b>	Productnaam	Schoonheidsbed
<b>NO</b>	Produktnavn	Skjønnhets seng
<b>SE</b>	Produktnamn	Skönhetsäng
<b>PT</b>	Nome do produto	Cama de beleza
<b>SK</b>	Názov produktu	Kozmetická postel'
<b>BG</b>	Име на продукта	Легло за красота
<b>EL</b>	Όνομα προϊόντος	Κρεβάτι ομορφιάς
<b>HR</b>	Naziv proizvoda	Beauty krevet
<b>LT</b>	Produkto pavadinimas	Grožio lova
<b>RO</b>	Numele produsului	Pat de frumusețe
<b>SL</b>	Ime izdelka	Lepotna postelja
<b>DE</b> Modell   <b>EN</b> Product model   <b>PL</b> Model produktu   <b>CZ</b> Model výrobku   <b>FR</b> Modèle   <b>IT</b> Modello   <b>ES</b> Modelo   <b>HU</b> Modell   <b>DA</b> Model   <b>FI</b> Tuotteen malli   <b>NL</b> Productmodel   <b>NO</b> Produktmodell   <b>SE</b> Produktmodell   <b>PT</b> Modelo do produto   <b>SK</b> Model   <b>BG</b> Модел на продукт   <b>EL</b> Μοντέλο προϊόντος   <b>HR</b> Model proizvoda   <b>LT</b> : Gaminio modelis   <b>RO</b> : Model de produs   <b>SL</b> : Model izdelka		Physa Bitonto Physa Teramo Physa Sanremo Physa Viterbo
<b>DE</b> Hersteller   <b>EN</b> Manufacturer   <b>PL</b> Producent   <b>CZ</b> Výrobce   <b>FR</b> Fabricant   <b>IT</b> Produttore   <b>ES</b> Fabricante   <b>HU</b> Termelő   <b>DA</b> Producent   <b>FI</b> Valmistaja   <b>NL</b> Producent   <b>NO</b> Produsent   <b>SE</b> Tillverkare   <b>PT</b> Fabricante   <b>SK</b> Výrobca   <b>BG</b> Производител   <b>EL</b> Κατασκευαστής   <b>HR</b> Proizvođač   <b>LT</b> Gamintojas   <b>RO</b> Producător   <b>SL</b> Proizvajalec		expondo Polska sp. z o.o. sp. k.
<b>DE</b> Anschrift des Herstellers   <b>EN</b> Manufacturer Address   <b>PL</b> Adres producenta   <b>CZ</b> Adresa výrobce   <b>FR</b> Adresse du fabricant   <b>IT</b> Indirizzo del produttore   <b>ES</b> Dirección del fabricante   <b>HU</b> A gyártó címe   <b>DA</b> Producentens adresse   <b>FI</b> Valmistajan osoite   <b>NL</b> Adres producent   <b>NO</b> Produsentens adresse   <b>SE</b> Tillverkarens adress   <b>PT</b> Endereço do fabricante   <b>SK</b> Adresa výrobcu   <b>BG</b> Адрес на производителя   <b>EL</b> : Διεύθυνση κατασκευαστή   <b>HR</b> Adresa proizvođača   <b>LT</b> Gamintojo adresas   <b>RO</b> Adresa producătorului   <b>SL</b> Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra   Poland, EU

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This User Manual has been translated for your convenience using machine translation. Reasonable efforts have been made to provide an accurate translation; however, no automated translation is perfect nor is it intended to replace human translators. The official User Manual is the English version. Any discrepancies or differences created in the translation are not binding and have no legal effect for compliance or enforcement purposes. If any questions arise related to the accuracy of the information contained in the User Manual, please refer to the English version of those contents which is the official version.

## Technical data

Parameter description	Parameter value			
Product name	Beauty bed			
Model	Physa Bitonto	Physa Teramo	Physa Sanremo	Physa Viterbo
Input voltage [V~] / frequency [Hz]	AC 200-240 / 50/60			
Rated power [W]	300			
Load capacity [kg]	150			
Weight [kg]	79	94.5	88.75	84.7


## Use guidelines

The product is suitable for professional treatment and can be used for massage and a wide range of physiotherapy programs. The design of this product is to comfortably lie on the bed surface for massage or physiotherapy, with the whole bed including three beds and two pairs of armrests, which can adjust from the different positions of the bed body, and the adjusting hand can be attached under the bed seat. The product design is suitable for those over 12 years old and with 146CM tall.

**The user is liable for any damage resulting from unintended use of the device.**

## Device description

### 1. Physa Bitonto

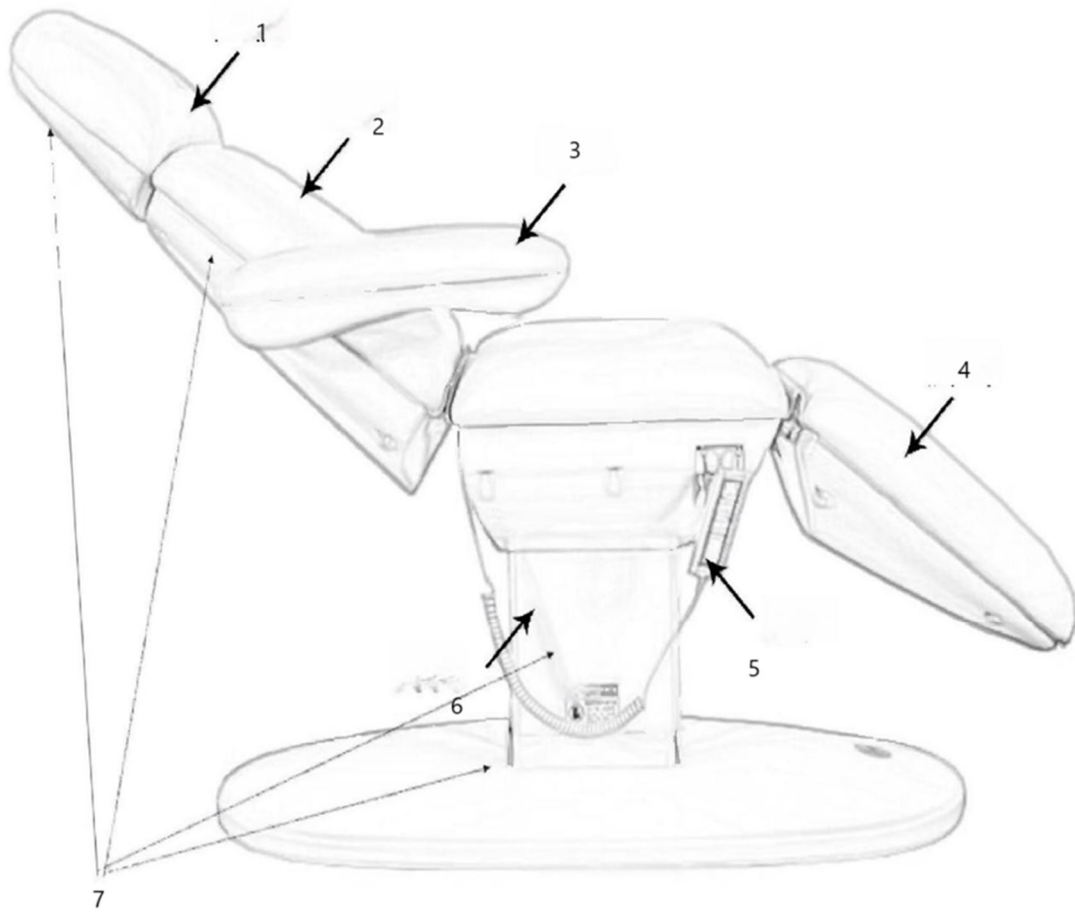

2. **Physa Teramo**

3. Physa Sanremo

4. Physa Viterbo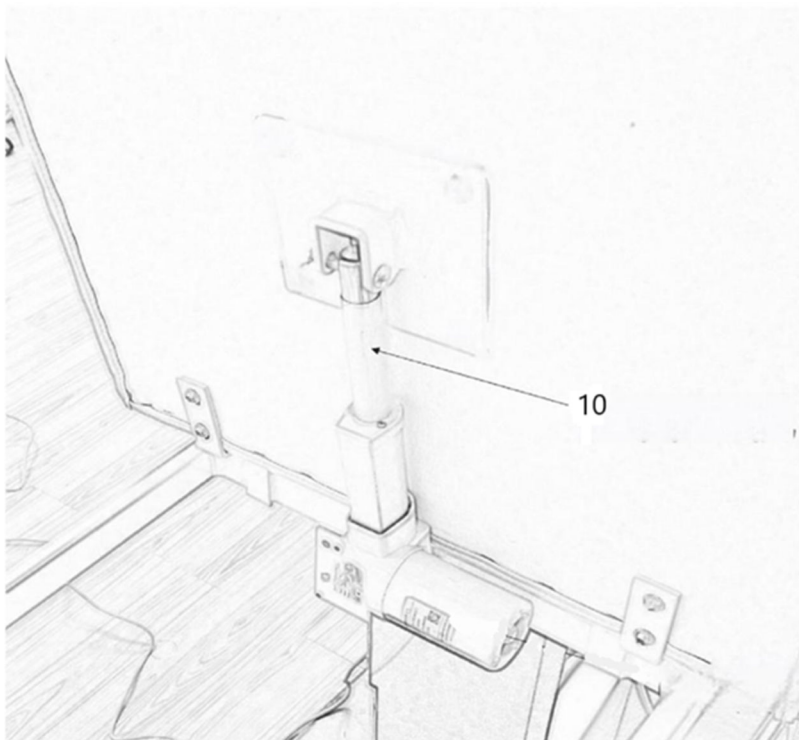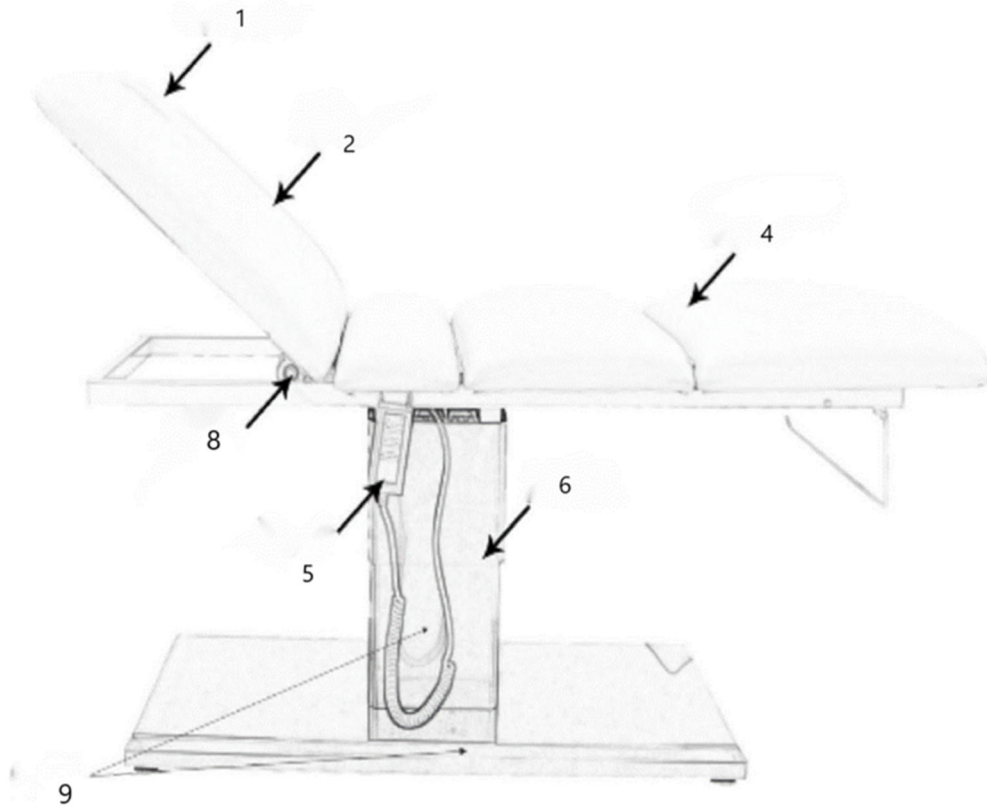


Part number	Description
1	Head pillow & breathing hole
2	Backrest
3	Armrest
4	Footrest

5	Remote control
6	Lifting and lowering
7	Cover
8	Electric motor
9	Stainless steel cover
10	Motor

## Load bearing

The maximum bearing of the back is 50 KG, and the maximum bearing of the cushion is 200 KG. Do not exceed the bed capacity bearing weight, do not directly jump on the back, and not damage the product structure.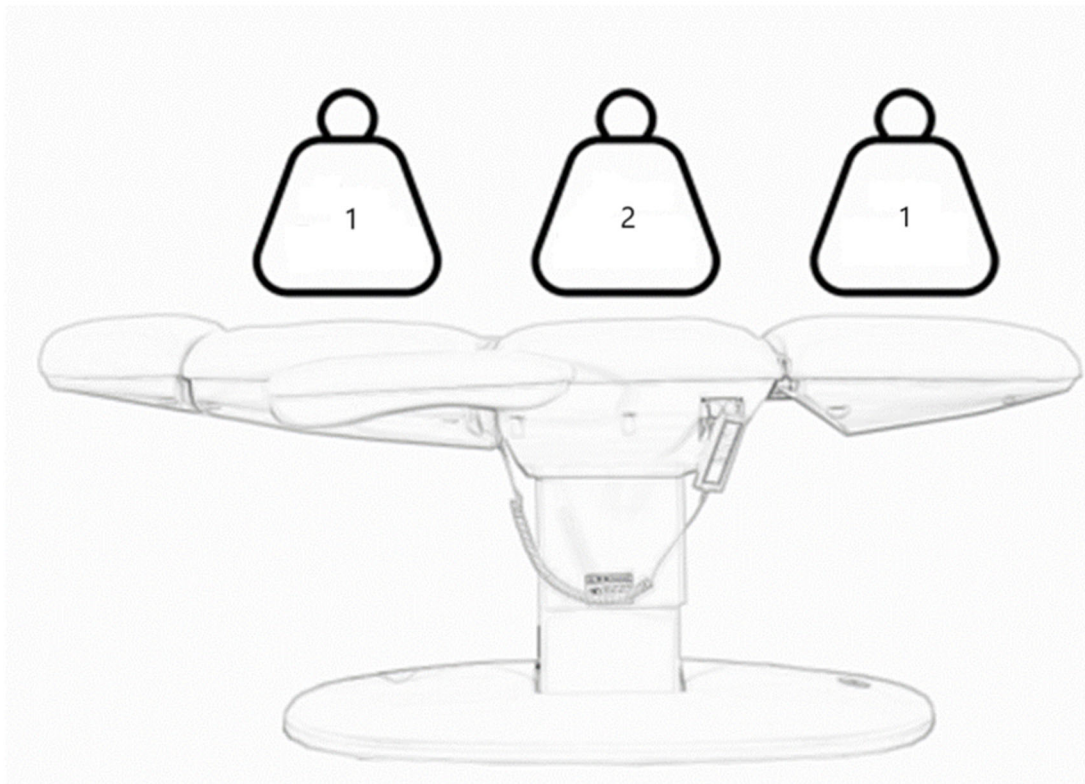


- 1- Max 50kg
- 2- Max 200kg

## Preparing for use

We pack the product by using a surrounding cardboard insert and an outer carton splint, providing a total of 10 layers of paper for excellent protection. We highly recommend keeping the original packaging for future use, especially in case of returns during the warranty period.

For safety reasons, two people are required to assist in lifting the beauty bed out of the packaging. Make sure to lift it from the head and foot ends of the bed.

Important! The beauty bed should only be lifted from the head and foot, never from the sides of the armrests.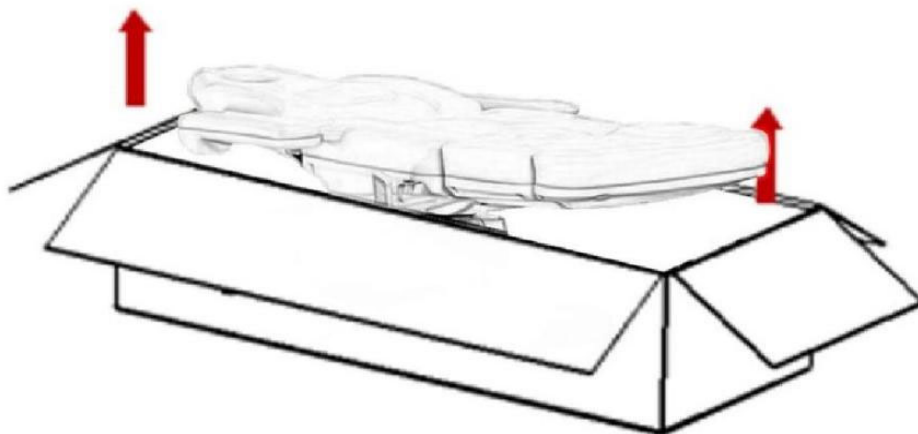

Before using the beauty bed, check whether the packaging is intact and if there was any damage during transportation. If any issues are found, please notify the transportation company or after-sales service within 2 days of receiving the product.

Ensure all parts are included upon delivery. If any accessories are missing, promptly contact the after-sales department.

The beauty bed must be used in a normal, dry indoor environment, with the room temperature maintained between +10°C and +40°C, and relative humidity between 30% and 75%.

If the beauty bed has been exposed to temperatures below 0°C, allow it to adjust to a suitable room temperature for at least 5 hours before use.

Before operating the beauty bed, familiarize yourself with the instructions and perform the following checks:



1. Ensure all packaging materials are removed.
2. Confirm that the beauty bed can move up and down freely.
3. Place the beauty bed in its intended location.
4. Plug the power cord into an outlet with a voltage that matches the motor's displayed voltage.

## Operation instructions


### 1. Physa Bitonto & Physa Teramo & Physa Sanremo

- **Hand control function embodiment**


(Back leaning angle 0~90°)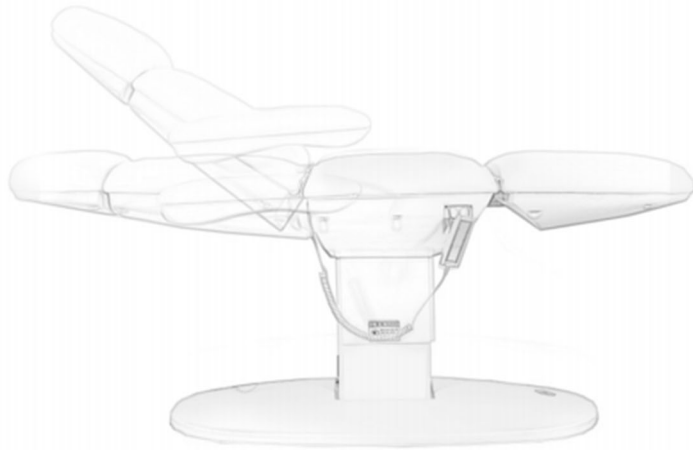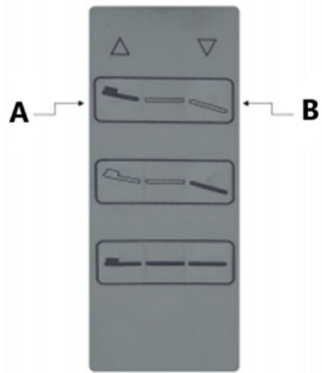

- **Leg adjustable range**  
(0~70cm)

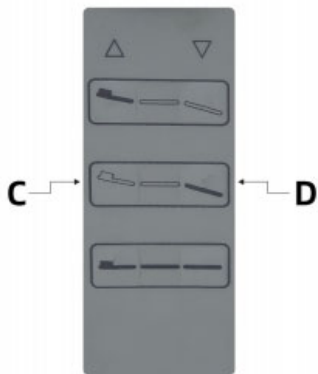- **Hand control function embodiment**  
(Lifting height 61~92cm)

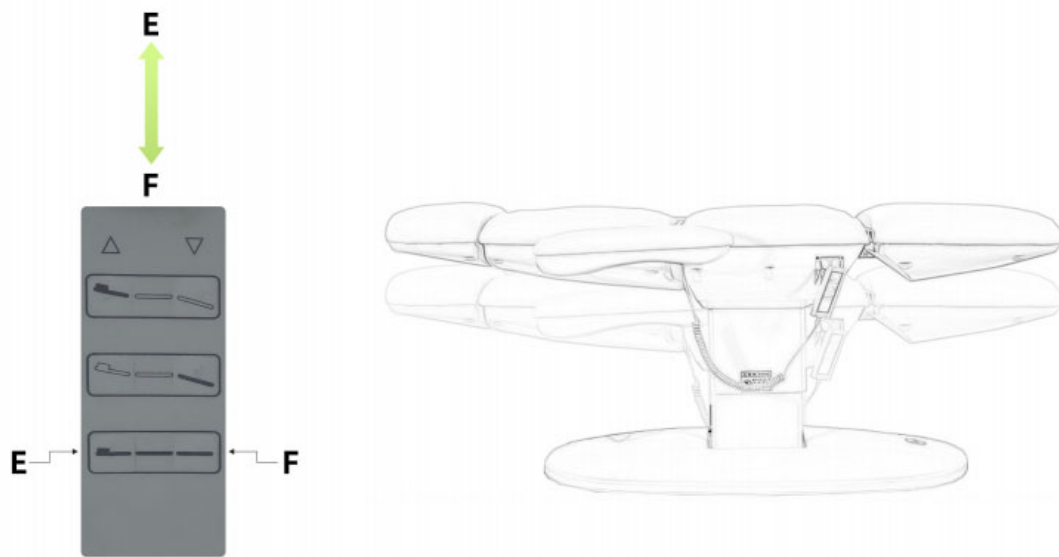
- **The armrest can be rotated**  
(0~180°)



## 2. **Physa Viterbo**


- **Hand control function embodiment**  
(Back leaning angle 0~90°)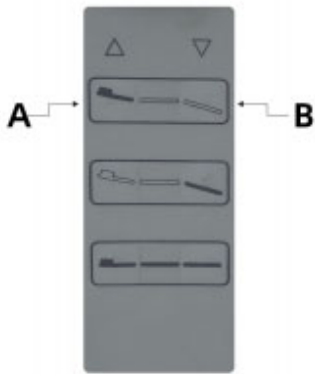

- **Leg adjustable range**  
(0~30cm)

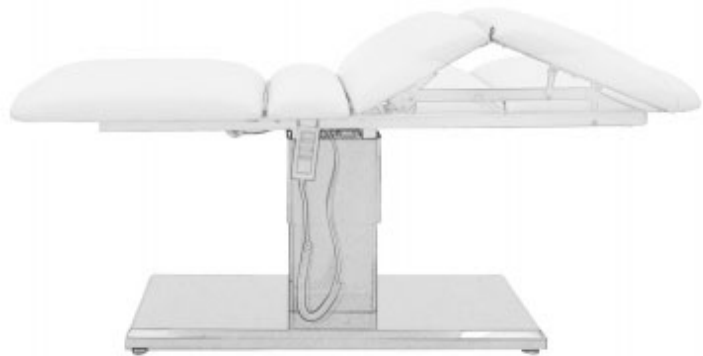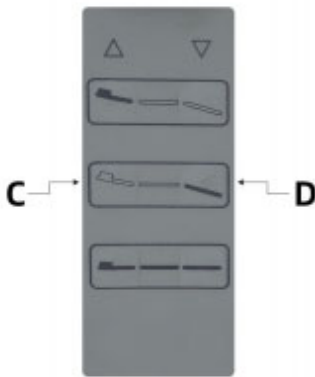- **Hand control function embodiment**  
(Lifting height 61~92cm)

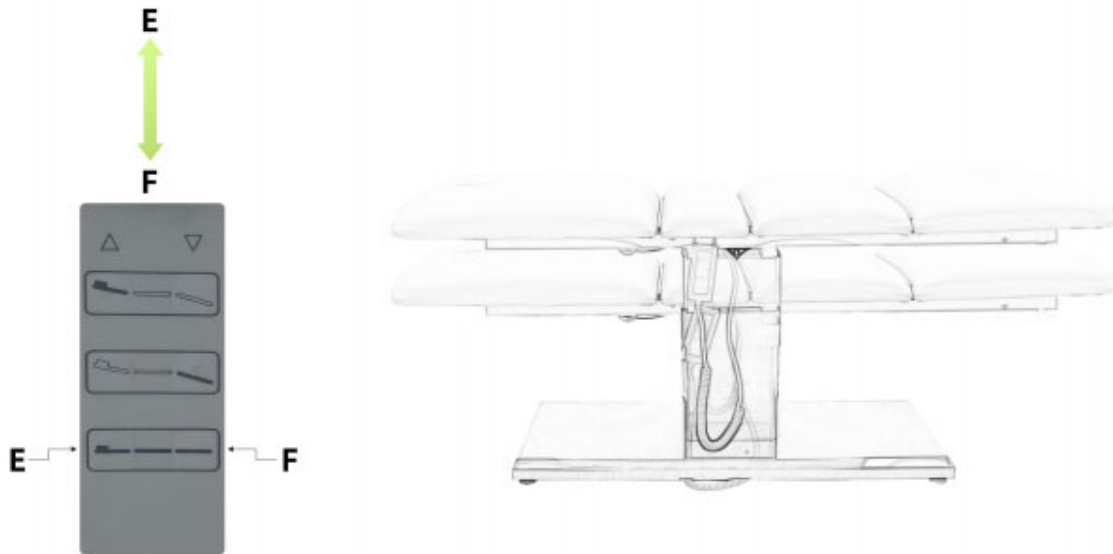## Cleaning and disinfection

**Attention!** Unplug the power cord before cleaning.

To keep the surface of the beauty bed in good condition, please clean it regularly. Ensure regular cleaning and disinfection before each use. A deep disinfection should be done once a month.

### 1. Metal and plastic surfaces

Wipe metal and plastic surfaces, including hand controllers, with a damp cloth and a mild alkaline cleaning solution. Use small brushes to clean corners and hard-to-reach areas. Rinse thoroughly and dry after cleaning, avoiding excess liquid. When disinfecting (with alcohol or chlorine), follow the instructions, allowing the surface to air dry at room temperature.

### 2. Stainless steel surface

These surfaces are highly chemical resistant and can be cleaned with a mild detergent solution. Stubborn stains can be treated with ammonia or suitable solvents. Avoid using sprays on chrome-plated metal surfaces.

All surfaces must be dry before connecting to the main power and equipment.

Do not use water spray (shower, pressure washer) for cleaning.

Avoid cleaning in high-temperature or high-humidity environments, and do not expose the equipment to an environment that may cause liquid pollution.

Do not use solvents or acids for cleaning.

Disinfection will wear off the surface. After disinfection, wipe the surface with a clean, damp cloth.

### 3. Bed surface

Use a soft cloth to wipe down the bed surface, then evenly apply a leather cleaner. Afterward, wipe the surface again with a clean, soft cloth. Avoid using wet cloths, hard objects, or chemicals such as acids or alkaline, as these may damage the product's surface and shorten its lifespan.

## Maintenance

**Attention!** Remove the power cord and verify that the function is off before maintenance.

1. Read the operation instructions carefully.

2. Experienced and manufacturer-authorized personnel are required to perform repairs. Maintenance performed by unauthorized personnel may cause injury or damage to equipment.
3. Only use the original equipment parts approved by us.
4. If the sofa is abnormal, do not use it or its accessories. Please unplug the power cord, use the safety switch, and contact the aftermarket.
5. All maintenance and repair operations must be documented.

Professional users are responsible for implementing the monthly and biannual measures.

Check controls and wiring, check status and functions weekly, according to the following checklist at least every six months:

1. Power cord and its fixation.
2. Motor wiring.
3. Control box and its wiring.
4. Whether the accessories are tight.
5. The normal operation of the lock system.
6. Complete all adjustments and make sure the sofa is working properly.

Please stop using the equipment and contact us if any defects are found, such as the equipment noise or not working. Only experienced or authorized personnel can open or replace the actuator/control unit.

If some parts of the equipment are damaged, remove the power cord, stop using the equipment, and then notify the aftermarket. Please ensure that any maintenance of all components was placed correctly after the action.

## Troubleshooting

Number	Problem	Solution
1	The hand control didn't respond.	Switch the position of the manual plug of the connection control box (Check for problems with the motor. Remember that the original position of the plug must be marked.)
2	The motor did not respond.	Switch the motor plug position.

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