USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

ELECTRIC BEAUTY BED

DE	Produktname	Kosmetikliege 3 Motoren golden + Rolling Stool Set
EN	Product name	Beauty bed 3 motors gold base + Rollhocker Set
PL	Nazwa produktu	Zestaw Fotel kosmetyczny 3 silniki złota podstawa + Krzesło kosmetyczne
CZ	Název výrobku	Kosmetická postel 3 motory zlatá základna + sada Rollhocker
FR	Nom du produit	Lit de beauté 3 moteurs base dorée + Rollhocker
IT	Nome del prodotto	Lettino estetica 3 motori base oro + set Rollhocker
ES	Nombre del producto	Camilla de belleza con 3 motores y base dorada + set Rollhocker
HU	Termék neve	Szépségágy 3 motoros arany alap + Rollhocker szett
DA	Produktnavn	Skønhedsseng 3 motorer guldbase + Rollhocker-sæt
FI	Tuotteen nimi	Kauneussänky 3 moottorin kultainen pohja + Rollhocker setti
NL	Productnaam	Schoonheidsbed 3 motoren gouden basis + Rollhocker set
NO	Produktnavn	Beauty seng 3 motorer gull base + Rollhocker sett
SE	Produktnamn	Skönhetssäng 3 motorer guldbas + Rollhocker set
PT	Nome do produto	Cama de beleza 3 motores base dourada + conjunto Rollhocker
SK	Názov produktu	Kozmetická posteľ 3 motory zlatá základňa + súprava Rollhocker
BG	Име на продукта	Легло за красота 3 мотора златна основа + Rollhocker комплект
EL	Όνομα προϊόντος	Κρεβάτι ομορφιάς 3 μωτέρ χρυσή βάση + σετ Rollhocker
HR	Naziv proizvoda	Kozmetički krevet sa 3 motora zlatna baza + Rollhocker set
LT	Produkto pavadinimas	Grožio lova 3 motorų auksinis pagrindas + Rollhocker rinkinys
RO	Numele produsului	Pat de frumusețe 3 motoare baza aurie + set Rollhocker
SL	Ime izdelka	Lepotna postelja s 3 motorji v zlati barvi + set Rollhocker
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		PHYSA SAVONA WHITE
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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This User Manual has been translated using machine translation. We have made every effort to ensure the translation is accurate, but please note that automated translations are not perfect and are not meant to replace human translators. The official version of the User Manual is in English. Any differences between the translated version and the original English are not legally binding. If you have any questions about the accuracy of the translation, please refer to the English version, which is the official reference.

Technical data

Parameter description	Parameter value
Product name	Beauty bed 3 motors gold base + Rollhocker set
Model	Physa savona white
Input voltage [V~] / frequency [Hz]	230 / 50
Rated power [W]	300 W
Protection class	I
Dimensions [width * depth * height; mm]	700*1900*630
Weight [kg]	86.5

Device description


The product is suitable for professional treatment and can be used for massage and a wide range of physiotherapy programs. The design of this product is to comfortably lie on the bed surface for massage or physiotherapy, with the whole bed including three beds and two pairs of armrests, which can adjust from the different positions of the bed body. The hand control, used for adjusting the bed, can be conveniently hung on hooks located under the seat. This product is intended for individuals over 12 years old and at least 146 cm in height.

The user is liable for any damage resulting from unintended use of the device.

Load bearing

The maximum bearing of the back is 50 KG, and the maximum bearing of the cushion is 200 KG. Do not exceed the bed capacity bearing weight, do not directly jump on the back, and do not damage the product structure.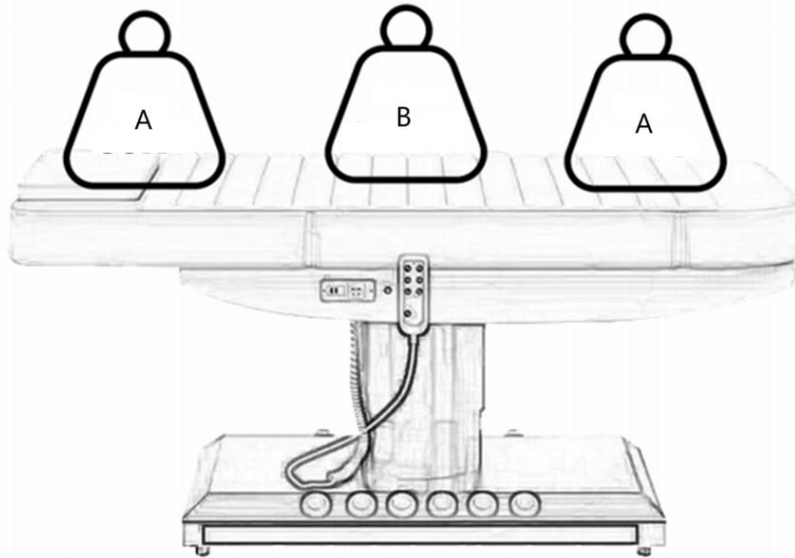


- A- Max 50kg
- B- Max 200kg

Preparing for use

We pack the product by using a surrounding cardboard insert and an outer carton splint, providing a total of 10 layers of paper for excellent protection. We highly recommend keeping the original packaging for future use, especially in case of returns during the warranty period.

For safety reasons, two people are required to assist in lifting the beauty bed out of the packaging. Make sure to lift it from the head and foot ends of the bed.

Important! The product should only be lifted from the head and foot, never from the sides of the armrests.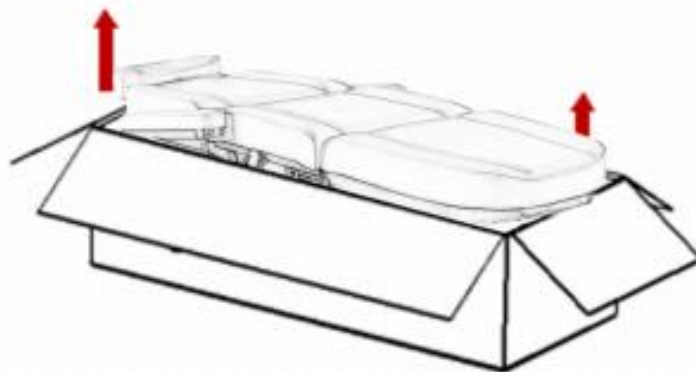

Before using the beauty bed, check whether the packaging is intact and if there was any damage during transportation. If any issues are found, please notify the transportation company or after-sales service within 2 days of receiving the product.

Ensure all parts are included upon delivery. If any accessories are missing, promptly contact the after-sales department.

The beauty bed must be used in a normal, dry indoor environment, with the room temperature maintained between +10°C and +40°C, and relative humidity between 30% and 75%.

If the beauty bed has been exposed to temperatures below 0°C, allow it to adjust to a suitable room temperature for at least 5 hours before use.


Before operating the beauty bed, familiarize yourself with the instructions and perform the following checks:

1. Ensure all packaging materials are removed.
2. Confirm that the beauty bed can move up and down freely.
3. Place the beauty bed in its intended location.
4. Plug the power cord into an outlet with a voltage that matches the motor's displayed voltage.

Operation instructions

1. Hand control function embodiment

(Angle of backrest tilt 0~62°)

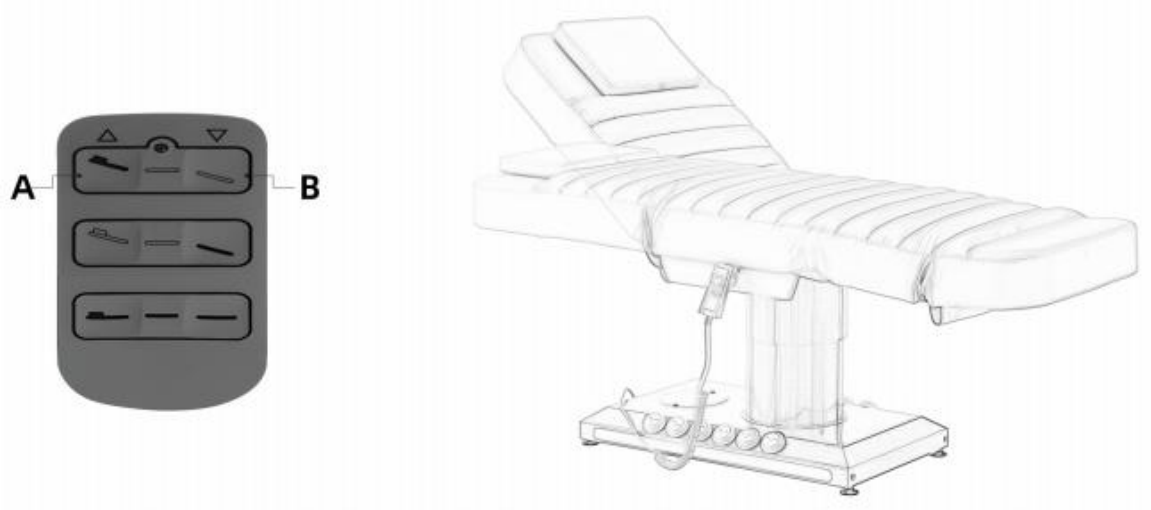2. Footrest and seat adjustable range

- 1- Footrest: 0~10°
- 2- Seat: 0-16°


3. Hand control function embodiment

(Lifting height 62~82cm)

4. LED ambient light & pedal button

- 1- LED ambient light
- 2- Pedal button
- 3- Button for turning on the light
- 4- Button for turning on the heating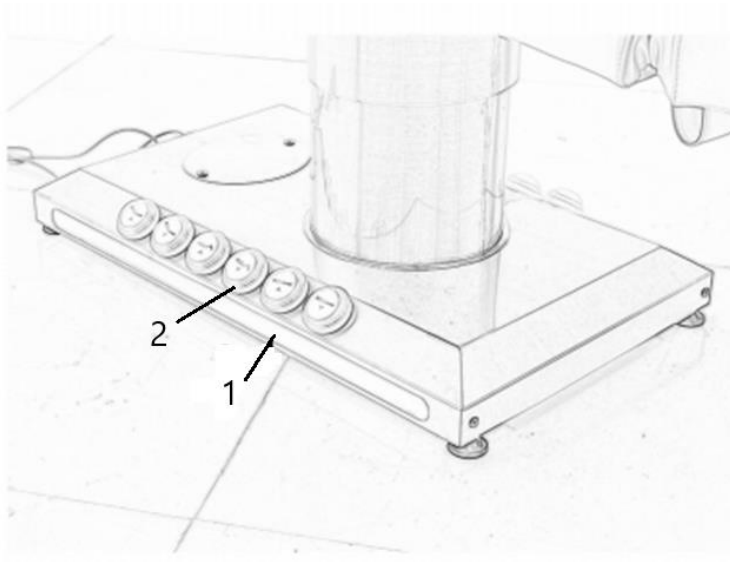

Cleaning and disinfection

Attention! Unplug the power cord before cleaning.

To keep the surface of the beauty bed in good condition, please clean it regularly. Ensure regular cleaning and disinfection before each use. A deep disinfection should be done once a month.

1. Metal and plastic surfaces

Wipe metal and plastic surfaces, including hand controllers, with a damp cloth and a mild alkaline cleaning solution. Use small brushes to clean corners and hard-to-reach areas. Rinse thoroughly and dry after cleaning, avoiding excess liquid. When disinfecting (with alcohol or chlorine), follow the instructions, allowing the surface to air dry at room temperature.

2. Stainless steel surface

These surfaces are highly chemical resistant and can be cleaned with a mild detergent solution. Stubborn stains can be treated with ammonia or suitable solvents. Avoid using sprays on chrome-plated metal surfaces.

All surfaces must be dry before connecting to the main power and equipment.

Do not use water spray (shower, pressure washer) for cleaning.

Avoid cleaning in high-temperature or high-humidity environments, and do not expose the equipment to an environment that may cause liquid pollution.

Do not use solvents or acids for cleaning.

Disinfection will wear off the surface. After disinfection, wipe the surface with a clean, damp cloth.

3. Bed surface

Use a soft cloth to wipe down the bed surface, then evenly apply a leather cleaner. Afterward, wipe the surface again with a clean, soft cloth. Avoid using wet cloths, hard objects, or chemicals such as acids or alkaline, as these may damage the product's surface and shorten its lifespan.

Maintenance

Attention! Remove the power cord and verify that the function is off before maintenance.

1. Read the operation instructions carefully.
2. Experienced and manufacturer-authorized personnel are required to perform repairs. Maintenance performed by unauthorized personnel may cause injury or damage to equipment.
3. Only use the original equipment parts approved by us.
4. If the sofa is abnormal, do not use it or its accessories. Please unplug the power cord, use the safety switch, and contact the aftermarket.
5. All maintenance and repair operations must be documented.

Professional users are responsible for implementing the monthly and biannual measures.

Check controls and wiring, check status and functions weekly, according to the following checklist at least every six months:

1. Power cord and its fixation.
2. Motor wiring.
3. Control box and its wiring.
4. Whether the accessories are tight.
5. The normal operation of the lock system.
6. Complete all adjustments and make sure the sofa is working properly.


Please stop using the equipment and contact us if any defects are found, such as the equipment noise or not working. Only experienced or authorized personnel can open or replace the actuator/control unit.

If some parts of the equipment are damaged, remove the power cord, stop using the equipment, and then notify the aftermarket. Please ensure that any maintenance of all components was placed correctly after the action.

Troubleshooting

Number	Problem	Solution
1	The hand control didn't respond.	Switch the position of the manual plug of the connection control box (Check for problems with the motor. Remember that the original position of the plug must be marked.)
2	The motor did not respond.	Switch the motor plug position.

Parts diagram

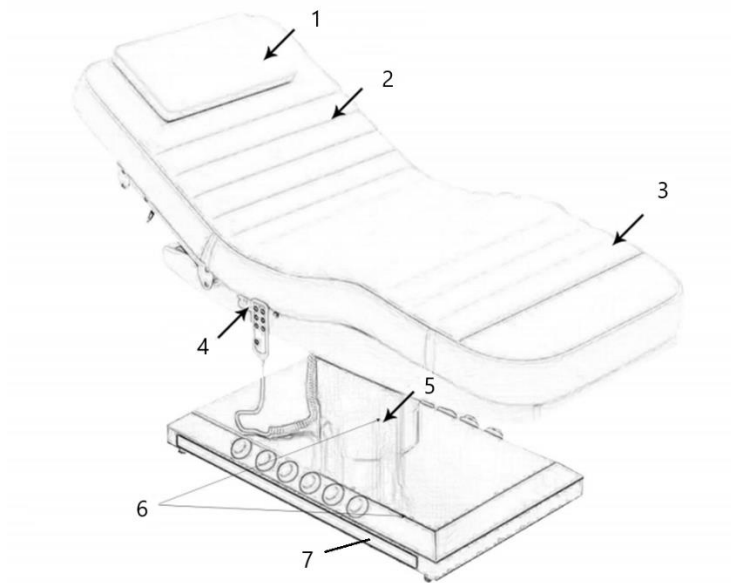Part number	Description
1	Headrest/Breathing hole
2	Backrest
3	Footrest
4	Remote control
5	Lifting and lowering
6	Stainless steel housing
7	LED ambient light
8	Rolling stool with backrest

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