USER MANUAL

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
ΡΥΚΟΒΟΔΙΣΤΕΒΟ ΖΑ ΥΠΟΤΡΕΒΑ
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

OXYGEN
CONCENTRATOR

DE	Produktname	Sauerstoffkonzentrator
EN	Product name	Oxygen concentrator
PL	Nazwa produktu	Koncentrator tlenu
CZ	Název výrobku	Kyslíkový koncentrátor
FR	Nom du produit	Concentrateur d'oxygène
IT	Nome del prodotto	Concentratore di ossigeno
ES	Nombre del producto	Concentrador de oxígeno
HU	Termék neve	Oxigénkoncentrátor
DA	Produktnavn	Iltkoncentrator
FI	Tuotteen nimi	Happikonsentraattori
NL	Productnaam	Zuurstofconcentrator
NO	Produktnavn	Oksygenkonsentrator
SE	Produktnamn	Syrgaskoncentrator
PT	Nome do produto	Concentrador de oxigénio
SK	Názov produktu	Kyslíkový koncentrátor
BG	Име на продукта	Кислороден концентратор
EL	Όνομα προϊόντος	Συμπυκνωτής οξυγόνου
HR	Naziv proizvoda	Koncentrator kisika
LT	Produkto pavadinimas	Deguonies koncentratorius
RO	Numele produsului	Concentrator de oxigen
SL	Ime izdelka	Koncentrator kisika
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		UNI_OC_100
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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This User Manual has been translated using machine translation. We have made every effort to ensure the translation is accurate, but please note that automated translations are not perfect and are not meant to replace human translators. The official version of the User Manual is in English. Any differences between the translated version and the original English are not legally binding. If you have any questions about the accuracy of the translation, please refer to the English version, which is the official reference. More language versions are available upon request via info@expondo.com.

1. Technical data

Table 1: Technical data of the product

Parameter description	Parameter value
Product name	Oxygen Concentrator
Model	UNI_OC_100
Rated voltage [V~] / frequency [Hz]	230/50
Rated power [W]	120
Protection class	II
Flow range [L/min]	1-6 (6-stage flow regulation)
Output pressure [kPa]	20-30
Dimensions [mm]	255x140x242
Noise level [dB(A)]	40
Weight [kg]	5,25

2. General description

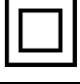
The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement. The device is designed to reduce noise emission risks to a minimum, taking into account technological progress and noise reduction opportunities.

2.1. Legend

Icon	Description
	The product satisfies the relevant safety standards.

	Read instructions before use.
	The product must be recycled.
	WARNING! or CAUTION! or REMEMBER! Applicable to the given situation. (general warning sign)
	ATTENTION! Electric shock warning!
	Class II protection device with double insulation.
	Do not smoke near the device.
	Only use indoors.

PLEASE NOTE! DRAWINGS IN THIS MANUAL ARE FOR ILLUSTRATION PURPOSES ONLY AND IN SOME DETAILS MAY DIFFER FROM THE ACTUAL PRODUCT.

3. Usage safety

ATTENTION! READ ALL SAFETY WARNINGS AND ALL INSTRUCTIONS. FAILURE TO FOLLOW THE WARNINGS AND INSTRUCTIONS MAY RESULT IN ELECTRIC SHOCK, FIRE AND/OR SERIOUS INJURY OR EVEN DEATH.

The terms "device" or "product" are used in the warnings and instructions to refer to: **Oxygen Concentrator**

3.1. Electrical safety

- a) The plug must fit the socket. Do not modify the plug in any way. Using original plugs and matching sockets reduces the risk of electric shock.
- b) Avoid touching earthed elements such as pipes, heaters, boilers and refrigerators. There is an increased risk of electric shock if the earthed device is exposed to rain, comes into direct contact with a wet surface or is operating in a damp environment. Water getting into the device increases the risk of damage to the device and of electric shock.
- c) Do not touch the device with wet or damp hands.

- d) Use the cable only for its designated use. Never use it to carry the device or to pull the plug out of a socket. Keep the cable away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cables increase the risk of electric shock.
- e) Do not use the device if the power cord is damaged or shows obvious signs of wear. A damaged power cord should be replaced by a qualified electrician or the manufacturer's service centre.
- f) f) To avoid electric shock, do not immerse the cord, plug or device in water or other liquids. Do not use the device on wet surfaces.
- g) **ATTENTION! DANGER TO LIFE!** While cleaning, never immerse the device in water or other liquids.
- h) Do not use in very humid environments or in the direct vicinity of water tanks.
- i) Prevent the device from getting wet. Risk of electric shock!

3.2. Safety in the workplace

- a) Do not use the device in a potentially explosive environment, for example in the presence of flammable liquids, gases or dust.
- b) If you discover damage or irregular operation, immediately switch the device off and report it to a supervisor without delay.
- c) Only the manufacturer's service point may repair the device. Do not attempt any repairs independently!
- d) In case of fire, use a powder or carbon dioxide (CO₂) fire extinguisher (one intended for use on live electrical devices) to put it out.
- e) Use the device in a well-ventilated space.
- f) Please keep this manual available for future reference. If this device is passed on to a third party, the manual must be passed on with it.
- g) Keep packaging elements and small assembly parts in a place not available to children.
- h) Keep the device away from children and animals.

REMEMBER! WHEN USING THE DEVICE, PROTECT CHILDREN AND OTHER BYSTANDERS.

3.3. Personal safety

- a) Do not use the device when tired, ill or under the influence of alcohol, narcotics or medication which can significantly impair the ability to operate the device.
- b) The device is not designed to be handled by persons (including children) with limited mental and sensory functions or persons lacking relevant experience and/or knowledge unless they are supervised by a person responsible for their safety or they have received instruction on how to operate the device.

- c) To prevent the device from accidentally switching on, make sure the switch is on the OFF position before connecting to a power source.
- d) The device is not a toy. Children must be supervised to ensure that they do not play with the device.

3.4. Safe device use

- a) Do not use the device if the ON/OFF switch does not function properly (does not switch the device on and off). Devices which cannot be switched on and off using the ON/OFF switch are hazardous, should not be operated and must be repaired.
- b) Disconnect the device from the power supply before commencement of adjustment, cleaning and maintenance. Such a preventive measure reduces the risk of accidental activation.
- c) When not in use, store in a safe place, away from children and people not familiar with the device who have not read the user manual. The device may pose a hazard in the hands of inexperienced users.
- d) Keep the device in perfect technical condition. Before each use check for general damage and especially check for cracked parts or elements and for any other conditions which may impact the safe operation of the device. If damage is discovered, hand over the device for repair before use.
- e) Keep the device out of the reach of children.
- f) Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- g) To ensure the operational integrity of the device, do not remove factory-fitted guards and do not loosen any screws.
- h) Avoid situations where the device stops working during use due to excessive loading. This may result in overheating of the drive elements and damage to the device.
- i) Do not move, adjust or rotate the device in the course of work.
- j) Do not leave this appliance unattended while it is in use.
- k) Clean the device regularly to prevent stubborn grime from accumulating.
- l) Do not cover the air intake and outlet.
- m) The device is not a toy. Cleaning and maintenance may not be carried out by children without supervision by an adult person.
- n) It is forbidden to interfere with the structure of the device in order to change its parameters or construction.
- o) Keep the device away from sources of fire and heat.
- p) Do not overload the device.
- q) Do not cover the ventilation openings!

- r) Smoking, open flame, using tools that generate flame or sparks, working with oils or grease, or leaving objects contaminated with oil or grease, is prohibited in the room treated with the device.
- s) The device should never be used as life support equipment

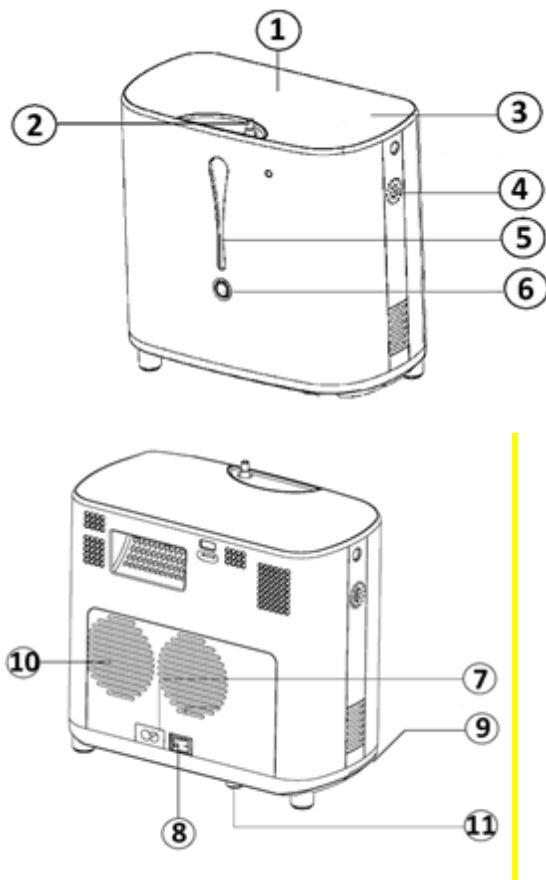
ATTENTION! DESPITE THE SAFE DESIGN OF THE DEVICE AND ITS PROTECTIVE FEATURES, AND DESPITE THE USE OF ADDITIONAL ELEMENTS PROTECTING THE OPERATOR, THERE IS STILL A SLIGHT RISK OF ACCIDENT OR INJURY WHEN USING THE DEVICE. STAY ALERT AND USE COMMON SENSE WHEN USING THE DEVICE.

4. Use guidelines

The device is designed to supply air with increased oxygen content. The product is outside the scope of MDD93/42/EEC regulations for medical devices. The product is intended for home use only!

The user is liable for any damage resulting from unintended use of the device.

4.1. Device description

1. Display

Keep the product away from hot surfaces. The device should always be used when positioned on an even, stable, clean, fireproof and dry surface, and be out of the reach of children and persons with limited mental and sensory functions. Position the device such that you always have access to the power plug. The power cord connected to the appliance must be properly grounded and correspond to the technical details on the product label. Do not leave the device in places exposed to direct sunlight. **WARNING!** If the transport or storage temperature is lower than 8 ° C, the device should be left in a room with a temperature between 10 ° C and 40 ° C for four hours before switching it on.

The device may generate a bit of sizzling noise during the operation - this is normal. We suggest to place the product in distance, where it would not cause any inconvenience.

4.3. Device use

3.3. DEVICE USE

Before the first use, check the device for any damage.

- a) Press the [6] button and pull out the water container [5]. Open the container filler by pulling out the silicone plug. Top up the tank with cold distilled water or cold clean water. **WARNING!** Do not exceed the maximum water level.
- b) Close the container with the cap and put it back in place. After inserting the container, you will hear a characteristic „click“.
- c) Connect the device to the power supply.
- d) Set the power switch [8] to the on position. The red lamp will light up. The device will be in standby mode.
- e) Press the „ON / OFF“ button [18], the display comes on.
- f) Select the appropriate parameters:
 1. Air flow level with increased oxygen content
 - To select the airflow level, use the buttons [16] and [17].

Level	Oxygen flow and concentration value
Level 6	6L, 28%
Level 5	5L, 30%
Level 4	4L, 32%
Level 3	3L, 40%
Level 2	2L, 60%
Level 1	1L, 90%

- Then connect the nasal cannula to the outlet [2].

2. Time setting

- Use the [14] and [15] buttons to set the automatic oxygen flow time. A maximum time of 3 hours can be set.
- Pressing the [15] button extends the operating time of the device by 15 minutes. Holding the [15] button increases the time continuously, ie. the time will increase until the button is released.
- Pressing the [14] button reduces the operating time of the device by 15 minutes. Holding down the [14] button reduces the time continuously, ie. the time will decrease until the button is released.
- After the set time has elapsed, the display will show „00:00“ and the oxygen flow will stop. To reset the time, press the [15] button.
- If the time value has not been set, the display will show „- H- -M“ means that automatic timing has not been activated. The device works in continuous mode until it is turned off.

g) Stopping /turning off the device

- To stop the device, press the „ON / OFF“ button [18].
- To turn the device off, hold down the „ON/ OFF“ button [18]. Disconnect the nasal cannula after use. On the back of the device, turn off the power using the switch [8], then remove the plug from the power source.

4.4. Cleaning and maintenance

4.4.1. General instructions

- a) Unplug the mains plug and allow the device to cool completely before each cleaning, adjustment or replacement of accessories, or if the device is not being used.
- b) Use only non-corrosive cleaners to clean the surface.
- c) After cleaning the device, all parts should be dried completely before using it again.
- d) Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.
- e) Do not spray the device with a water jet or submerge it in water.
- f) Do not allow water to get inside the device through vents in the housing of the device.
- g) Clean the vents with a brush and compressed air.
- h) The device must be regularly inspected to check its technical efficiency and spot any damage.
- i) Use a soft, damp cloth for cleaning.
- j) Do not use sharp and/or metal objects for cleaning (e.g. a wire brush or a metal spatula) because they may damage the surface material of the appliance.
- k) Clean the water container once a week.

- l) Do not leave water in the tank when the machine is not going to be used for a long time or when transporting the machine.

4.4.2. Filter

- a) Before removing the filter, pull out the mains plug and cool the device completely.
- b) Clean the filter with a mild detergent (for contact with food), then rinse thoroughly with clean water.
- c) Thoroughly dry the filter before placing it in the appliance.
- d) Clean the filter element every 15 days.
- e) Check the condition of the filter element (dirt can be seen through the transparent walls) and replace if necessary.

The location of the filter:

4.4.3. Disposing of used devices

Do not dispose of this device in municipal waste systems. Hand it over to an electric and electrical device recycling and collection point. Check the symbol on the product, instruction manual and packaging. The plastics used to construct the device can be recycled in accordance with their markings. By choosing to recycle you are making a significant contribution to the protection of our environment. Contact local authorities for information on your local recycling facility

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