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IR THERMOMETER

USER MANUAL

IR THERMOMETER

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TERMOMETRO INFRAROSSI

MANUAL DE INSTRUCCIONES

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SBS-IR-360-B

Deutsch	3
English	6
Polski	8
Česky	11
Français	13
Italiano	16
Español	19

PRODUKTNAME	IR THERMOMETER
PRODUCT NAME	IR THERMOMETER
NAZWA PRODUKTU	TERMOMETR NA PODCZERWIEŃ
NÁZEV VÝROBKU	INFRAČERVENÝ TEPLOMĚR
NOM DU PRODUIT	THERMOMÈTRE INFRAROUGE
NOME DEL PRODOTTO	TERMOMETRO INFRAROSSI
NOMBRE DEL PRODUCTO	TERMÓMETRO DE INFRARROJOS
MODELL	SBS-IR-360-B
PRODUCT MODEL	
MODEL PRODUKTU	
MODEL VÝROBKU	
MODÈLE	
MODELLO	
MODELO	
IMPORTEUR	EXPONDO POLSKA SP. Z O.O. SP. K.
IMPORTER	
IMPORTER	
DOVOZCE	
IMPORTATEUR	
IMPORTATORE	
IMPORTADOR	
ADRESSE VON IMPORTEUR	UL. NOWY KISIELIN-INNOWACYJNA 7, 66-002 ZIELONA GÓRA POLAND, EU
IMPORTER ADDRESS	
ADRES IMPORTERA	
ADRESA DOVOZCE	
ADRESSE DE L'IMPORTATEUR	
INDIRIZZO DELL'IMPORTATORE	
DIRECCIÓN DEL IMPORTADOR	

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Produktname	IR Thermometer	
Modell	SBS-IR-360-B	
Batterietyp [V]	9	
Temperaturmessbereich [°C]/[°F]	32°C~43°C (89-109°F)	
Messgenauigkeit [°C]/[°F]	32-34,9°C/ 89,6-94,8°F	±0,3°C (0,5°F)
	35-42°C/ 95-107,6°F	±0,2°C (0,4°F)
	42,1-43,0°C/ 107,7-109,4°F	±0,3°C (0,5°F)
Messauflösung [°C]	0,1	
Angabezeit [ms]	500	

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1. GENERAL DESCRIPTION

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement.

LEGEND

	The product satisfies the relevant safety standards.
	Read instructions before use.
	The product must be recycled.
	Only use indoors.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details may differ from the actual product.

The original operation manual is in German. Other language versions are translations from German.

2. USAGE SAFETY

ATTENTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in an electric shock, fire and/or serious injury or even death.

The terms "device" or "product" are used in the warnings and instructions to refer to < IR Thermometer > .

2.1. ELECTRICAL SAFETY

Do not touch the device with wet or damp hands.

2.2. SAFETY IN THE WORKPLACE

- If there are any doubts as to the correct operation of the device, contact the manufacturer's support service.
- Only the manufacturer's service point may repair the device. Do not attempt any repairs independently!
- Regularly inspect the condition of the safety labels. If the labels are illegible, they must be replaced.
- Please keep this manual available for future reference. If this device is passed on to a third party, the manual must be passed on with it.
- Keep packaging elements and small assembly parts in a place not available to children.
- Keep the device away from children and animals.

2.3. PERSONAL SAFETY

The device is not a toy. Children must be supervised to ensure that they do not play with the device

2.4. SAFE DEVICE USE

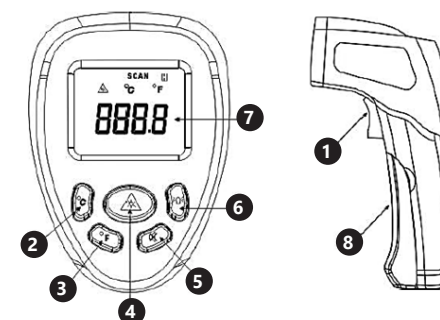
- When not in use, store in a safe place, away from children and people not familiar with the device who have not read the user manual. The device may pose a hazard in the hands of inexperienced users.
- Keep the device in perfect technical condition. Before each use check for general damage and especially check for cracked parts or elements and for any other conditions which may impact the safe operation of the device. If damage is discovered, hand over the device for repair before use.
- Keep the device out of the reach of children.
- Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- To ensure the operational integrity of the device, do not remove factory-fitted guards and do not loosen any screws.
- Clean the device regularly to prevent stubborn grime from accumulating.
- It is forbidden to interfere with the structure of the device in order to change its parameters or construction.
- Keep the device away from sources of fire and heat.
- It is recommended to remove the batteries from the device if it is not used for a long time.
- Do not expose the product to mechanical pressure.
- Keep unused batteries out of reach of children.
- When replacing the battery, ensure the polarity is correct.
- If the IR thermometer is subjected to a thermal shock during use (big or sudden changes in ambient temperature), wait for about 1 hour before the next measurement for the device to stabilize. Failure to comply with this recommendation may result in distorted measurement results.
- The device must be protected against electromagnetic radiation.

3. USE GUIDELINES

The device is intended for contactless temperature measurement and thermocouple temperature measurements. The product is intended for home use only! **The user is liable for any damage resulting from unintended use of the device.**

3.1. DEVICE DESCRIPTION

3.1.1. CONSTRUCTION OF THE DEVICE

- Trigger/ Measurement trigger
- Celsius scale button / down
- Fahrenheit scale button / up
- On/Off/ Indicator light
- Buzzer
- LCD backlight button
- LCD display
- Handle with battery compartment

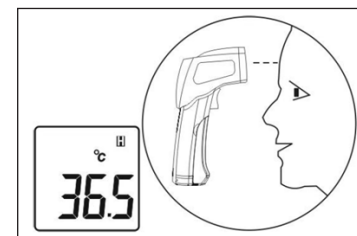
3.2. PREPARING FOR USE / DEVICE USE

3.2.1 INSTALLING THE BATTERIES

- Pull the battery cover away from the handle shaft.
 - Place the battery in the chamber, mind the correct polarity.
 - Close the battery cover.
- Note: If the device display shows low battery and it is necessary to replace the battery with a new one (the battery icon will appear in the upper left corner of the display), proceed in the same way as above.

3.2.2 CARRYING OUT MEASUREMENTS CORRECT PERFORMANCE OF MEASUREMENT:

- Turn on the thermometer [4].
- Place the thermometer in front of your forehead at a distance of 1-3 cm (do not touch the skin).
- Press the measurement button [1], the temperature will appear on the display [7].

- Turn off the device [4] or wait 5s - after this time the device will turn off automatically
NOTE: The display may show "LO" - which means that the measured temperature is lower than the thermometer scale or "HI" - which indicates a higher temperature than the thermometer scale. Usually, this message appears when the measurement has been made incorrectly or the temperature of an object other than that for which the device is intended has been measured.

TIPS FOR PERFORMING CORRECT MEASUREMENT:

- maintain an ambient temperature of 25°C +/- 30°C (the maximum ambient temperature for proper operation of the device is 40°C);
- measure on a clean, dry forehead not covered by hair;
- wait 30 minutes after bathing or exercising;
- after transport and change of the ambient temperature, wait with measurement for approx. 20-40 min. till the sensor adapts to the environment;
- if in doubt as to whether the result is correct, repeat the measurement or use another device to compare the results;
- keep in mind that human body temperature changes at different times of the day and is also affected by other conditions such as age, gender and skin colour.

CALIBRATION

When irregularities in the operation of the device are noticed, it is recommended to calibrate it. Make a test measurement and compare it with the measurement by another device already tested. If the measurements do not match:

- Remove the battery for 10 seconds.
- Hold the button [6] for 3 seconds while installing the battery (the button [6] being pressed all the time).
- After inserting the battery the screen will display 00.0. You can release the button [6].
- Adjust the difference using the button [2] - each press increases the value of the temperature reading by 0.1°C and button [3] - each press reduces the value of the temperature reading by 0.1°C.
- After calibration, remove and reinsert the batteries.

3.3. CLEANING AND MAINTENANCE

- Use only non-corrosive cleaners to clean the surface.
- Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.
- Do not spray the device with a water jet or submerge it in water.
- The device must be regularly inspected to check its technical efficiency and spot any damage.
- Do not leave the battery in the device if it will not be used for a longer period of time.
- Use a soft, damp cloth or sponge and a mild solution of water and soap to clean the case.
- It is recommended to use clean compressed air to remove fine dirt from the lens surface. Remove the remaining dirt with a soft and lint-free cloth.

SAFE REMOVAL OF BATTERIES AND RECHARGEABLE BATTERIES

9V (6F22) batteries are installed in the devices. Remove used batteries from the device using the same procedure by which you installed them. Recycle batteries with the appropriate organisation or company.

DISPOSING OF USED DEVICES

Do not dispose of this device in municipal waste systems. Hand it over to an electric and electrical device recycling and collection point. Check the symbol on the product, instruction manual and packaging. The plastics used to construct the device can be recycled in accordance with their markings. By choosing to recycle you are making a significant contribution to the protection of our environment. Contact local authorities for information on your local recycling facility.

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