USER MANUAL

Bedienungsanleitung
Instrukcja obsługi
Návod k použití
Manuel d'utilisation
Istruzioni per l'uso
Manual de instrucciones
Használati útmutató
Brugsanvisning
Käyttöohje
Gebruiksaanwijzing
Bruksanvisning
Instruções de utilização
Používateľská príručka
Ръководство за употреба
Οδηγίες χρήσης
Upute za uporabu
Naudojimo instrukcija
Manual de utilizare
Navodila za uporabo

BODY HEIGHT SCALE

DE	Produktname	Körpergrößenmessgerät
EN	Product name	Body height scale
PL	Nazwa produktu	Waga wzrostomierz
CZ	Název výrobku	Měřič výšky těla
FR	Nom du produit	Toise pour mesurer la taille
IT	Nome del prodotto	Altimetro per altezza corporea
ES	Nombre del producto	Escala de altura corporal
HU	Termék neve	Testmagasságmérő
DA	Produktnavn	Højdemåler
FI	Tuotteen nimi	Korkeusmittari
NL	Productnaam	Lichaamslengteschaal
NO	Produktnavn	Kroppshøydemåler
SE	Produktnamn	Kroppshöjdsåtare
PT	Nome do produto	Escala de altura corporal
SK	Názov produktu	Merač telesnej výšky
BG	Име на продукта	Уред за измерване на ръст
EL	Όνομα προϊόντος	Ζυγαριά μέτρησης ύψους σώματος
HR	Naziv proizvoda	Mjerač visine tijela
LT	Produktas pavadinimas	Kūno aukščio matuoklis
RO	Numele produsului	Scară de înălțime corporală
SL	Ime izdelka	Merilnik telesne višine
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		SBS-BS-101
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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This User Manual has been translated using machine translation. We have made every effort to ensure the translation is accurate, but please note that automated translations are not perfect and are not meant to replace human translators. The official version of the User Manual is in English. Any differences between the translated version and the original English are not legally binding. If you have any questions about the accuracy of the translation, please refer to the English version, which is the official reference. More language versions are available upon request via info@expondo.com.

Technical data

Parameter description	Parameter value
Product name	Body height scale
Model	SBS-BS-101
Battery	3.7V 3,600mAh
Capacity [kg]/Division [g]	180 / 50
Platform Size [mm]	300 x 400
Max. Height [cm]	200
Dimensions [width x depth x height; mm]	300 x 655 x 895
Weight [kg]	6.62

1. General description

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement. The device is designed to reduce noise emission risks to a minimum, taking into account technological progress and noise reduction opportunities.

Legend

The product satisfies the relevant safety standards.

Read instructions before use.

The product must be recycled.

WARNING! or **CAUTION!** or **REMEMBER!** Applicable to the given situation. (general warning sign)

Read the instructions.

ATTENTION! Electric shock warning!

Class II protection device with double insulation.

Only use indoors.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details may differ from the actual product.

2. Usage safety

Electrical equipment:

ATTENTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in electric shock, fire and/or serious injury or even death.

The terms "device" or "product" are used in the warnings and instructions to refer to:
Body height scale

2.1. Electrical safety

- a) The plug must fit the socket. Do not modify the plug in any way. Using original plugs and matching sockets reduces the risk of electric shock.
- b) Do not touch the device with wet or damp hands.
- c) Use the cable only for its designated use. Never use it to carry the device or to pull the plug out of a socket. Keep the cable away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cables increase the risk of electric shock.
- d) Do not use the device if the power cord is damaged or shows obvious signs of wear. A damaged power cord should be replaced by a qualified electrician or the manufacturer's service centre.
- e) To avoid electric shock, do not immerse the cord, plug or device in water or other liquids. Do not use the device on wet surfaces.
- f) **ATTENTION! DANGER TO LIFE!** While cleaning, never immerse the device in water or other liquids.
- g) Do not use in very humid environments or in the direct vicinity of water tanks.
- h) Prevent the device from getting wet. Risk of electric shock!

2.2. Safety in the workplace

- a) If you discover damage or irregular operation, immediately switch the device off and report it to a supervisor without delay.
- b) If you are unsure about whether the product is operating correctly or if you find damage, please contact the manufacturer's service centre.
- c) Only the manufacturer's service centre may make repairs to the product. Do not attempt to make repairs yourself!
- d) In case of fire, use a powder or carbon dioxide (CO₂) fire extinguisher (one intended for use on live electrical devices) to put it out.
- e) Please keep this manual available for future reference. If this device is passed on to a third party, the manual must be passed on with it.
- f) Keep packaging elements and small assembly parts in a place not available to children.
- g) Keep the device away from children and animals.

Remember! When using the device, protect children and other bystanders.

2.3. Safe device use

- a) Disconnect the device from the power supply before commencement of adjustment, cleaning and maintenance. Such a preventive measure reduces the risk of accidental activation.
- b) When not in use, store in a safe place, away from children and people not familiar with the device who have not read the user manual. The device may pose a hazard in the hands of inexperienced users.
- c) Keep the device out of the reach of children.

- d) Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- e) The device is not a toy. Cleaning and maintenance may not be carried out by children without supervision by an adult person.
- f) Keep the device away from sources of fire and heat.
- g) Do not overload the device.
- h) Exceeding the maximum permissible user weight can damage the product.
- i) Do not cover the ventilation openings!

ATTENTION! Despite the safe design of the device and its protective features, and despite the use of additional elements protecting the operator, there is still a slight risk of accident or injury when using the device. Stay alert and use common sense when using the device.

3. Use guidelines

This device is used to measure both body weight and height, providing accurate and consistent results. It combines these functions into one convenient solution, saving space and simplifying use.

This scale is not suitable for commercial transactions, legal purposes, medical uses, pharmaceutical applications, price determination in sales.

The user is liable for any damage resulting from unintended use of the device.

3.1. Device description

- | | | | |
|---|--------------------------|---|----------------------|
| 1 | Screen | 3 | On/Off Tare button |
| 2 | "CAL" Calibration button | 4 | Unit change button |

3.2. Preparing for use

APPLIANCE LOCATION

The temperature of environment must not be higher than 40°C and the relative humidity should be less than 85%. Ensure good ventilation in the room in which the device is being used. There should be at least 10 cm distance between each side of the device and the wall or other objects. The device should always be used when positioned on an even, stable, clean, fireproof and dry surface, and be out of the reach of children and persons with limited mental and sensory functions. Position the device such that you always have access to the power plug. The power cord connected to the appliance must be properly grounded and correspond to the technical details on the product label.

3.3. Device use

Button Descriptions and Functions:

1. **"CAL"** (Calibration):
 - This button is used to calibrate the scale. Calibration ensures accurate measurements by resetting the scale to its default settings.
2. **"Unit"**:
 - This button switches the measurement units displayed on the scale.
 - Common options include kilograms (kg), pounds (lb).
 - Press this button to toggle through the available unit options.
3. **"ON/OFF TARE"**:
 - **ON/OFF**:
 - Turns the scale on or off. Press the button once to turn the device on. Press and hold it to turn it off.
 - **"TARE"**:
 - Resets the scale to zero when an object is already on the platform. This allows you to measure the weight of additional items without including the initial weight.
 - Example: Place a container on the scale, press the TARE button to reset the display to zero, and then add items to the container to measure their weight directly.

Calibration Instructions

Standard Calibration with Buttons

The calibration process can be performed using a **single-point calibration** method with either a 50 kg or 25 kg calibration weight.

1. **Enter Calibration Mode:**
 - Ensure the scale is powered on.
 - Press and hold the **"CAL"** button until the display enters calibration mode.
2. **Calibrate with 50 kg:**
 - When the display shows **CAL** → internal code → **50.00KG**, place a 50 kg calibration weight on the scale.
 - Wait until the display shows **PASS**, indicating that the calibration is complete.
3. **Calibrate with 25 kg:**
 - After entering calibration mode (as above), the display will initially show **50.00KG**.
 - Press the **"CAL"** button again to switch to 25 kg calibration mode. The display will change to **25.00KG**.
 - Place a 25 kg calibration weight on the scale.
 - Wait until the display shows **PASS**, indicating that the calibration is complete.

Calibration Instructions

Standard Calibration with Buttons

The calibration process can be performed using a **single-point calibration** method with either a 50 kg or 25 kg calibration weight.

1. **Enter Calibration Mode:**
 - Ensure the scale is powered on.
 - Press and hold the "CAL" button until the display enters calibration mode.
2. **Calibrate with 50 kg:**
 - When the display shows **CAL** → internal code → **50.00KG**, place a 50 kg calibration weight on the scale.
 - Wait until the display shows **PASS**, indicating that the calibration is complete.
3. **Calibrate with 25 kg:**
 - After entering calibration mode (as above), the display will initially show **50.00KG**.
 - Press the "CAL" button again to switch to 25 kg calibration mode. The display will change to **25.00KG**.
 - Place a 25 kg calibration weight on the scale.
 - Wait until the display shows **PASS**, indicating that the calibration is complete.

Calibration Without Buttons or Power Switch

If the buttons or power switch are not operational, follow these steps to calibrate the scale:

1. **Prepare the Scale:**
 - Remove the battery from the device.
 - Short-circuit the battery terminals (connect them briefly with a metallic conductor).
 - Reinsert the battery to power on the device.
2. **Enter Calibration Mode:**
 - Once powered on, the display will show **0.00**.
 - Remove the short-circuit connector **immediately** after **0.00** appears (you have 2 seconds to perform this step before the display changes).
3. **Perform Calibration:**
 - The display will show **50.00**, indicating the required calibration weight.
 - Place a **single 50 kg calibration weight** (do not use multiple smaller weights).
 - Wait until the display shows **PASS**, confirming that the calibration is successful.

Notes:

- Always use accurate calibration weights for reliable results.
- Ensure the scale is placed on a stable and level surface during calibration.
- For best performance, avoid using makeshift weights or multiple smaller weights for calibration.

3.4. Cleaning and maintenance

- a) Always unplug the device before cleaning or putting it away.
- b) Use only non-corrosive cleaners to clean the surface.
- c) After cleaning the device, all parts should be dried completely before using it again.
- d) Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.
- e) Do not spray the device with a water jet or submerge it in water.
- f) Do not allow water to get inside the device through vents in the housing of the device.
- g) Clean the vents with a brush and compressed air.
- h) The device must be regularly inspected to check its technical efficiency and spot any damage.
- i) Use a soft cloth for cleaning.
- j) Do not leave the battery in the device if it will not be used for a longer period of time.
- k) Use a soft, damp cloth for cleaning.

- l) Do not use sharp and/or metal objects for cleaning (e.g. a wire brush or a metal spatula) because they may damage the surface material of the appliance.
- m) Do not clean the device with an acidic substance, agents of medical purposes, thinners, fuel, oils or other chemical substances because it may damage the device.

SAFE REMOVAL OF BATTERIES AND RECHARGEABLE BATTERIES:

3.7V 3,600mAh batteries are installed in the devices.

Remove used batteries from the device using the same procedure by which you installed them.

Recycle batteries with the appropriate organisation or company.

DISPOSING OF USED DEVICES:

Do not dispose of this device in municipal waste systems. Hand it over to an electric and electrical device recycling and collection point. Check the symbol on the product, instruction manual and packaging. The plastics used to construct the device can be recycled in accordance with their markings. By choosing to recycle you are making a significant contribution to the protection of our environment.

Contact local authorities for information on your local recycling facility.

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