USER MANUAL

expondo.de

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES

PHYSA ROSARIO WHITE

PEDICURE CHAIR

DE	3
EN	4
PL	5
CZ	6
FR	7
IT	8
ES	9

PRODUKTNAME	FUSSPFLEGESTUHL
PRODUCT NAME	PEDICURE CHAIR
NAZWA PRODUKTU	FOTEL PODOLOGICZNY
NÁZEV VÝROBKU	PEDIKÉRSKÉ KŘESLO
NOM DU PRODUIT	FAUTEUIL DE PÉDICURE
NOME DEL PRODOTTO	SEDIA PER PEDICURE
NOMBRE DEL PRODUCTO	SILLA DE PEDICURA
MODELL	PHYSA ROSARIO WHITE
PRODUCT MODEL	
MODEL PRODUKTU	
MODEL VÝROBKU	
MODÈLE	
MODELLO	EXPONDO POLSKA SP. Z O.O. SP. K.
MODELO	
NAME DES HERSTELLERS	
MANUFACTURER NAME	
NAZWA PRODUCENTA	
NÁZEV VÝROBCE	UL. NOWY KISIELIN-INNOWACYJNA 7, 66-002 ZIELONA GÓRA POLAND, EU
NOM DU FABRICANT	
NOME DEL PRODUTTORE	
NOMBRE DEL FABRICANTE	
ANSCHRIFT DES HERSTELLERS	
MANUFACTURER ADDRESS	
ADRES PRODUCENTA	
ADRESA VÝROBCE	
ADRESSE DU FABRICANT	
INDIRIZZO DEL FORNITORE	
DIRECCIÓN DEL FABRICANTE	

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TECHNICAL DATA

Parameter description	Parameter value
Product name	Pedicure chair
Model	PHYSA ROSARIO WHITE
Maximum load [kg]	200
Length [cm]	183
Width [cm]	81
Adjustment range of the backrest angle [°]	0-60
Adjustment range of the footrest angle [°]	0-65
Adjustment range of the seating [cm]	60.5-78.5
Seat adjustment angle [°]	0-3
Seat thickness [cm]	10
Rotation angle [°]	0-360
Weight [kg]	59

Read instructions before use.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details it may differ from the actual product.

The original operation manual is in German. Other language versions are translations from German.

1. Lever for adjusting the footrest angle
2. Lever for adjusting the backrest angle
3. Seat height adjustment lever

Assembly:
Before use: assemble the product correctly and put it in the right place.

A		M6x8	8
C		M8x35	4
C		M10x30	4
D		M8	4

ATTENTION! This product's exploded view can be found on the last page of the operating instructions (pp. 10).

ATTENTION! Do not overload the device. Exceeding the maximum permissible user weight can damage the product.

Do not shake or hit the product because the seat may fall down and injure the user.

Do not place the product in direct sunlight as this may cause discoloration.

Backrest adjustment

- 1) Pull and hold the backrest adjustment lever.
- 2) Grip the backrest with the other hand and set the backrest at the desired angle.

NOTE: Hold the backrest handle by one hand in the course of pulling and holding the backrest adjustment lever. Otherwise, the backrest may fall uncontrollably and cause harm to the user.

Adjustment of the footrest.

- 1) Pull and hold the footrest adjustment lever.
- 2) Hold the footrest with your other hand and set it at the desired angle.

NOTE: Hold the footrest handle by one hand in the course of pulling and holding the footrest adjustment lever. Otherwise, the footrest may fall uncontrollably and cause harm to the user.

Seat height adjustment

Press the seat height adjustment lever in order to increase the seat height. Do not press the lever to the very end. In order to lower the seat height, it is necessary to press the lever to the very end and hold the lever. Release the lever when the bed is lowered to the desired height.

NOTE

Do not suddenly burden the device with a weight above 100kg. It can lead to damage of the device.
The device is not used to store objects on it.

Cleaning and maintenance

- Use cleaners without corrosive substances to clean each surface.
- **Note!** The product should be washed after each use. It is recommended to use a mild disinfectant for this purpose.
- Fabrics with a different colour than the upholstery may cause discoloration on the product.
- Use a soft cloth for cleaning.
- Store the unit in a dry, cool place, free from moisture and direct exposure to sunlight.

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