USER MANUAL

expondo.com

BEDIENUNGSANLEITUNG
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ
MANUEL D'UTILISATION
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
HASZNÁLATI ÚTMUTATÓ
BRUGSANVISNING
KÄYTTÖOHJE
GEBRUIKSAANWIJZING
BRUKSANVISNING
INSTRUÇÕES DE UTILIZAÇÃO
POUŽÍVATEĽSKÁ PRÍRUČKA
РЪКОВОДСТВО ЗА УПОТРЕБА
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
UPUTE ZA UPORABU
NAUDOJIMO INSTRUKCIJA
MANUAL DE UTILIZARE
NAVODILA ZA UPORABO

COSMETIC AND PODIATRY CHAIR

DE	Produktname	Kosmetik- und podologiestuhl
EN	Product name	Cosmetic and podiatry chair
PL	Nazwa produktu	Fotel kosmetyczno-podologiczny
CZ	Název výrobku	Kosmetické a podologické křeslo
FR	Nom du produit	Fauteuil d'esthétique et de podologie
IT	Nome del prodotto	Poltrona per estetica e podologia
ES	Nombre del producto	Sillón de estética y podología
HU	Termék neve	Kozmetikai és lábápoló szék
DA	Produktnavn	Stol til kosmetik og fodpleje
FI	Tuotteen nimi	Kosmeettinen ja jalkahoitotuoli
NL	Productnaam	Cosmetische en podotherapie stoel
NO	Produktnavn	Kosmetisk og fotpleiestol
SE	Produktnamn	Kosmetik- och fotvårdsstol
PT	Nome do produto	Cadeira de estética e podologia
SK	Názov produktu	Kozmetické a podologické kreslo
BG	Име на продукта	Стол за козметика и подология
EL	Όνομα προϊόντος	Καρέκλα αισθητικής και ποδολογίας
HR	Naziv proizvoda	Kozmetička i podijatrijska stolica
LT	Produkto pavadinimas	Kosmetikos ir podologijos kėdė
RO	Numele produsului	Scaun cosmetic si podiatrie
SL	Ime izdelka	Kozmetični in pediatrični stol
DE Modell EN Product model PL Model produktu CZ Model výrobku FR Modèle IT Modello ES Modelo HU Modell DA Model FI Tuotteen malli NL Productmodel NO Produktmodell SE Produktmodell PT Modelo do produto SK Model BG Модел на продукт EL Μοντέλο προϊόντος HR Model proizvoda LT : Gaminio modelis RO : Model de produs SL : Model izdelka		PHY-PC-01
DE Hersteller EN Manufacturer PL Producent CZ Výrobce FR Fabricant IT Produttore ES Fabricante HU Termelő DA Producent FI Valmistaja NL Producent NO Produsent SE Tillverkare PT Fabricante SK Výrobca BG Производител EL Κατασκευαστής HR Proizvođač LT Gamintojas RO Producător SL Proizvajalec		expondo Polska sp. z o.o. sp. k.
DE Anschrift des Herstellers EN Manufacturer Address PL Adres producenta CZ Adresa výrobce FR Adresse du fabricant IT Indirizzo del produttore ES Dirección del fabricante HU A gyártó címe DA Producentens adresse FI Valmistajan osoite NL Adres producent NO Produsentens adresse SE Tillverkarens adress PT Endereço do fabricante SK Adresa výrobcu BG Адрес на производителя EL : Διεύθυνση κατασκευαστή HR Adresa proizvođača LT Gamintojo adresas RO Adresa producătorului SL Naslov proizvajalca		ul. Nowy Kisielin – Innowacyjna 7, 66-002 Zielona Góra Poland, EU

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This User Manual has been translated using machine translation. We have made every effort to ensure the translation is accurate, but please note that automated translations are not perfect and are not meant to replace human translators. The official version of the User Manual is in English. Any differences between the translated version and the original English are not legally binding. If you have any questions about the accuracy of the translation, please refer to the English version, which is the official reference. More language versions are available upon request via info@expondo.com.

Technical data

Parameter description	Parameter value
Product name	Cosmetic and podiatry chair
Model	PHY-PC-01
Rated voltage [V~] / frequency [Hz]	230 / 50
Rated power [W]	15
Protection class	II
Dimensions [width * length * height; mm]	710*1200*1170
Weight [kg]	36.5

1. General description

The user manual is designed to assist in the safe and trouble-free use of the device. The product is designed and manufactured in accordance with strict technical guidelines, using state-of-the-art technologies and components. Additionally, it is produced in compliance with the most stringent quality standards.

DO NOT USE THE DEVICE UNLESS YOU HAVE THOROUGHLY READ AND UNDERSTOOD THIS USER MANUAL.

To increase the product life of the device and to ensure trouble-free operation, use it in accordance with this user manual and regularly perform maintenance tasks. The technical data and specifications in this user manual are up to date. The manufacturer reserves the right to make changes associated with quality improvement. The device is designed to reduce noise emission risks to a minimum, taking into account technological progress and noise reduction opportunities.

Legend

The product satisfies the relevant safety standards.

Read instructions before use.

The product must be recycled.

WARNING! or **CAUTION!** or **REMEMBER!** Applicable to the given situation.
(general warning sign)

ATTENTION! Electric shock warning!

ATTENTION! Hand crush hazard!

Only use indoors.

PLEASE NOTE! Drawings in this manual are for illustration purposes only and in some details may differ from the actual product.

2. Usage safety

ATTENTION! Read all safety warnings and all instructions. Failure to follow the warnings and instructions may result in electric shock, fire and/or serious injury or even death.

The terms "device" or "product" are used in the warnings and instructions to refer to:

Cosmetic and podiatry chair

2.1. Electrical safety

- a) The plug must fit the socket. Do not modify the plug in any way. Using original plugs and matching sockets reduces the risk of electric shock.
- b) Use the cable only for its designated use. Never use it to carry the device or to pull the plug out of a socket. Keep the cable away from heat sources, oil, sharp edges or moving parts. Damaged or tangled cables increase the risk of electric shock.
- c) Do not use the device if the power cord is damaged or shows obvious signs of wear. A damaged power cord should be replaced by a qualified electrician or the manufacturer's service centre.
- d) To avoid electric shock, do not immerse the cord, plug or device in water or other liquids. Do not use the device on wet surfaces.
- e) Do not use in very humid environments or in the direct vicinity of water tanks.
- f) Before the first use, please check whether the main voltage type and current comply with the indicated data on the type plate.

2.2. Safety in the workplace

- a) Make sure the workplace is clean and well lit. A messy or poorly lit workplace may lead to accidents. Try to think ahead, observe what is going on and use common sense when working with the device.
- b) If there are any doubts as to the correct operation of the device, contact the manufacturer's support service.
- c) Only the manufacturer's service centre may make repairs to the device. Do not attempt to make repairs yourself!
- d) In case of fire, use a powder or carbon dioxide (CO₂) fire extinguisher (one intended for use on live electrical devices) to put it out.
- e) Regularly inspect the condition of the safety labels. If the labels are illegible, they must be replaced.
- f) Please keep this manual available for future reference. If this device is passed on to a third party, the manual must be passed on with it.
- g) Keep packaging elements and small assembly parts in a place not available to children.

- h) Keep the device away from children and animals.
- i) If this device is used together with another equipment, the remaining instructions for use shall also be followed.

Remember! When using the device, protect children and other bystanders.

2.3. Personal safety

- a) Do not use the device when tired, ill or under the influence of alcohol, narcotics or medication which can significantly impair the ability to operate the device.
- b) The device is not designed to be handled by persons (including children) with limited mental and sensory functions or persons lacking relevant experience and/or knowledge unless they are supervised by a person responsible for their safety or they have received instruction on how to operate the device.
- c) The device can be handled only by physically fit persons who are capable of handling it, properly trained, familiar with this manual and trained within the scope of occupational health and safety.
- d) When working with the device, use common sense and stay alert. Temporary loss of concentration while using the device may lead to serious injuries.
- e) To prevent the device from accidentally switching on, make sure the switch is on the OFF position before connecting to a power source.
- f) Do not overestimate your abilities. When using the device, keep your balance and remain stable at all times. This will ensure better control over the device in unexpected situations.
- g) The device is not a toy. Children must be supervised to ensure that they do not play with the device.
- h) Do not put your hands or other items inside the device while it is in use!

2.4. Safe device use

- a) Do not overload the device. Use the appropriate tools for the given task. A correctly-selected device will perform the task for which it was designed better and in a safer manner.
- b) Keep the device in perfect technical condition. Before each use check for general damage, especially check moving components for cracked parts or elements, and for any other conditions which may impact the safe operation of the device. If damage is discovered, hand over the device for repair before use.
- c) Device repair or maintenance should be carried out by qualified persons, only using original spare parts. This will ensure safe use.
- d) To ensure the operational integrity of the device, do not remove factory-fitted guards and do not loosen any screws.
- e) When transporting and handling the device between the warehouse and the destination, observe the occupational health and safety principles for manual transport operations which apply in the country where the device will be used.
- f) Avoid situations where the device stops working during use due to excessive loading. This may result in overheating of the drive elements and damage to the device.
- g) Do not touch articulated parts or accessories unless the device has been disconnected from the power source.
- h) Do not move, adjust or rotate the device in the course of work.
- i) Clean the device regularly to prevent stubborn grime from accumulating.

- j) It is forbidden to interfere with the structure of the device in order to change its parameters or construction.
- k) Keep the device away from sources of fire and heat.
- l) Do not overload the device.
- m) Exceeding the maximum permissible user weight can damage the product.
- n) Keep handles dry, clean and free of oil or grease.

ATTENTION! Despite the safe design of the device and its protective features, and despite the use of additional elements protecting the operator, there is still a slight risk of accident or injury when using the device. Stay alert and use common sense when using the device.


3. Use guidelines

The product is designed based on the distribution of foot acupuncture points and bone structure, ensuring effective massage for all foot sizes.

It is designed to stimulate acupuncture points on the feet, promoting gradual relief from discomfort, reducing fatigue, and alleviating symptoms of neurasthenia. This enhances overall body relaxation and comfort, helping to prevent and manage various types of pain and chronic conditions.

The user is liable for any damage resulting from unintended use of the device.

3.1. Device description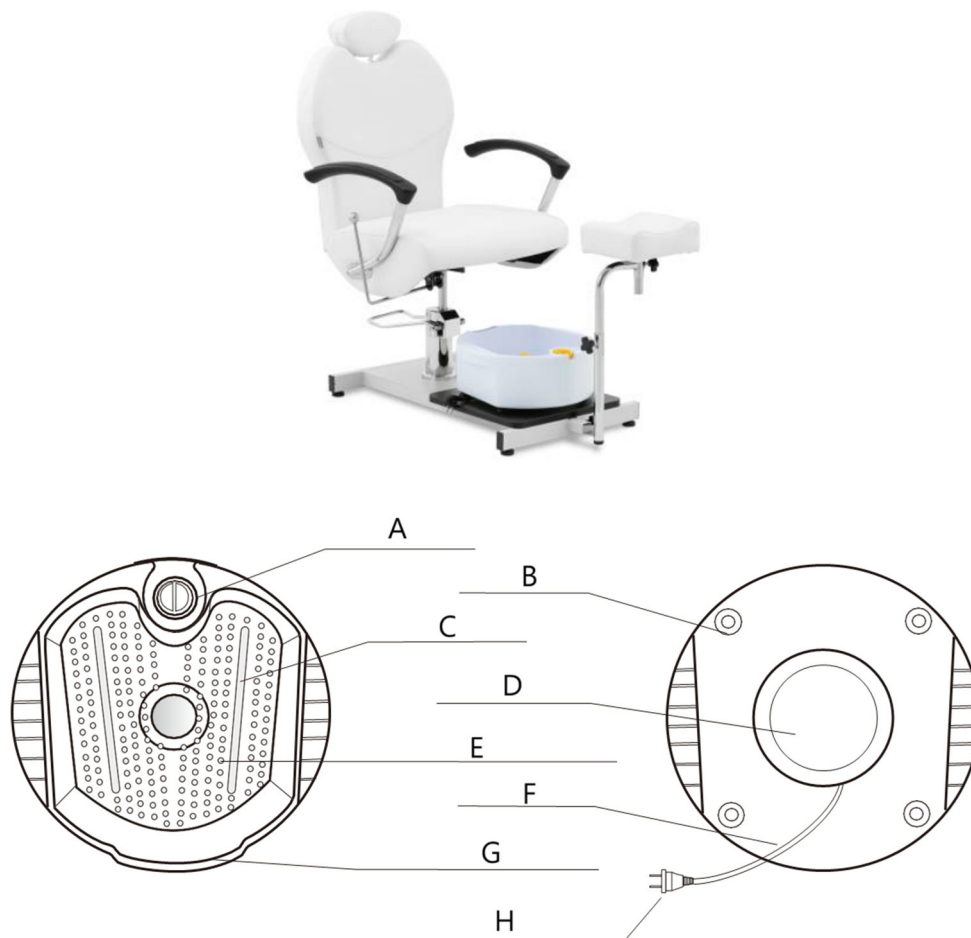

Part number	Description
A	Switch knob
B	Rubber feet
C	Spray pore
D	Ground gland
E	Massage bead
F	Power wire
G	Water level of lean
H	Power supply plug

3.2. Preparing for use

The device operates by positioning the feet in the designated area. Equipped with a temperature-controlled heating element at its base, the machine requires water for use, enabling warmth regulation while functioning. With approximately 400 massage balls, the machine offers adjustable massage intensity for comprehensive foot therapy.

Safety guidelines

To ensure safe operation, please adhere to the following precautions:

- Do not connect or disconnect the power plug while your feet are immersed in water.
- Always switch the machine to the off position “O” before plugging or unplugging the power cord.
- Do not submerge the entire machine in water or other liquids during cleaning or use.
- Sit while using the machine; do not stand on it or place objects inside during operation.
- Switch off the machine when not in use.
- Do not use the machine if the power cord or plug is damaged. If damaged or dropped, have it inspected or repaired at an authorized service centre.
- Avoid pulling the power cord when moving the machine.
- Supervise children near the device to prevent accidents.
- Do not use the machine while sleeping or if feeling unwell.
- Stop use immediately and consult a doctor if experiencing discomfort, pain, swelling, inflammation, or foot injury.

Packing

Ensure the power cord is unplugged from the socket before packing. Wrap the cord around the circular wire holder, wipe any moisture from the container, and store the machine in a dry, clean location.

Do not use the cord to lift or hang the massage machine.

3.3. Device use

This machine requires water to be added before operation. Before connecting to the power supply, ensure all switches are set to the off position “O”. Do not place your feet in the foot basin while the machine is connected to the mains. Sit comfortably, place your feet on the massage balls, and select your preferred function, such as heat function, massage function and heat and massage function.


NOTICE!


- Fill the basin with warm water, ensuring the water level does not exceed the marked depth line.

- This product is intended to maintain water temperature but cannot heat water.
- To empty the basin, pour the water into a bathroom drain or another suitable location, then clean and dry the basin thoroughly.


Massage functions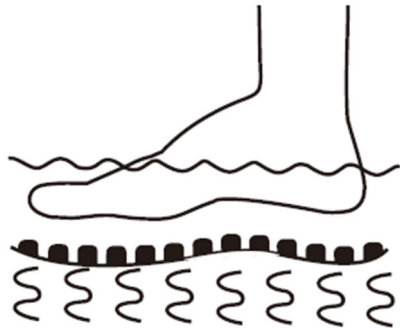


There is (not) water massaging

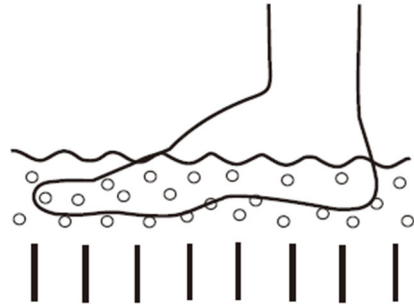Warm water vapor bubble massages *●●

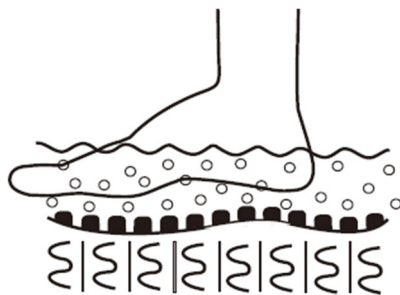Warm water vapor bubble massages *⋈●●

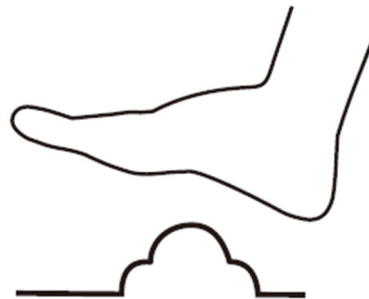Massage ball attachment

Only massage

Adjust the switch to the position “⋈” to begin your massage.

Heat massage (when using water) *⋈●●

To enjoy a warm bubble massage, pour warm water into the basin. The built-in temperature control heat wire helps maintain the water temperature, reducing heat loss over time.

Bubble heat bath (when using water) *●●

The ideal time for a bubble heat bath is after a heat massage, as it helps relax muscles more effectively.

If you want to target specific areas of your foot, use the large massage ball located in the centre of the machine for focused massage relief.

After completing your session, turn all switches to the off position “O”, remove your feet from the basin, and unplug the machine from the power source.

3.4. Cleaning and maintenance

- After use, clean the container portion with water. Do not immerse the machine in water. For overall cleaning, wipe the surface with a damp cloth and mild detergent.
- This product does not contain any user-serviceable parts. Do not open or attempt to repair it yourself.

DISPOSING OF USED DEVICES

Do not dispose of this device in municipal waste systems. Hand it over to an electric and electrical device recycling and collection point. Check the symbol on the product, instruction manual and packaging. The plastics used to construct the device can be recycled in accordance with their markings. By choosing to recycle you are making a significant contribution to the protection of our environment.

Contact local authorities for information on your local recycling facility.

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